# Foam Cutter

---

REFERENCE GUIDE

**CIVCO**

## INTENDED USE

The device is intended to cut foam for molds.

## CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

### WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of safety hazard control to co-workers and themselves. To avoid safety hazards, wear all recommended protective equipment and follow safety control policies established by your facility.*
- *Routinely check foam cutter and power cords for signs of deterioration. Do not use if device appears damaged.*
- *Consult your facilities manager before assembly and installation.*

## USING FOAM CUTTER

### WARNING

- *Voltage requirements: MT-900 is 115 VAC, 60 Hz; MT-900-EX is 230 VAC, 50 Hz.*
- *Wire and tension arm become hot while in use.*
- *Foam thickness to not exceed: 1" (2.54cm).*
- *Use only high-quality oncology grade foam as provided by CIVCO.*

1. Trace the shape representing tumor area onto foam.
2. Turn power switch off.
3. Plug foam cutter into appropriate GFCI (ground fault circuit interrupter) receptacle.
4. Turn power switch on to pre-heat cutting wire.
5. Cut out traced shape.
6. Allow wire to burn off excess foam.
7. Turn off foam cutter when not in use.

## INSTALLING CUTTING WIRE

### WARNING

- *Foam cutter is compatible with CIVCO cutting wires only.*

1. Turn power switch off and unplug foam cutter.
2. Wait for wire to cool.
3. Remove wire from slot in tension arm.
4. Turn foam cutter on its side to expose male wire connector and remove wire from connector.
5. Run new cutting wire loop through the hole on work surface.
6. Place connector through the center of the wire loop and pull opposite end of cutting wire through until bottom male connector is in plastic bushing groove.
7. Gently push down on tension arm and slip cutting wire into groove.
8. Rest aluminum pin in groove cut on tension arm.
9. Ensure lower loop is centered into groove and wire is not touching the sides.
10. Set foam cutter upright.

## MAINTENANCE

- NOTE:
- Inspect product for damage and general wear after use.
  - If device does not function per instructions, contact CIVCO.

REPROCESSING

1. Remove visible contaminants using a soft damp cloth.
2. Wipe dry with lint-free cloth or allow to air dry.

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COPYRIGHT © 2018 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.