IPPS™ Couchtop

for use with Accuray RoboCouch™

REFERENCE GUIDE

CIVCO

CE

INTENDED USE

The device is intended to support and aid in positioning a patient during radiologic and other medical procedures.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

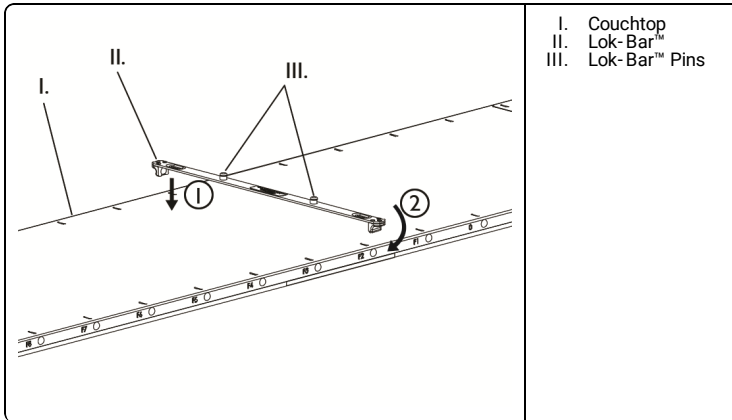
- Do not use if device appears damaged.
- Do not exceed a patient weight of 500 lbs (226.8 kg).
- Device is intended for use with the Accuray® system only. Please contact Accuray® for installation instructions.
- Ensure device is secure prior to use.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

USING Lok-Bar™**⚠ WARNING**

- Do not use to secure positioning devices.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

**REPROCESSING****⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Disinfect with a mild, non-abrasive cleanser.
2. Wipe dry with lint-free cloth or allow to air dry prior to reassembly and use.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2020 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. IPSS AND LOK-BAR ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.