Couchtop Panels

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INTENDED USE

The device is intended to support and aid in positioning a patient during radiologic and other medical procedures.

CAUTION

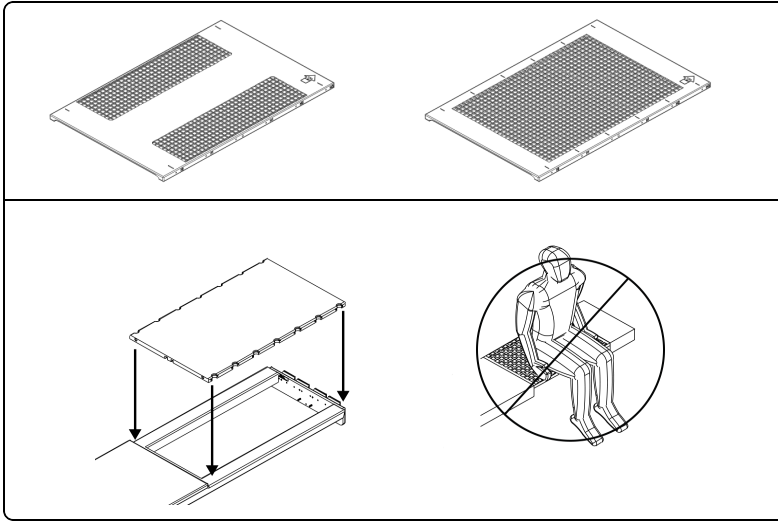
Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- Do not use if device appears damaged.
- Ensure device is secure prior to use.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- MTIL300512, MTIL301001, MTIL301002: Do not apply a load exceeding 400 lbs (181.4 kg).
- MTIL3103, MTIL3202: Do not apply a load exceeding 450 lbs (204 kg).
- MTZXT244069CS, MTZXT244069CF: Do not apply a load exceeding 250 lbs (113 kg).

INSTALLATION

1. Place panel onto couchtop.

2. Ensure that the orientation is correct and securely attached.

REPROCESSING

WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

STORAGE

NOTE: Store flat in a cool, dry place. Do not expose to direct sunlight, extreme cold, or excessive heat for prolonged periods of time.

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COPYRIGHT © 2019 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.