Foot Extension

INSTRUCTIONS FOR USE (IFU)

INTENDED USE

Device is to support and aid in patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

**WARNING**

- Do not use if device appears damaged.
- Do not allow patient to reposition themselves.
- Do not reposition device with patient on it.
- Do not apply treatment through device.
- Do not use Lok-Bar™ to secure positioning devices.
- Do not exceed 160lbs (72.7 kg).

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

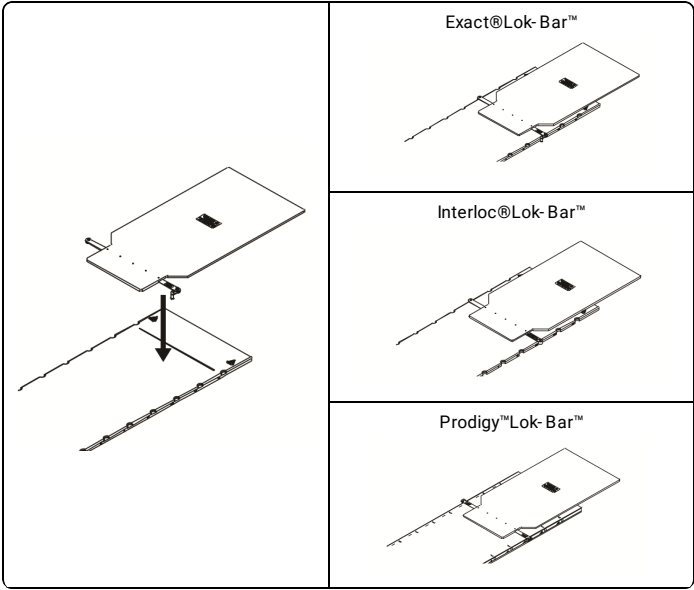
USING FOOT EXTENSION

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.

**WARNING**

- Do not extend inscribed line past end of table.

2. Position patient as dictated by setup protocol.

REPROCESSING

**WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Remove visible contaminants using a soft damp cloth.
2. Wipe dry with lint-free cloth or allow to air dry.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. LOK-BAR AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
**MEDTEC LLC**