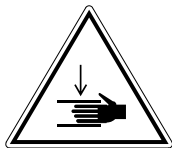
Uniframe® Baseplates

REFERENCE GUIDE

CIVCO
Radiotherapy™

CE

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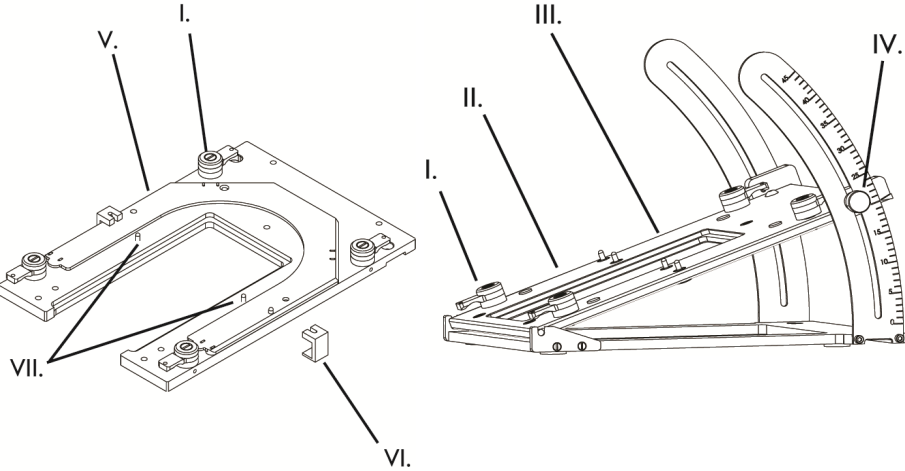
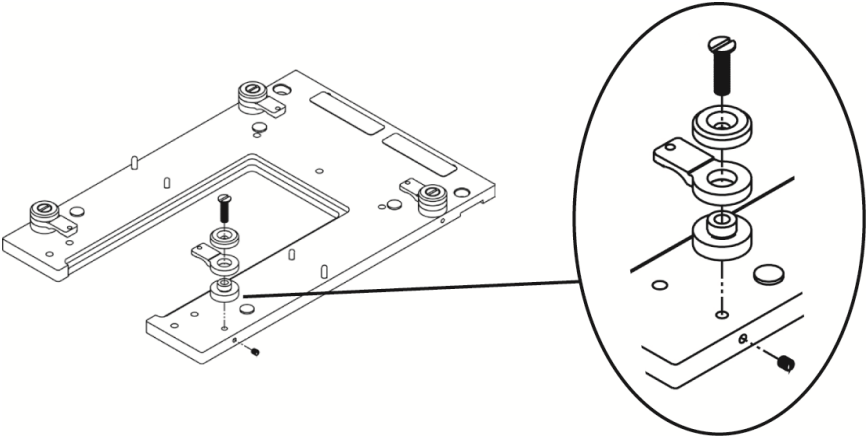
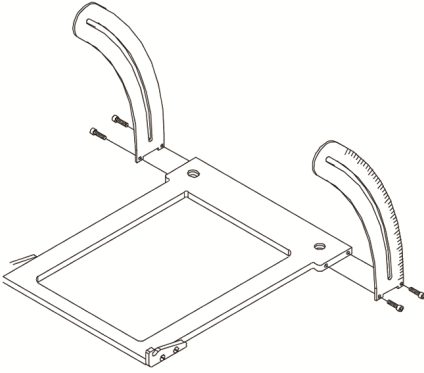
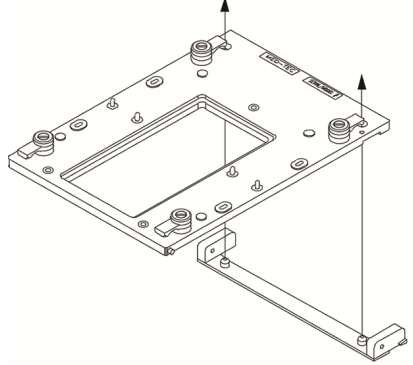
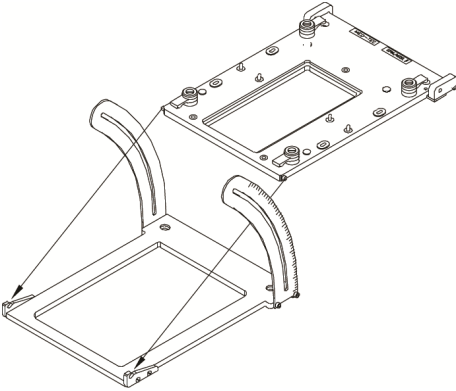
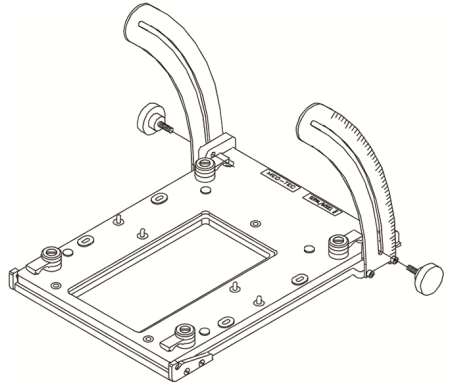
4.

INTENDED USE

Device is intended for patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not reposition device with patient on it.
- Do not use Lok-Bar™ to secure positioning devices.
- Ensure device is secure prior to use.

NOTE: • If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
• Baseplates require the use of a headrest (*refer to Headrest instruction*).

USING UNI-FRAME® BASEPLATE

1. Place Uni-frame® baseplate onto Lok-Bar™ pins (*refer to Lok-Bar™ instruction*).
2. Turn swivel clamps (4) to point outward.
3. Place shim (*provided*) onto baseplate pins, if desired.

NOTE: Only use shim when first forming thermoplastic mask.

4. Tilting Uni-frame® baseplates:
 - a. Adjust upper plate to desired angle (0-45°).
 - b. Tighten lock-knobs to secure angle.
5. Place desired headrest onto baseplate.
6. Position patient onto baseplate for thermoplastic setup (*refer to thermoplastic instruction*).
7. Turn swivel clamps (4) inward to secure thermoplastic.
8. Attach C-clamps to Uni-frame® baseplate and thermoplastic frame.

REPROCESSING

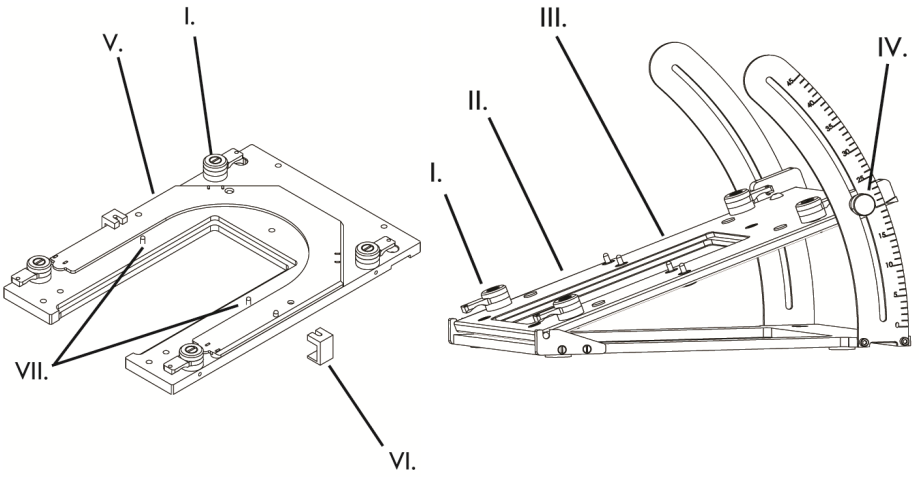
WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Clean device with a mild, non-abrasive cleaner as needed.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. UNI-FRAME IS A REGISTERED TRADEMARK OF MEDTEC LLC. LOK-BAR IS A TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.