





Wing Board™

INSTRUCTIONS FOR USE (IFU)

CQ MEDICAL™

CE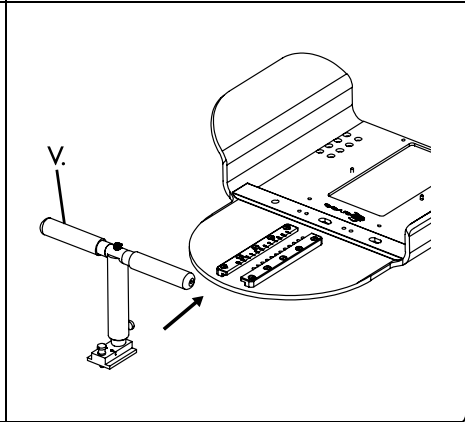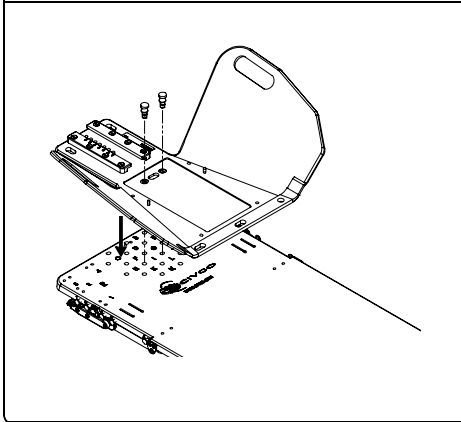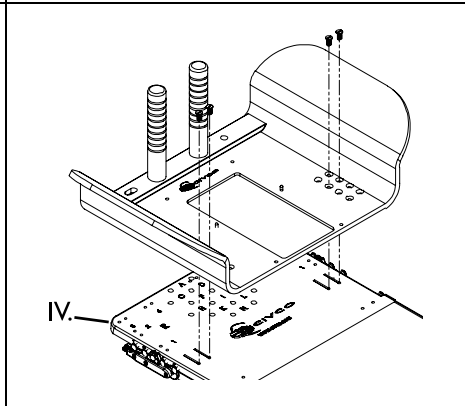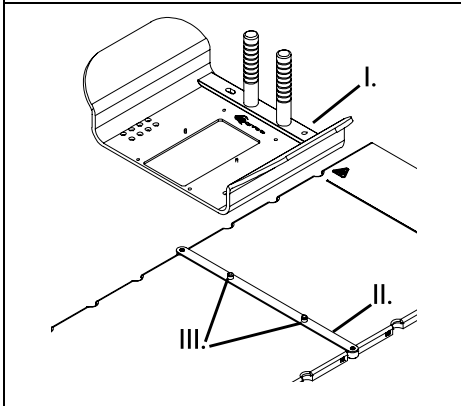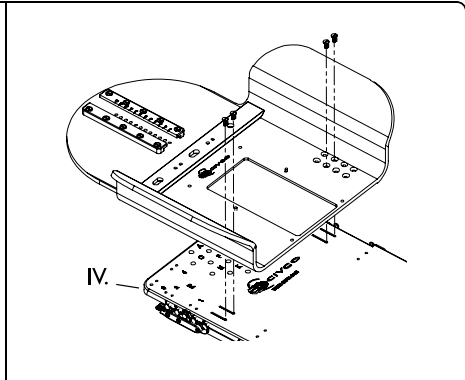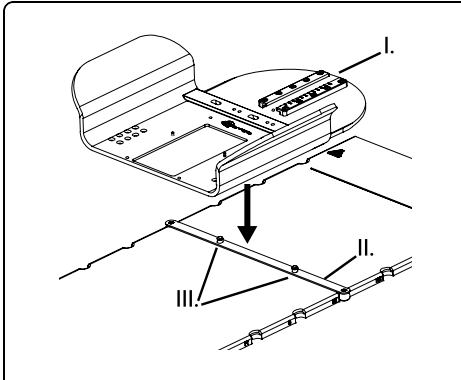

- I. Wing Board™
- II. Lok-Bar™
- III. Lok-Bar™ Pins

- IV. Breast Board
- V. Handle

INTENDED USE

- The device is intended to support and aid in positioning a patient during radiologic and other medical procedures.
- Devices are used to aid in the support and positioning of patients during an MRI.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

- Do not use if device appears damaged.
- Do not allow patient to reposition themselves.
- Do not reposition device with patient on it.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at www.CQmedical.com.
- Verify patient position with completed setup sheet prior to treatment.
- Do not use Lok-Bar™ to secure positioning devices.
- Do not exceed 200 lbs (91 kg) on wings.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Ensure device is secure prior to use.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

MRI SAFETY INFORMATION

MR Safe

- Devices are MR Safe.
- MR image quality may be compromised if the area of interest is relatively close to the inferior bottom edge of the Wing Board™. Optimize MR imaging parameters prior to treating patient. Image artifact extends approximately 5mm from the inferior bottom edge of the Wing Board™ when scanned in nonclinical testing using a gradient echo sequence in a 3.0T Siemens Skyra MR system with built-in body coil.

USING WING BOARD™

1. Place Wing Board™ onto Lok-Bar™ pins (refer to Lok-Bar™ instruction) or attach onto breast board (refer to Breast Board instruction).
2. Slide desired handle onto Wing Board™ and engage locating pin to secure.
3. Place desired headrest into recessed area of Wing Board™ or onto Wing Board™ pins (refer to Headrest instruction).
4. If desired, place Vac-Lok™ cushion onto Wing Board™ (refer to Vac-Lok™ instruction).
5. Position patient onto Wing Board™ as dictated by setup protocol.
6. Instruct patient to grasp handle.


REPROCESSING**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Clean device with a mild, non-abrasive cleaner as needed.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

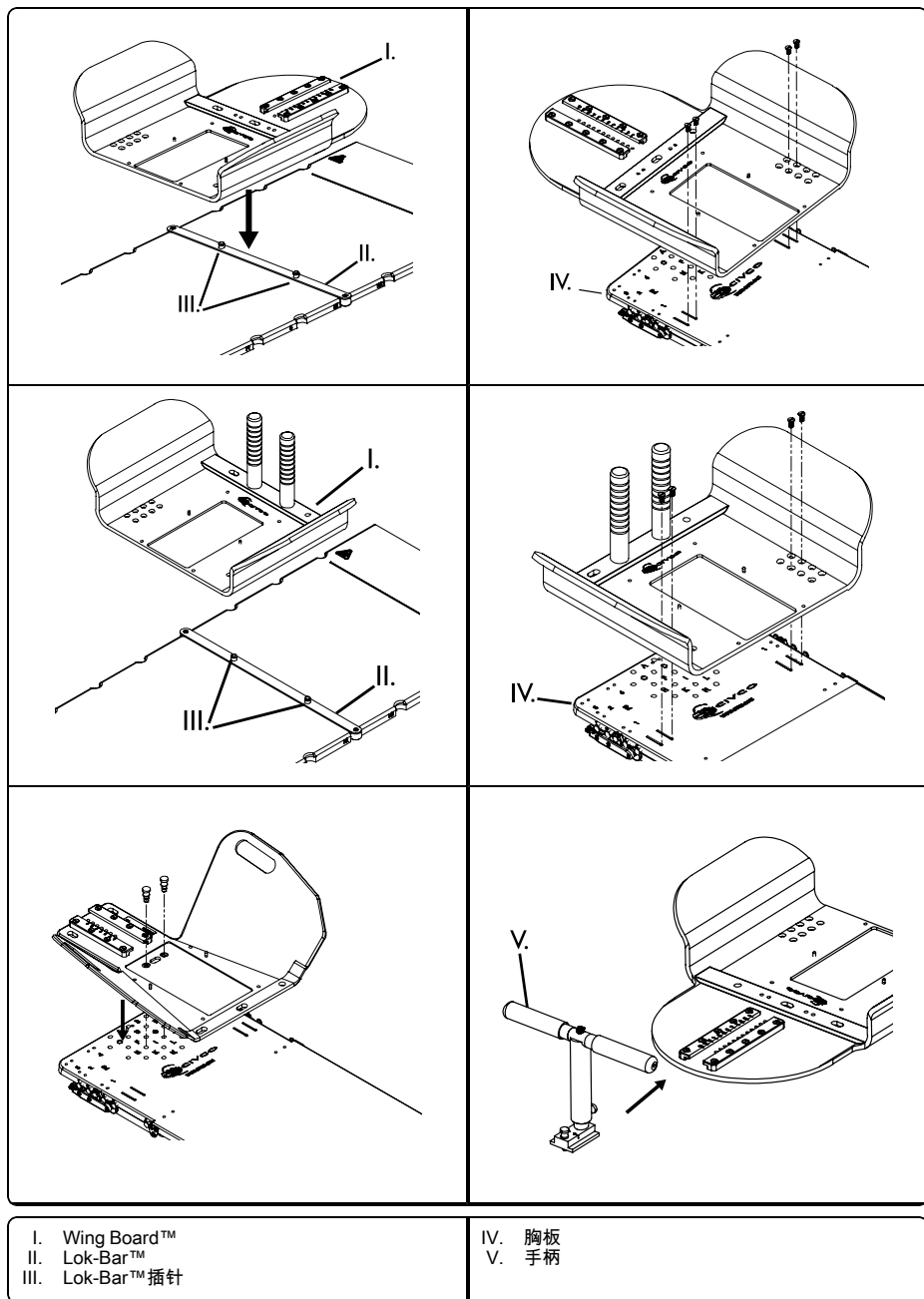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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. LOK-BAR, VAC-LOK AND WING BOARD ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC