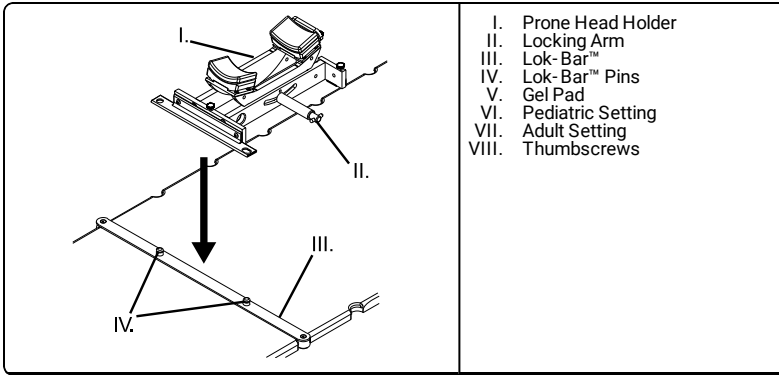
Prone Head Holder

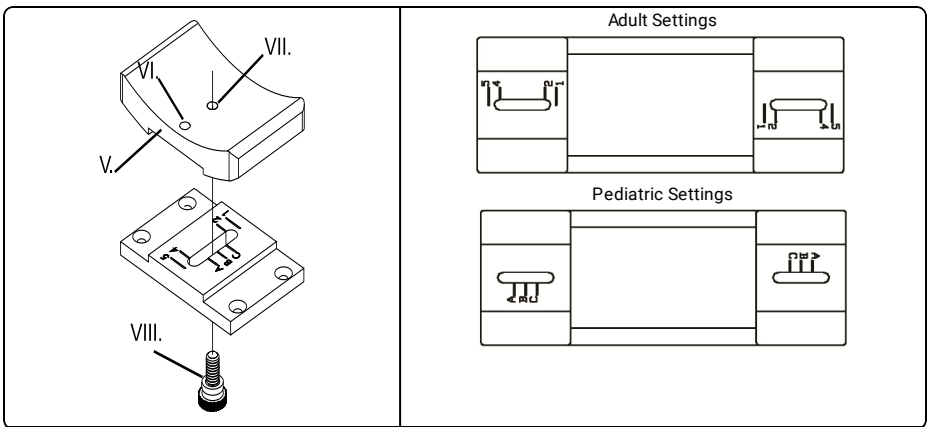
REFERENCE GUIDE

CIVCO


CE

- I. Prone Head Holder
- II. Locking Arm
- III. Lok-Bar™
- IV. Lok-Bar™ Pins
- V. Gel Pad
- VI. Pediatric Setting
- VII. Adult Setting
- VIII. Thumbscrews


ATTACHING VAC-LOK™ CUSHION

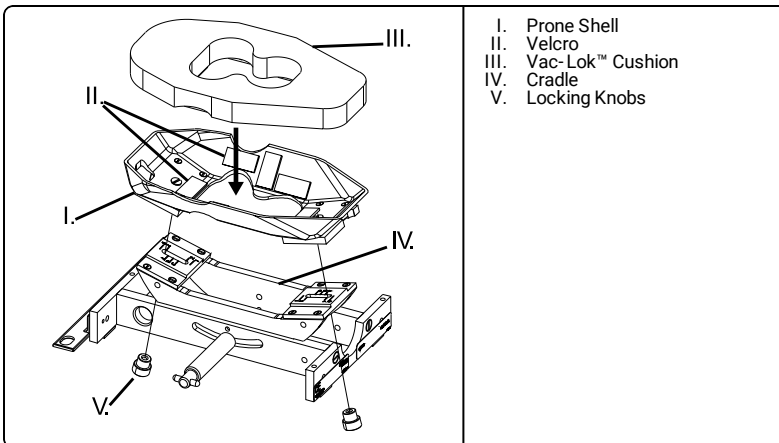
- I. Prone Shell
- II. Velcro
- III. Vac-Lok™ Cushion
- IV. Cradle
- V. Locking Knobs

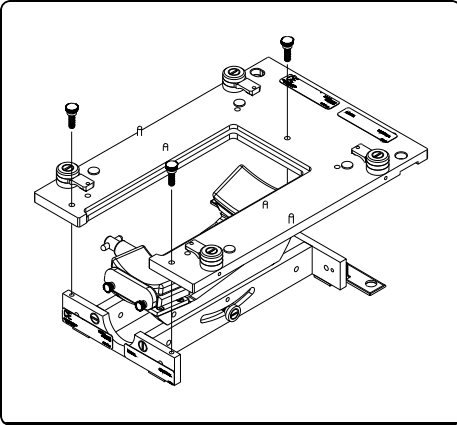ATTACHING UNI-FRAME® BASEPLATE

ATTACHING TWO-PIN ADAPTER PLATE

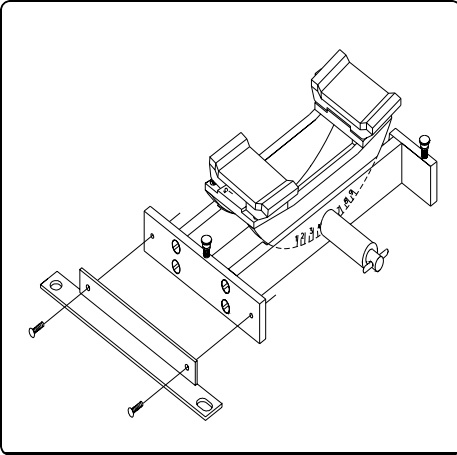INTENDED USE

The device is intended to aid in patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

- Do not use if device appears damaged.
- Do not reposition device with patient on it.
- Do not allow patient to reposition themselves.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at WWW.CIVCORT.COM.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not use Lok-Bar™ to secure positioning devices.
- Verify patient position with completed setup sheet prior to treatment.
- Ensure device is secure prior to use.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

USING PRONE HEAD HOLDER

1. Place prone head holder onto tabletop or Lok-Bar™ pins as desired (*Refer to Lok-Bar™ instruction*).
2. Loosen thumbscrew to adjust gel pad position.
3. Loosen locking arm to adjust cradle rotation on prone head holder as needed.
4. Position patient as dictated by setup protocol.

USING PRONE HEAD HOLDER WITH VAC-LOK™ CUSHION

NOTE:

- Vac-Lok™ cushion should be flaccid prior to positioning patient on it.
- Ensure integrity of Vac-Lok™ cushion is not compromised during daily handling.
- Place female valve connector onto cushion valve to let air into Vac-Lok™ cushion without using pump.
- Store Vac-Lok™ cushion with valve cover on.

1. Remove gel pads using thumbscrews.
2. Attach prone shell to cradle using locking knobs.
3. Place Vac-Lok™ onto prone shell Velcro®.
4. Smooth Vac-Lok™ cushion to evenly disperse beads.
5. Attach vacuum hose to cushion valve.
6. Position patient onto Vac-Lok™ cushion. Distribute beads around regions to be immobilized.
7. Evacuate air from Vac-Lok™ cushion while forming around patient until desired level of vacuum is achieved.

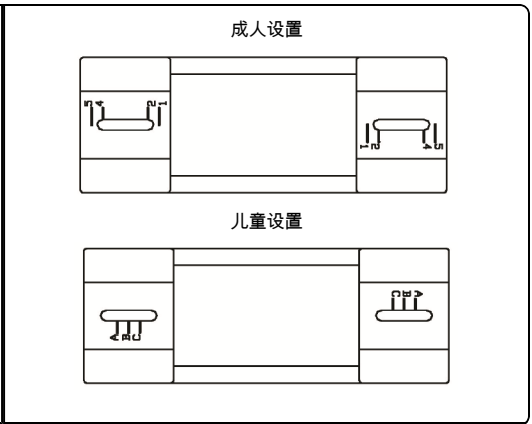
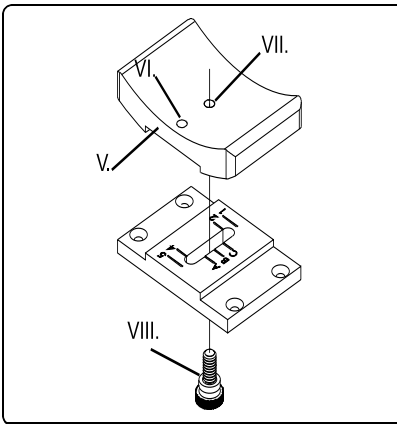
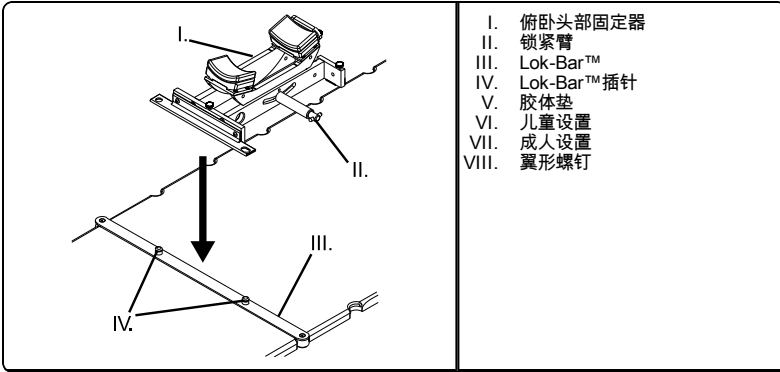
REPROCESSING**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.


1. Remove visible contaminants using a soft damp cloth.
2. Wipe dry with lint-free cloth or allow to air dry.

MAINTENANCE

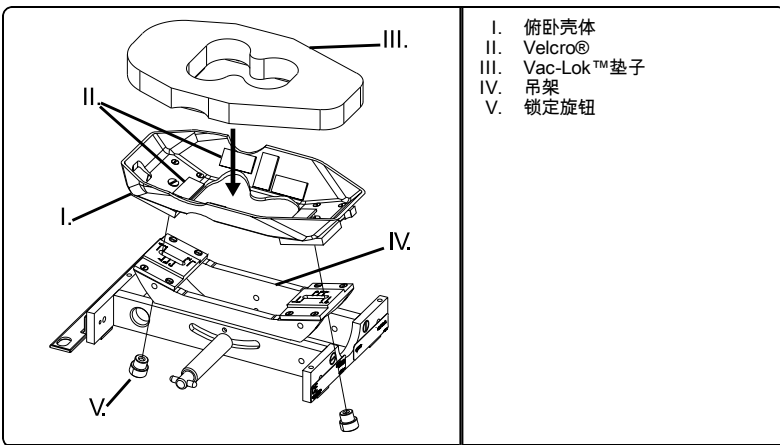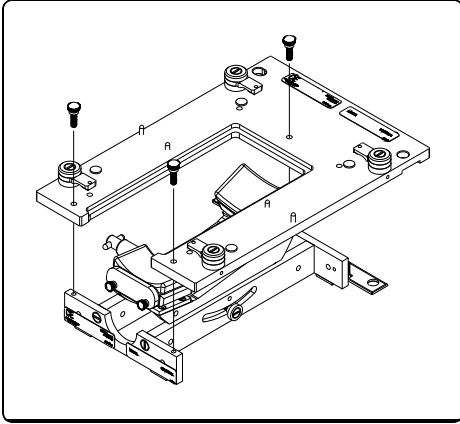
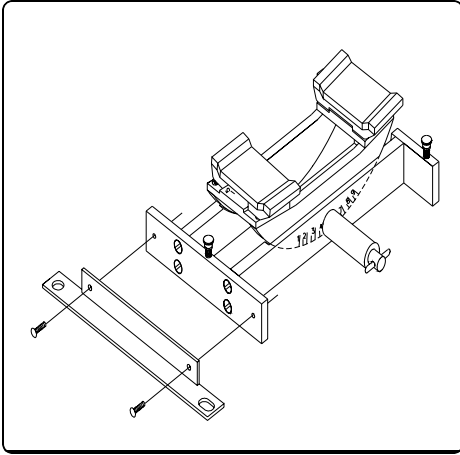
NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2021 ALL RIGHTS RESERVED. CIVCO AND UNIFRAME ARE REGISTERED TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. LOK-BAR AND VAC-LOK ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.