# Eye Shields

---

REFERENCE GUIDE

**CIVCO**  
Radiotherapy™

**INTENDED USE**

The device is intended to protect the eye from unnecessary radiation exposure.

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.

 **WARNING**

- Do not use if device appears damaged.
- Users of this product have an obligation and responsibility to provide the highest degree of safety hazard control to co-workers and themselves. To avoid safety hazards, wear all recommended protective equipment and follow safety control policies established by your facility.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.

**USING EYE SHIELDS** **WARNING**

- Device is packaged non-sterile and is reusable. To avoid possible patient infection, ensure device is properly cleaned and sterilized before initial use.
- Device should be cleaned and stored in a clean container/area between uses on an individual patient. In some cases it may be necessary to sterilize device between uses on an individual patient. Refer to standard hospital infection control guidelines for determination of sterilization need between uses on an individual patient.
- Device must be sterilized and stored in a sterilized container/area between uses on different patients.
- Apply sterile saline solution between eye and eye shield during removal to reduce suction against eyeball.

1. Eye shield is placed under eyelid.

**REPROCESSING** **WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.
- Do not use alcohol, betadine or iodine to clean or sterilize eye shields.
- Destroy device after use by a patient with known Creutzfeldt-Jakob Disease. Do not attempt to reprocess.
- Do not autoclave.
- Do not use low temperature vaporized hydrogen peroxide gas to sterilize eye shields. Using this method is known to cause a cytotoxic effect.

**CLEANING**

1. Clean under hot water with hospital grade soap and a nonmetallic brush.
2. Rinse with distilled water and dry.

**STERILIZING**

NOTE: Gas Sterilization: Set to 130°F (54.4°C). Warm eye shields for one hour. Condition for 30 minutes. Inject 100% Ethylene Oxide and hold for 75 minutes. Mechanically aerate for 20 hours.

**MAINTENANCE**

NOTE: Inspect device prior to use for damage such as cracks or breakage. If damage is evident, discontinue use.

**STORAGE** **WARNING**

- Do not store eye shields in sterilizing solution.

1. Store eye shields in protective container.

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COPYRIGHT © 2022. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.