# IPPS™ Couchtop

*for use with Accuray RoboCouch™*

INSTRUCTIONS FOR USE (IFU)

---

**INTENDED USE**

The device is intended to aid in patient positioning.

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.


**⚠ WARNING**

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not exceed a patient weight of 500 lbs (226.8 kg).
- Device is intended for use with the Accuray® system only. Please contact Accuray® for installation instructions.
- Ensure all fixtures are locked prior to treatment.
- Do not reposition seat device with patient on it.
- Ensure device is secure prior to use.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

**USING SEAT**


1. Adjust angle of seat by utilizing small pin (A).
2. Adjust position of seat by utilizing large pin (B).
3. Place cushions as desired.
4. Extend foot extension by pulling on handle.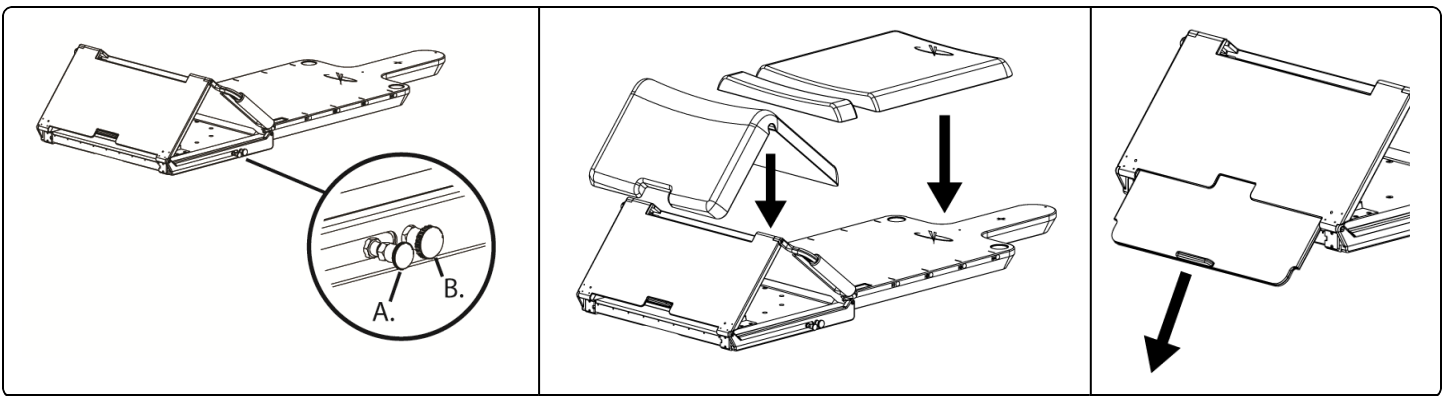

**USING UNI-FRAME® BASEPLATE**

1. Attach baseplate to couchtop using screw (provided).
2. Adjust baseplate to desired longitudinal position.
3. Turn swivel clamps (4) to point outward.
4. Place shim (provided) onto baseplate pins, if desired.
5. Place desired headrest onto baseplate.
6. Position patient onto baseplate for thermoplastic setup (refer to thermoplastic instruction).
7. Turn swivel clamps (4) inward to secure thermoplastic.


NOTE: Only use shim when first forming thermoplastic mask.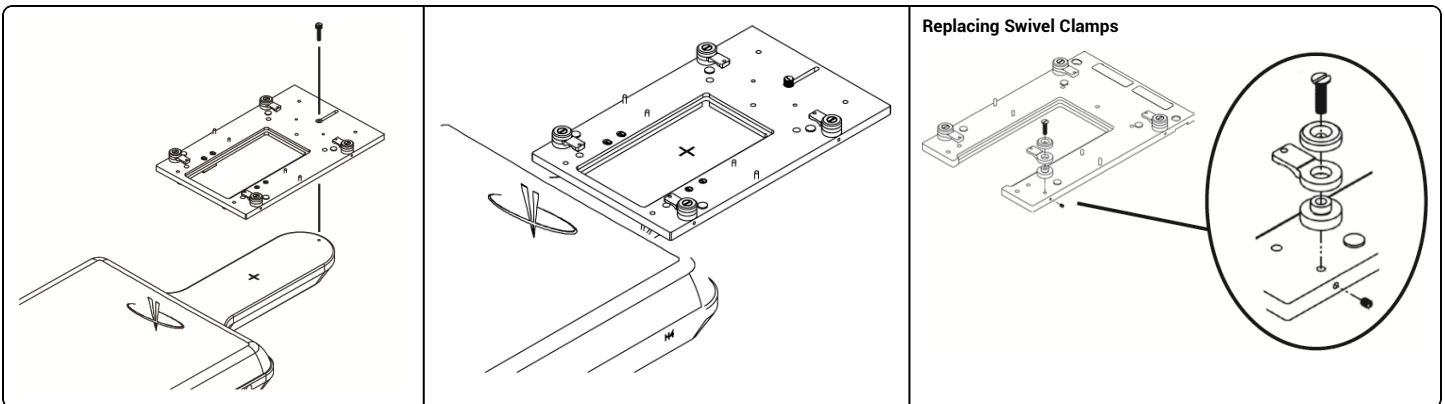

USING Lok-Bar™**⚠ WARNING**

- Do not use Lok-Bar™ to secure positioning devices.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

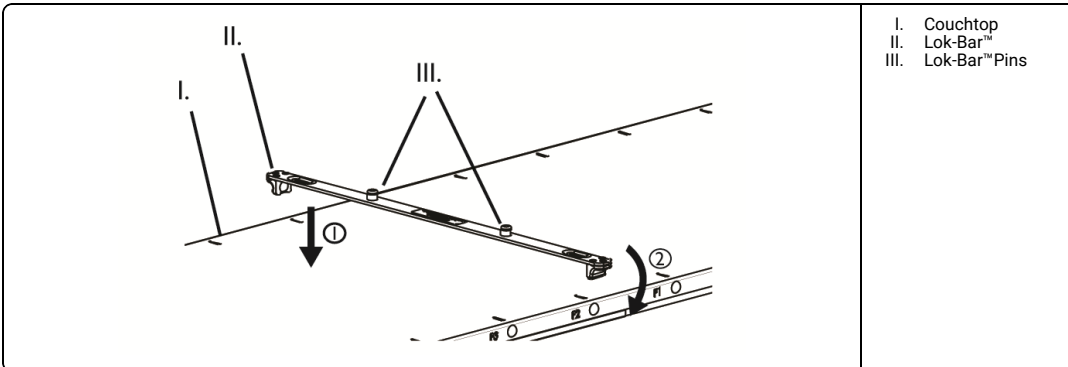REPROCESSING**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. IPPS AND LOK-BAR ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC