IPPS™ Overlay

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INTENDED USE

The device is intended to aid in patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

MRI SAFETY INFORMATION

- Device is safe for use within MR environment.

MR Safe

⚠ WARNING


- Do not use if device appears damaged.
- Ensure accessories are securely attached in correct position prior to treatment.
- Users of this product have an obligation and responsibility to provide the highest degree of safety hazard control to co-workers and themselves. To avoid safety hazards, wear all recommended protective equipment and follow safety control policies established by your facility.
- Do not exceed a patient weight of 440 lbs (200 kg).
- Ensure device is secure prior to use.

ASSEMBLING COMPONENTS


1. Place template (provided) at head of overlay and slide template to right hand side and towards foot end of table.
2. Mark area for button and remove paint with abrasive paper (provided).
3. Adhere button with adhesive (provided).

NOTE: Use minimum amount of adhesive needed to cover entire button surface.

4. Remove excess adhesive from around button using a wet cloth.

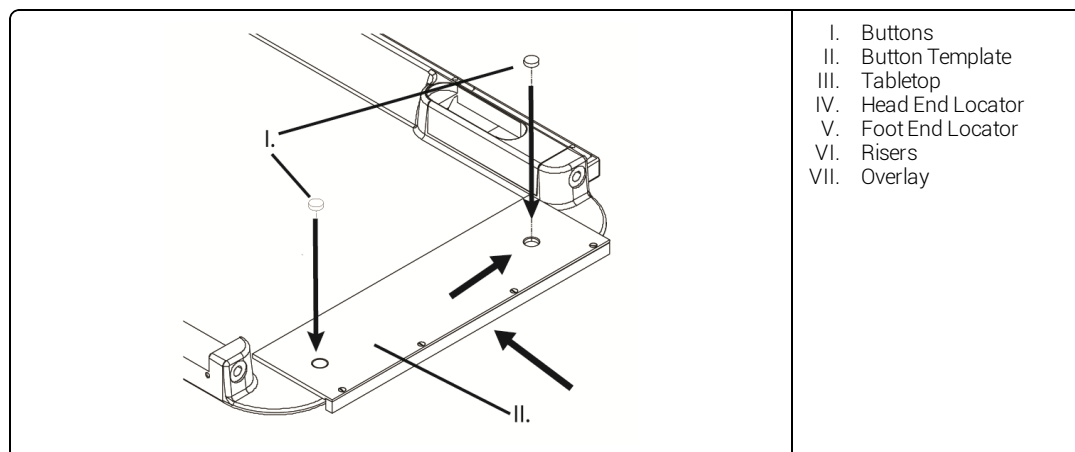
5. Wait a minimum of 15 minutes before proceeding to next step.
6. Place head end locator on buttons at head end of table.
7. Place foot end locator at foot end and place overlay on locators.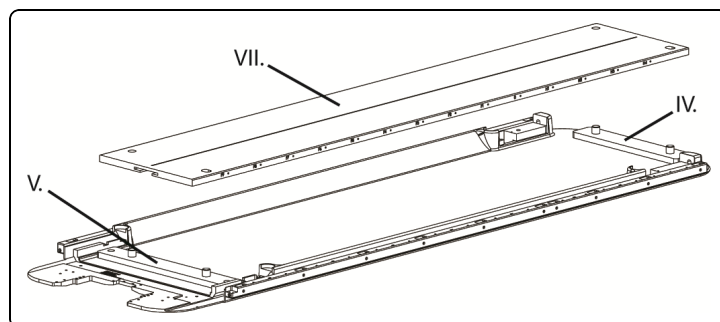

8. Position foot end locator so overlay is centered on table.
9. Mark area for buttons through foot end locator.

10. Remove overlay from table.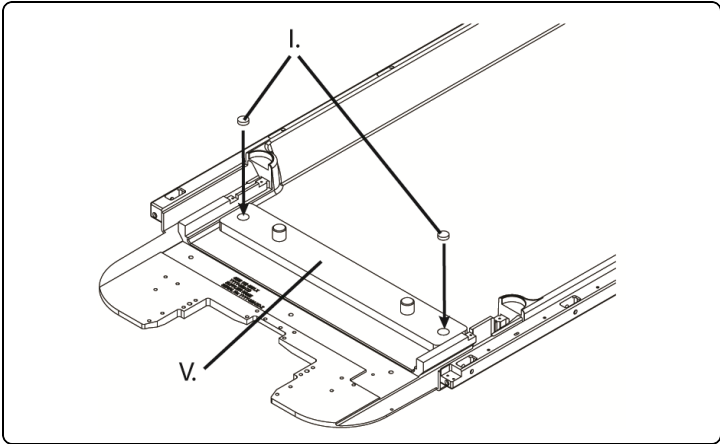

11. Adhere button with adhesive.


12. Remove excess adhesive from around button using wet cloth.

NOTE: Button Template can be removed and stored after initial installation of buttons.


ATTACHING OVERLAY

1. Place locators (2) onto buttons.

NOTE: Place risers onto tabletop buttons to adjust height.

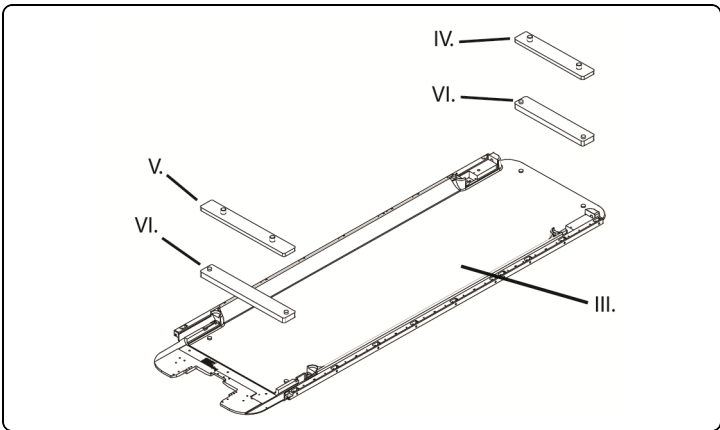
2. Slide adjustable supports (2) into overlay and place at desired longitudinal position.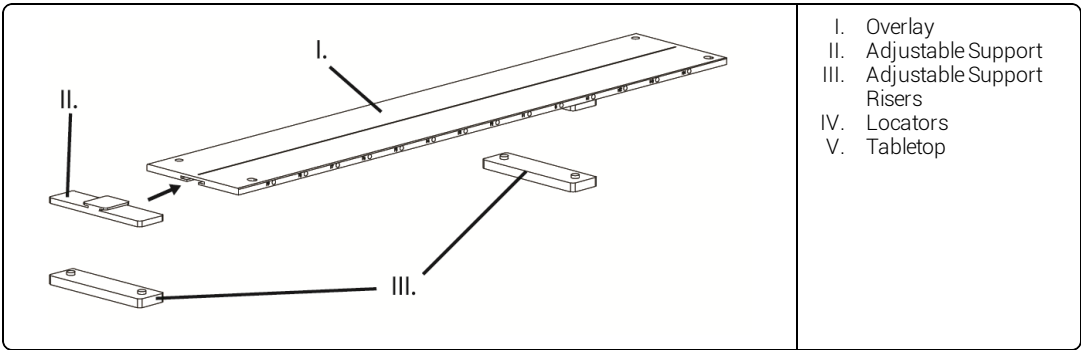

3. Place overlay onto locators or adjustable support risers (*if applicable*).

- NOTE:
- Overlay height: 18.5mm above tabletop
 - Overlay height with risers: 47mm above tabletop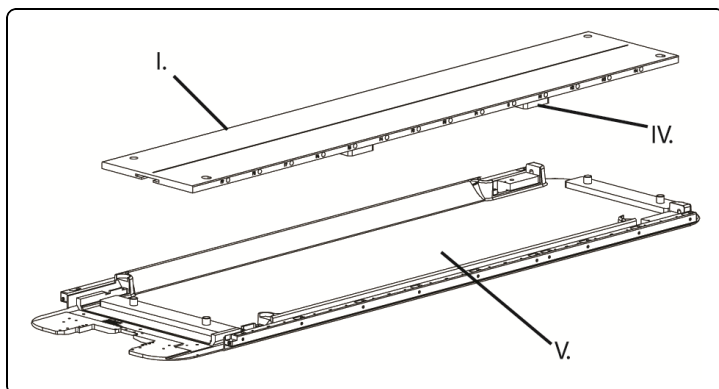


USING Lok-Bar™

⚠ WARNING

- Do not use Lok-Bar™ to secure positioning devices.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

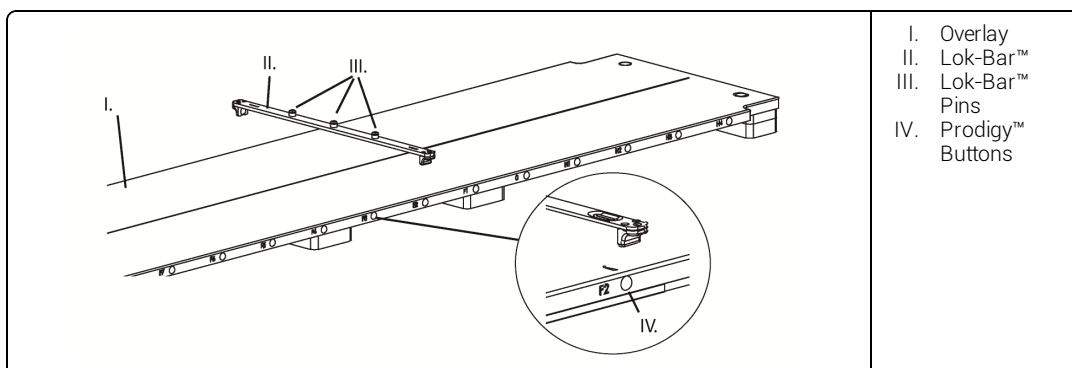- I. Overlay
- II. Lok-Bar™
- III. Lok-Bar™ Pins
- IV. Prodigy™ Buttons

REPROCESSING

⚠ WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2019 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. IPPS, LOK-BAR, AND PRODIGY ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.