Bolus

REFERENCE GUIDE

CIVCO
Radiotherapy™

INTENDED USE

Bolus is intended as a tissue equivalent gel pad.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- *Verify all angles of treatment and attenuation characteristics prior to treating patients.*

USING BOLUS

WARNING

- *Do not use if device appears damaged.*

1. Place device on intended treatment area.
2. Form device to patient skin.

REPROCESSING

WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *Do not autoclave or gas sterilize device.*
- *Do not use alcohol, petroleum, or mineral based substances with device, or strong undiluted disinfectants.*

NOTE:

- If device becomes contaminated or can no longer be cleaned, discontinue use and replace.
- Clean device between each use.

1. To remove visual contamination, clean Bolus using mild soap and water; 1:10 dilution of bleach- to-water or other solutions approved by your facility, not containing alcohol or any other strong, undiluted disinfectants.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

STORAGE

WARNING

- *Store device on clean flat surface to prevent tears/punctures.*

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COPYRIGHT © 2022. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.