# Belly Boards

Contoura™ | Carbon Fiber

---

INSTRUCTIONS FOR USE (IFU)

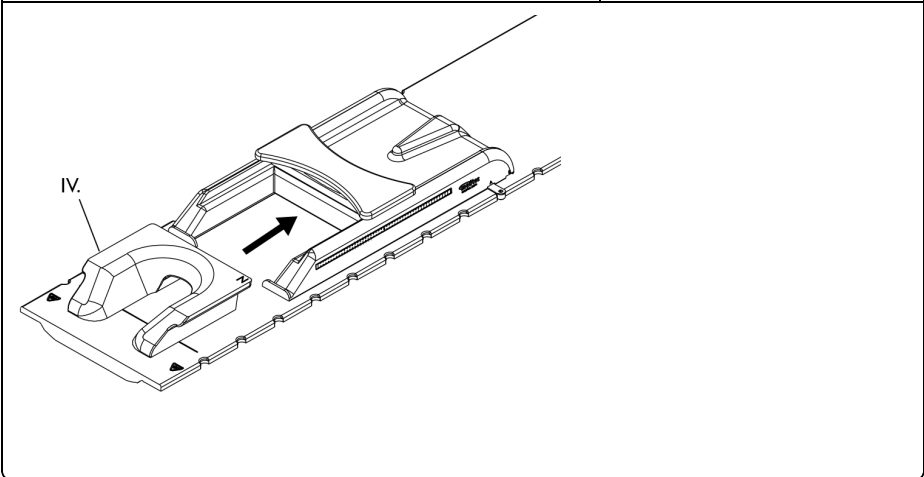
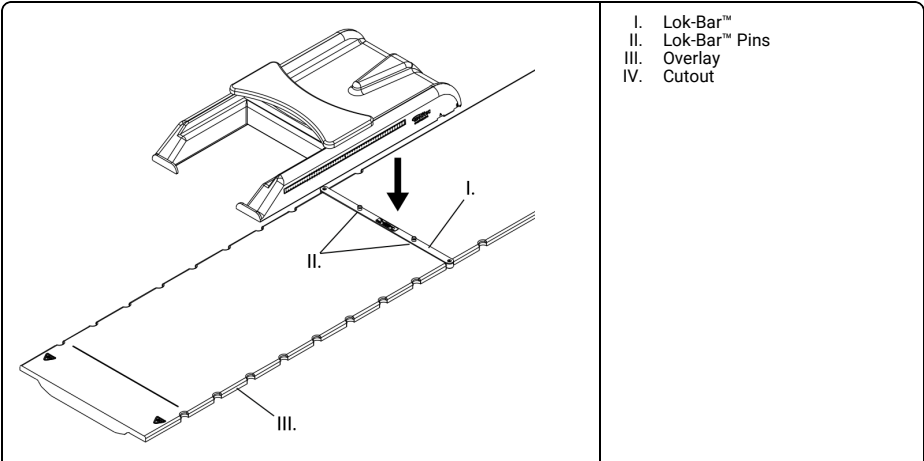
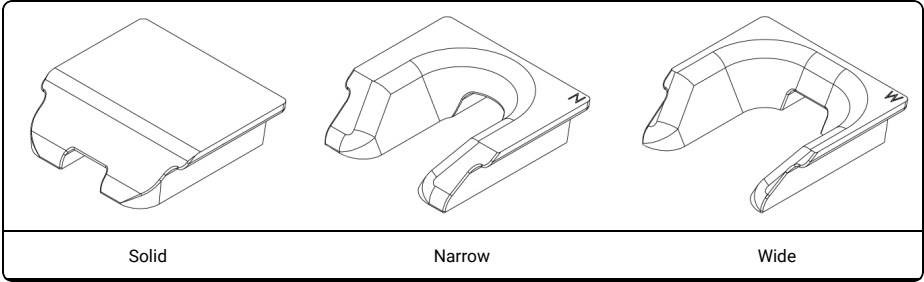
**CQ MEDICAL™**




CE

CONTOURA™ BELLY BOARD

**CARBON FIBER BELLY BOARD**

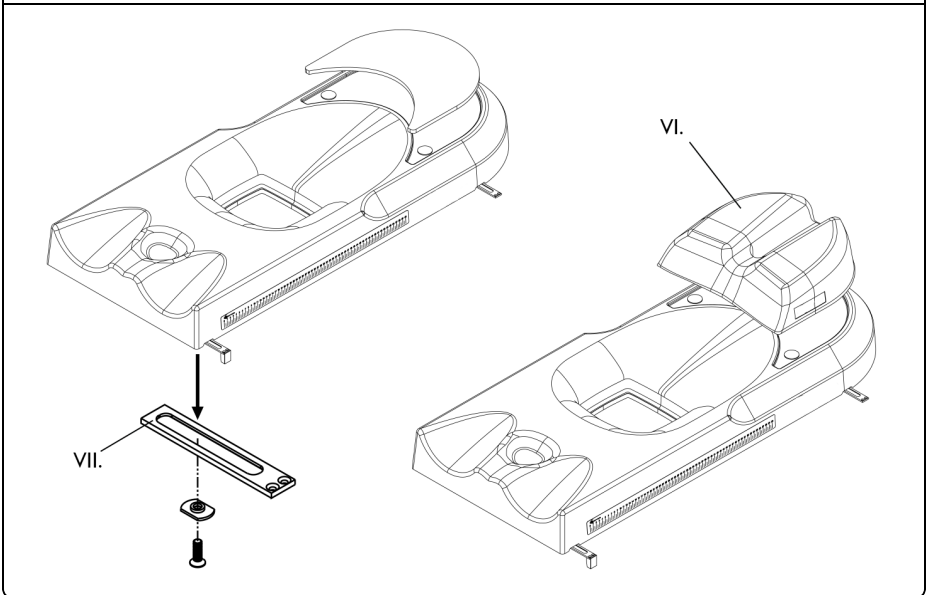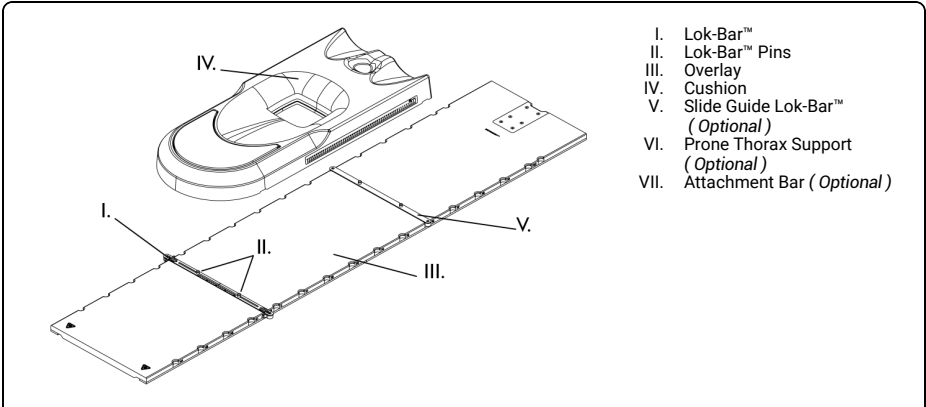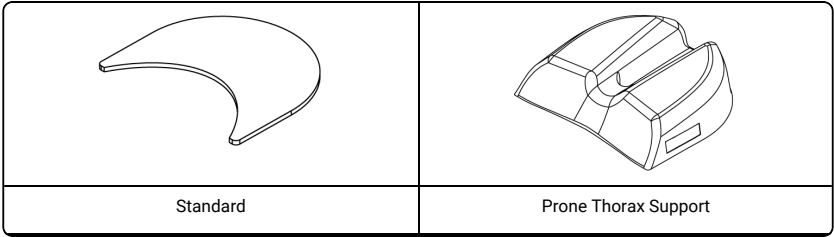**INTENDED USE**

Device is intended for patient positioning.

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.

**⚠ WARNING**

- Do not use if device appears damaged.
- Do not extend device beyond tabletop with patient in place.
- Do not use for patient weight support while mounting or dismounting tabletop.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not reposition device with patient on it.
- Ensure patient mounts and dismounts board from foot end.

**NOTE:** If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

**USING CONTOURA™ BELLY BOARD****⚠ WARNING**

- Do not use Lok-Bar™ to secure positioning devices.
- Do not exceed a maximum patient weight of 400 lbs (182 kg).
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at [www.CQmedical.com](http://www.CQmedical.com).

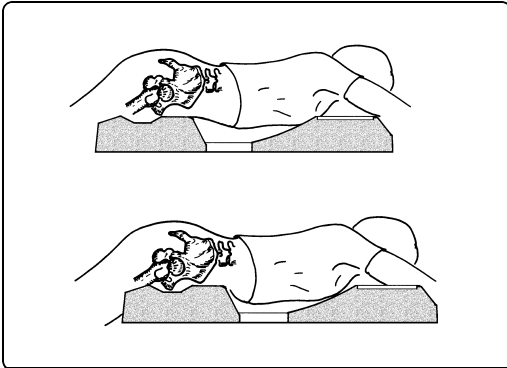
1. Place Contoura™ Belly Board onto Lok-Bar™ pins (refer to Lok-Bar™ instruction).
2. Insert appropriate cutout.

**USING CARBON FIBER BELLY BOARD****⚠ WARNING**

- Do not use Lok-Bar™ to secure positioning devices.
- Do not exceed a maximum patient weight of 400 lbs (182 kg).

**NOTE:** Slide Guide Lok-Bar™ must be used if more than one Lok-Bar™ is preferred.

1. Place Carbon Fiber Belly Board onto Lok-Bar™ pins (refer to Lok-Bar™ instruction).
2. Place desired cushion on superior end of belly board.
3. When positioning patient, start slightly superior and move in an inferior direction to final position.

**NOTE:** Carbon Fiber Belly Board has lead wires to provide cross-hairs for imaging purposes.

REPROCESSING**WARNING**


- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. Clean device with a mild, non-abrasive cleaner as needed.
2. Store cushions on a flat surface. Do not stack.


MAINTENANCE

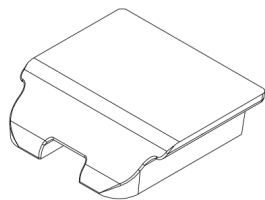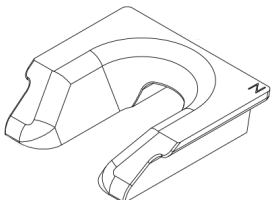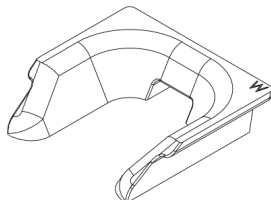
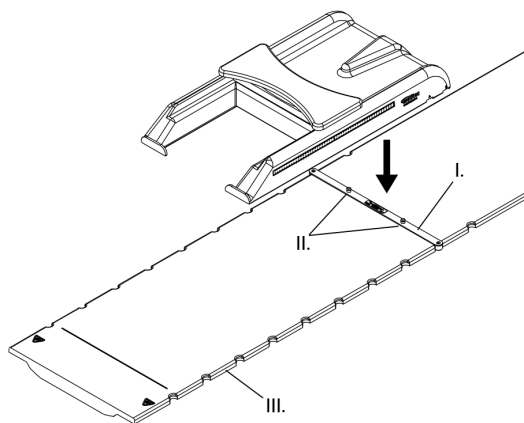
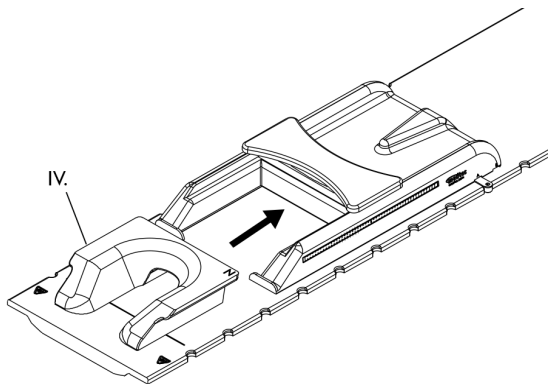
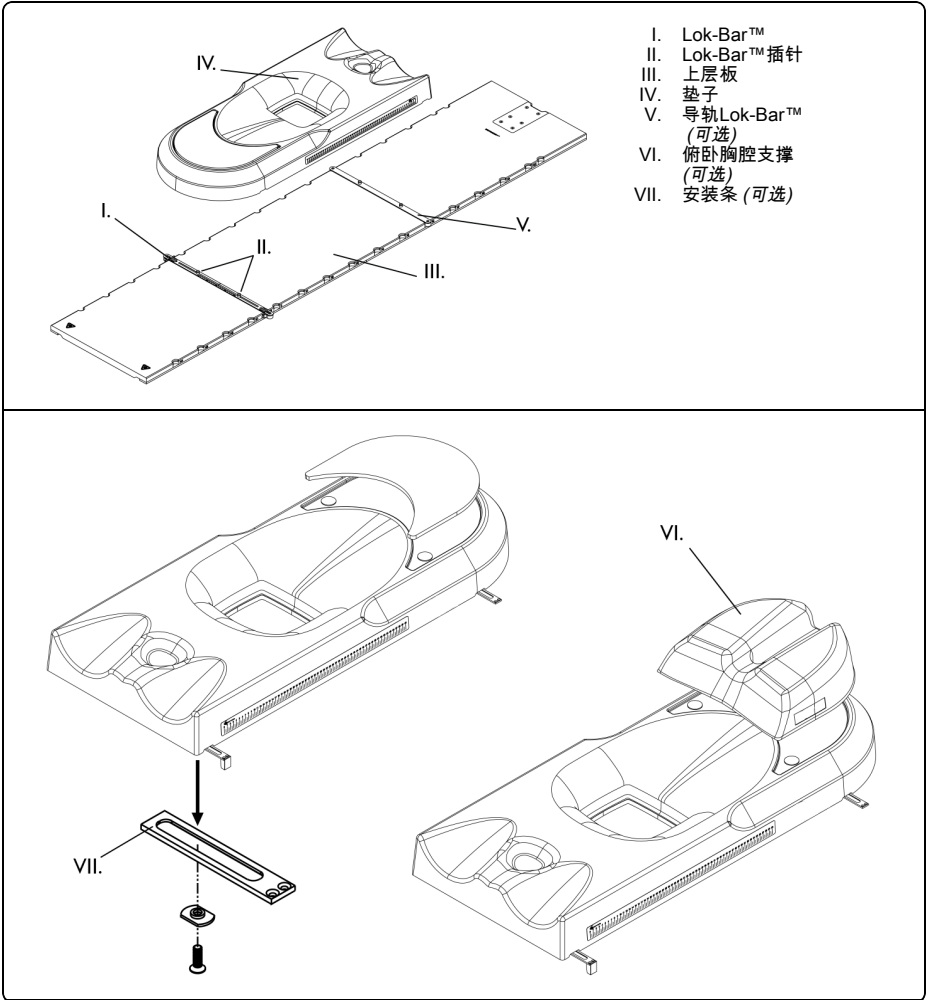
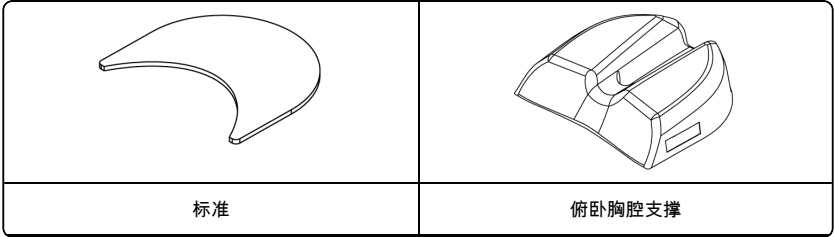
NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. CONTOURA AND LOK-BAR ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
**MEDTEC LLC**