Helping Hands™

REFERENCE GUIDE

CIVCO
Radiotherapy™

CE

INTENDED USE

Device is intended for patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- *Do not use if device appears damaged.*

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

USING HELPING HANDS™

1. Place Helping Hands™ onto tabletop.
2. Place Vac-Lok™ cushion onto tabletop, between Helping Hands™.
3. Push Helping Hands™ inward while deflating the Vac-Lok™ bag.
4. Remove Helping Hands™ after bag has been fully deflated.

REPROCESSING

WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2023 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. HELPING HANDS AND VAC-LOK ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.