# IPPS™ Overlay

*An indexed patient positioning system for use with GE Healthcare*

---

REFERENCE GUIDE

**CIVCO**  
Radiotherapy™

CE

**INTENDED USE**

The device is intended to aid in patient positioning.

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.

**⚠ WARNING**

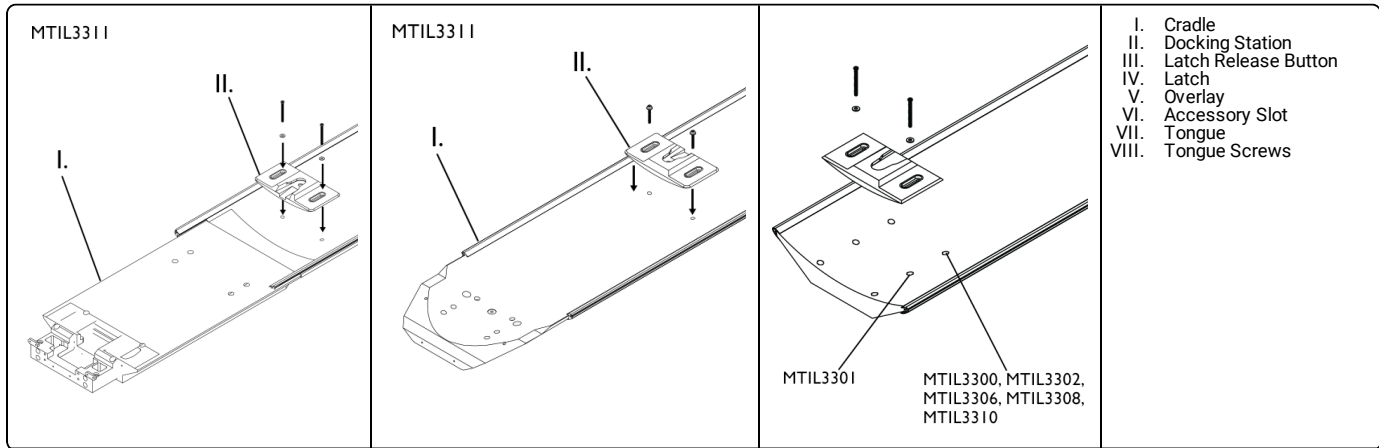
- Do not use if device appears damaged.
- Overlay weight: 40 lbs (18.1 kg). For total table load rating, including accessories, refer to your system's manual.
- Ensure device is secure prior to use.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

**ATTACHING DOCKING STATION TO CRADLE**


- NOTE:
- MT-IL3311 Tools Required: 5/32" hex wrench, M5 hex wrench, flat head screwdriver and level (*not provided*).
  - MT-IL3300, MT-IL3301, MT-IL3302, MT-IL3306, MT-IL3308, MT-IL3310 Tools Required: 5/32" hex wrench, 3/16" hex wrench, flat head screwdriver and level (*not provided*).

1. Remove all accessories from cradle.
2. Fasten docking station loosely using screws, docking bushings, and washers (*provided*).

**ATTACHING OVERLAY**

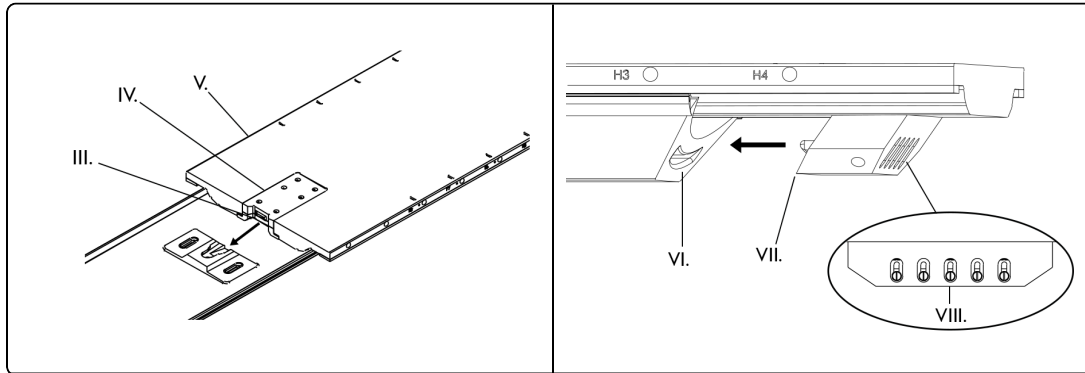1. Loosen tongue screws with screwdriver.
2. Slide overlay into docking station, ensure tongue fits properly into accessory slot.
3. Ensure latch is locked into docking station.


4. Tighten tongue screws to lock tongue into place.
5. Note position of docking station. Remove overlay with latch release button.
6. Tighten docking station with M5 hex wrench (MT-IL3311) or 3/16" hex wrench (MT-IL3300, MT-IL3301, MT-IL3302, MT-IL3306, MT-IL3308, MT-IL3310).

NOTE: Ensure docking station location allows for easy installation and removal of overlay.

7. Attach overlay to docking station.
8. Ensure overlay is level to within 1° laterally and longitudinally.

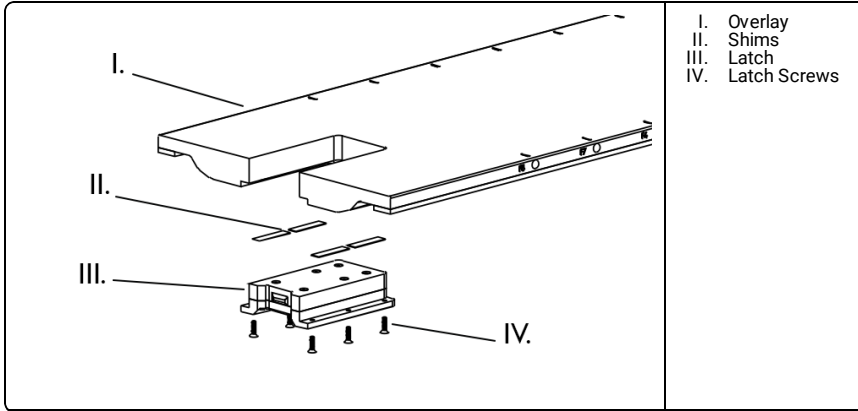
NOTE: If vertical height adjustment is necessary: remove latch and adjust by adding or removing shims.

**USING Lok-Bar™**

**WARNING**  
 Do not use Lok-Bar™ to secure positioning devices.

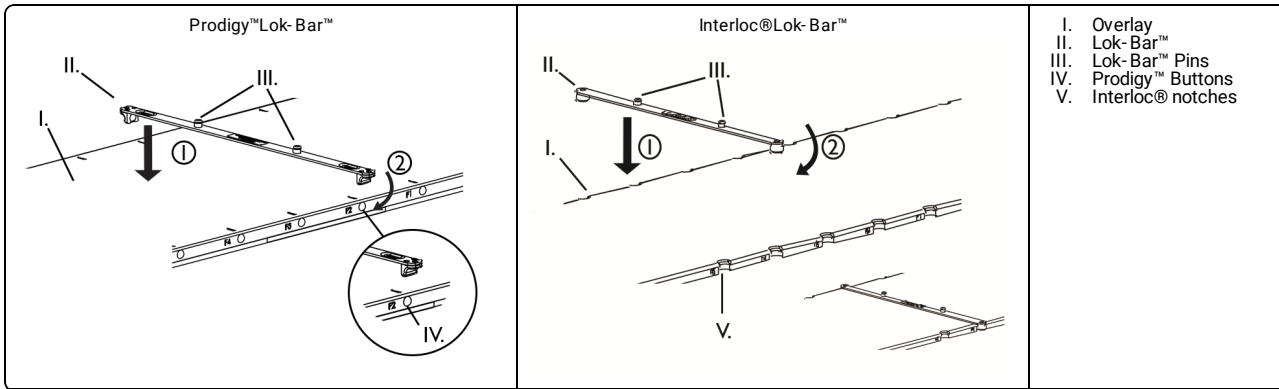1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.



NOTE: For optimal imaging, begin scanning after scan line.

**WARNING**

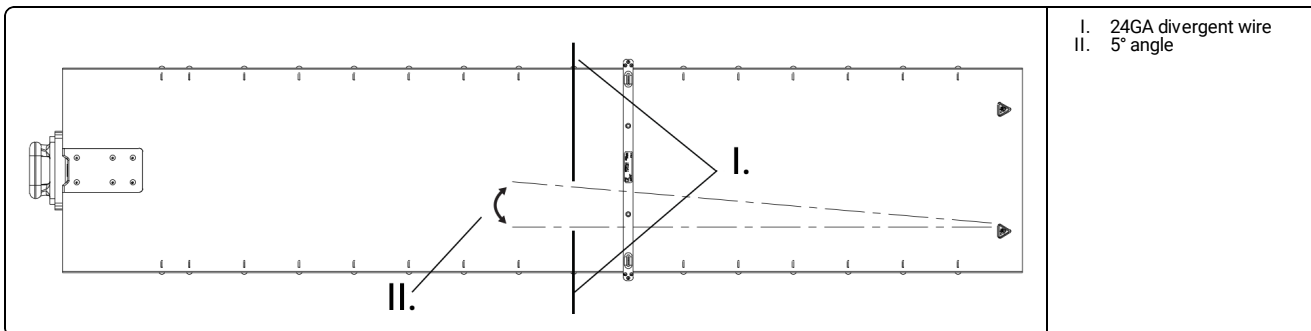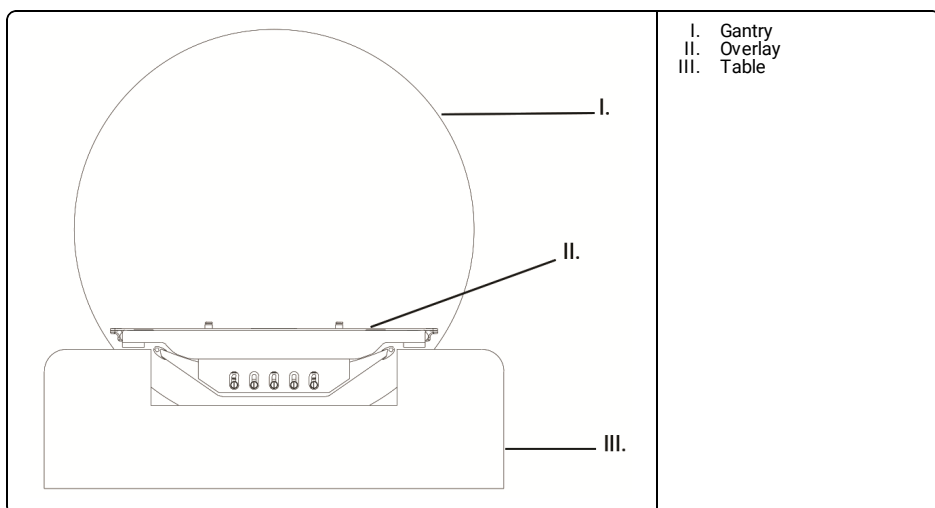- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.
- Potential pinch point occurs between overlay and gantry when scanning with the table in lowest position.
- Potential pinch point occurs between overlay and table when retracting overlay.
- Attach foam bumpers to head end of the table top to aid in pinch point prevention (MT-IL3300, MT-IL3301, and MT-IL3308).

NOTE: Spacing between divergent wires (2) can be used as a reference for slice location when viewing multiple images containing axial slices (MT-IL3300, MT-IL3301, MT-IL3302, and MT-IL3311).

### REPROCESSING

#### **WARNING**

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

### MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. INTERLOC IS A REGISTERED TRADEMARK OF MEDTEC LLC. IPPS, LOK-BAR, AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.