IPPS™ Overlay

An indexed patient positioning system for use with Toshiba Aquilion Systems.

INSTRUCTIONS FOR USE (IFU)

CE

INTENDED USE

The device is intended to aid in patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING


- Do not use if device appears damaged.
- Ensure device is secure prior to use.

NOTE: • If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
 • Overlay weight: 30 lbs (14 kg). For total table load rating, including accessories, refer to your system's manual.

ATTACHING DOCKING STATION TO CRADLE


NOTE: MT-IL3704, MT-IL3705, MT-IL3707 Tools Required: 5/32" hex wrench, 10mm hex wrench, flat head screwdriver and level (*not provided*).

1. Remove all accessories from cradle.
2. Fasten docking station loosely on the foot end of the table using screws and washers (*provided*).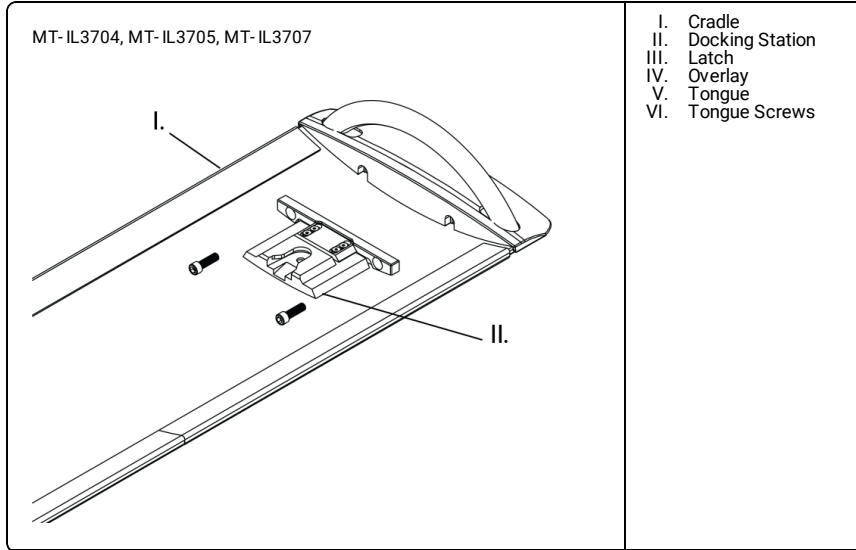


ATTACHING OVERLAY

1. Place overlay onto cradle with tongue over anterior edge.
2. Loosen tongue screws with screwdriver, adjust height as desired, retighten.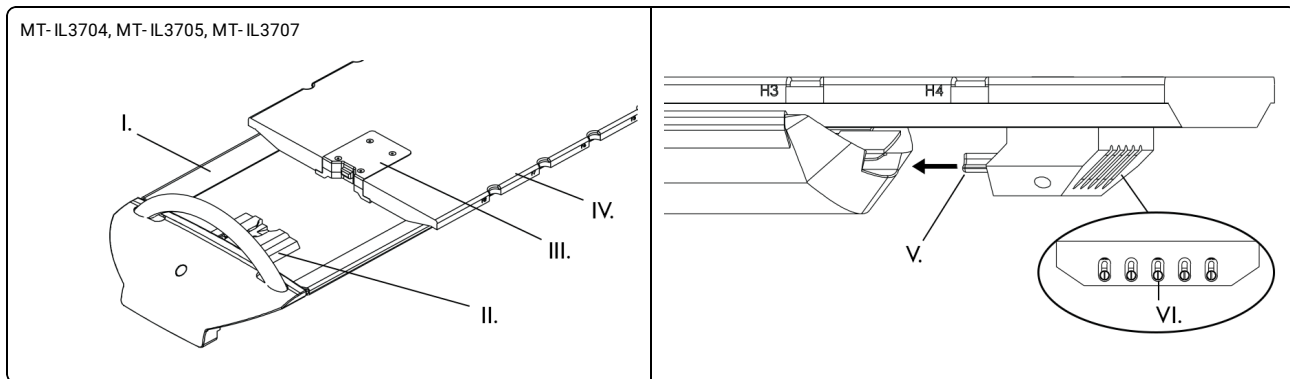

3. Insert latch into foot end of cradle until latch grasps cradle handle.
4. Ensure overlay is level to within 1° laterally and longitudinally.
5. Remove overlay with latch release button.


NOTE: • If vertical height adjustment is necessary; remove latch and adjust by adding or removing shims.
 • Adjust latch longitudinally by loosening adjustment screws.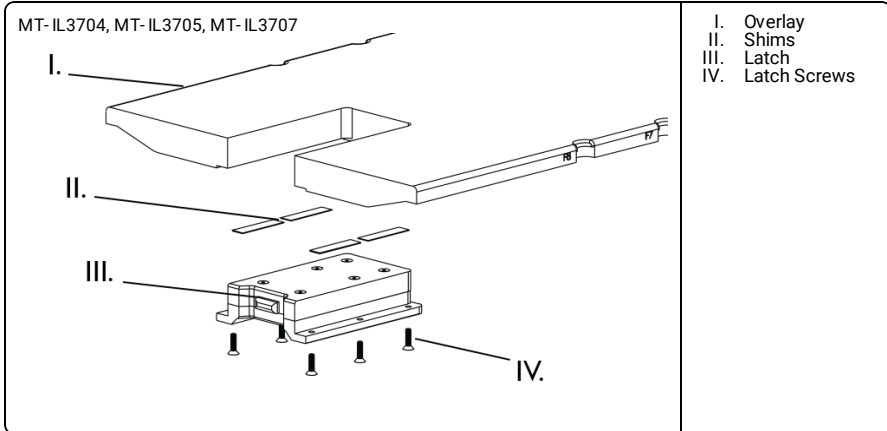


USING Lok-Bar™

WARNING
 • Do not use Lok-Bar™ to secure positioning devices.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.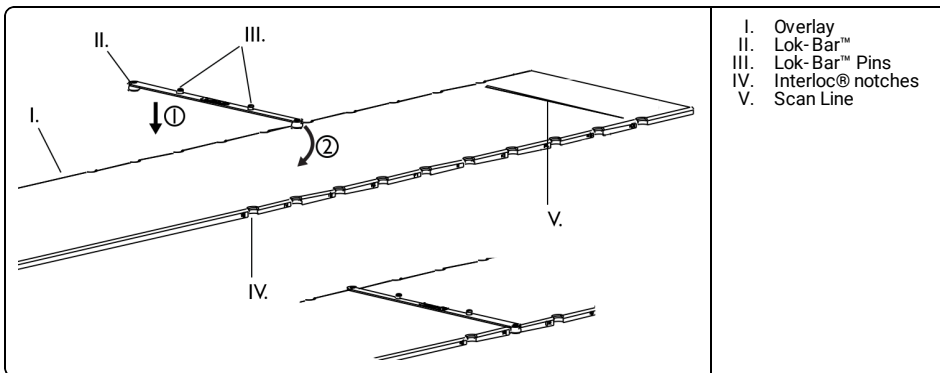


NOTE: Spacing between divergent wires (2) can be used as a reference for slice location when viewing multiple images containing axial slices.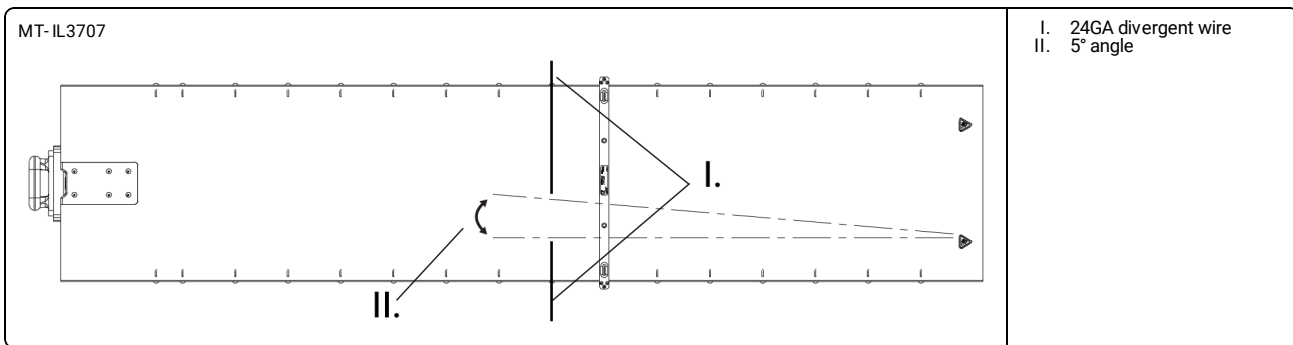

NOTE: For optimal imaging, begin scanning after scan line.

⚠ WARNING

- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.
- Potential pinch point occurs between overlay and gantry when scanning with the table in lowest position.
- Potential pinch point occurs between overlay and table when retracting overlay.

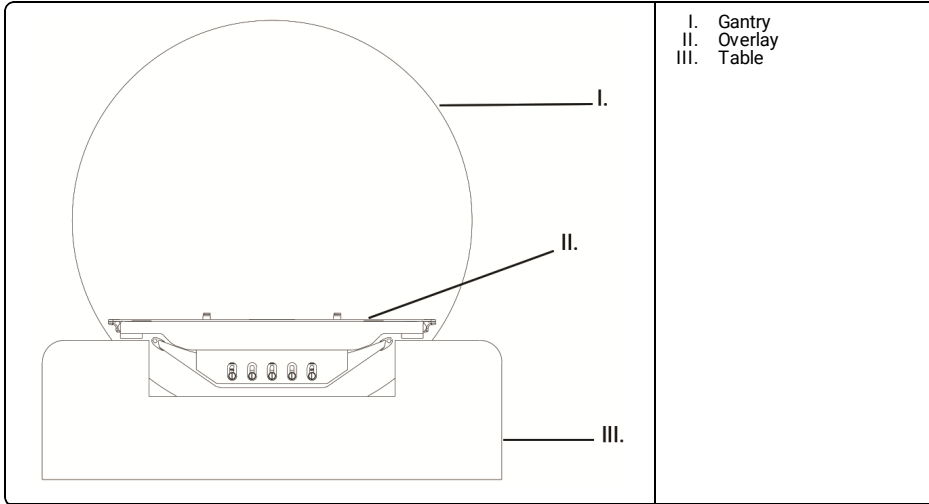**REPROCESSING****⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. IPPS, LOK-BAR, AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. INTERLOC IS A REGISTERED TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC