

Overlay Storage Box

REFERENCE GUIDE




ASSEMBLING/MOUNTING STORAGE BOX

WARNING


- Do not use if device appears damaged.

NOTE: Storage box must be mounted onto 16" on-center studs.

1. Drill pilot holes for screws: 0.149 (#25 drill) for sheet metal screws or 3/32 for wood screws.
2. Mount bottom box to wall by inserting screws through rubber bumpers and then through back into wall stud.

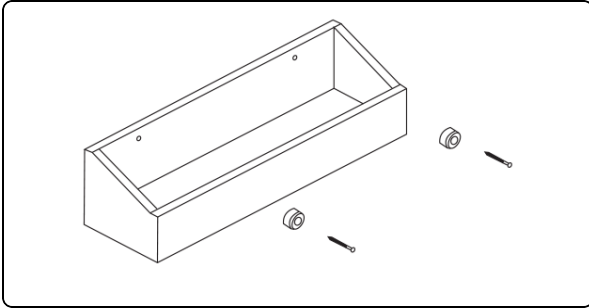

3. Determine desired distance from bottom box to top section.
4. Mount top section to wall by inserting screws through rubber bumpers and then through back into wall stud.

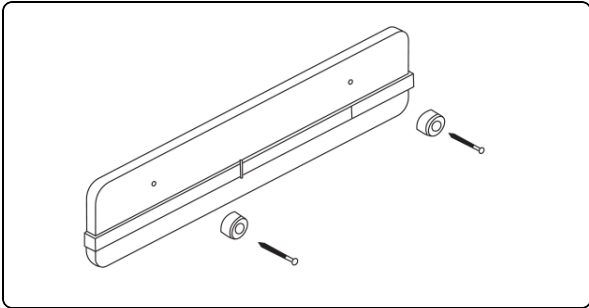

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

STORING COUCHTOP

1. Place couchtopy into bottom box and lean against top section.
2. Adhere velcro strap around couchtopy to hold in place.

CIVCO

Radiotherapy™

 CIVCO RADIOTHERAPY

1401 8th Street SE
Orange City, IA 51041
United States

800.842.8688, +1 712.737.8688

info@civcort.com

COPYRIGHT © 2022. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.

WWW.CIVCORT.COM