Posirest-2™

REFERENCE GUIDE

CIVCO

CE

INTENDED USE

Device is intended to position patients' upper extremities during breast and lung-diagnostics and radiotherapy treatment.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.


WARNING

- Do not use if device appears damaged.
- Do not allow patient to reposition themselves.
- Ensure patient remains still for duration of setup and treatment.
- Do not reposition device with patient on it.
- Verify patient position with completed setup sheet prior to treatment.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at WWW.CIVCORT.COM.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Verification image should be taken to confirm patient position prior to treatment.
- Only use CIVCO compatible accessories.
- Ensure device is secure prior to use.
- Ensure all accessories are attached securely prior to treatment.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

USING POSIREST™-2

1. Attach appropriate arm positioner to top of Posirest™-2 base.
2. Place Posirest™-2 onto Lok-Bar™.

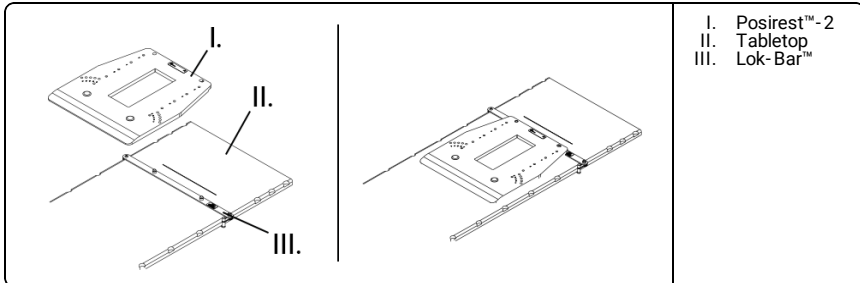- I. Posirest™-2
- II. Tabletop
- III. Lok-Bar™

3. Place desired headrest onto Posirest™-2 (refer to Headrest instruction).
4. Position patient onto Posirest™-2 as dictated by setup protocol.

REPROCESSING**WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Clean device with a mild, non-abrasive cleaner as needed.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2020 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. POSITILT IS A TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.