# IPPS™ Overlay

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**INTENDED USE**

Overlay is intended to provide a flat top with indexing for radiation treatment planning.

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.

**⚠ WARNING**

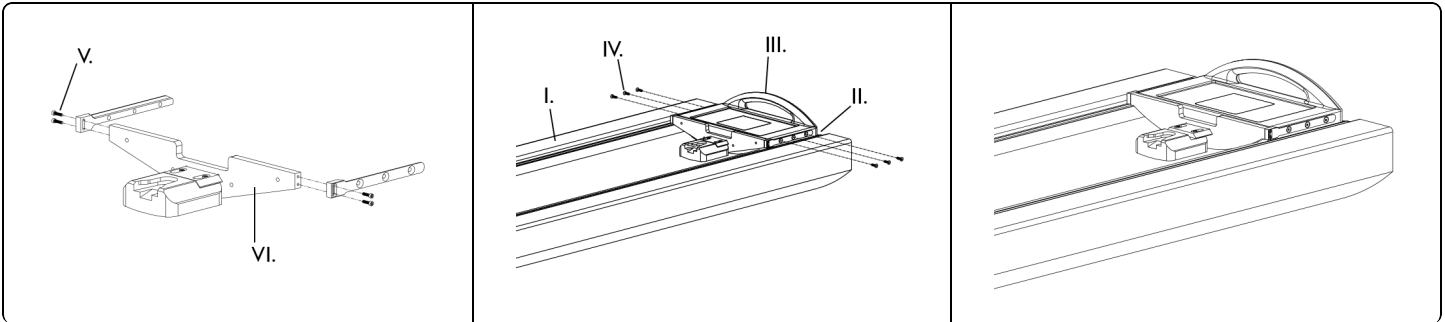
- Do not use if device appears damaged.
- Overlay weight: 30 lbs (14 kg). For total table load rating, including accessories, refer to your system's manual.
- Ensure device is secure prior to use.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

**ATTACHING DOCKING STATION TO CRADLE**

NOTE: Tools Required: 5/32" hex wrench, M5 allen wrench, 9/64" hex wrench (provided), flat head screwdriver and level (not provided).

1. Remove all accessories from cradle.
2. Attach the docking station to the side rails.
3. Attach side rails to PMM Box.
4. Ensure bottom of docking station is in contact with the cradle.
5. Loosen docking station screws and slide toward anterior end.


**ATTACHING OVERLAY**

1. Loosen tongue screws with screwdriver.
2. Slide overlay into docking station, ensure tongue fits properly into accessory slot.
3. Ensure latch is locked into docking station.
4. Tighten tongue screws to lock tongue into place.
5. Note position of docking station. Remove overlay with latch release button.
6. Tighten docking station with 5/32" hex wrench.

NOTE: Ensure docking station location allows for easy installation and removal of overlay.

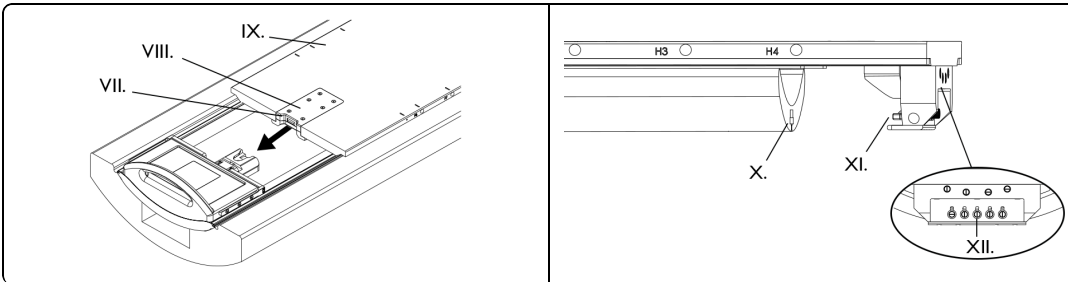7. Attach overlay to docking station.
8. Ensure overlay is level to within 1° laterally and longitudinally.

NOTE: If vertical height adjustment is necessary: remove latch and adjust by adding or removing shims.


	<ul style="list-style-type: none"> <li>I. Cradle</li> <li>II. Side Rail</li> <li>III. PMM Box</li> <li>IV. M6 Flat Head Screws</li> <li>V. #8 Socket Head Screws</li> <li>VI. Docking Station</li> <li>VII. Latch Release Button</li> </ul>	<ul style="list-style-type: none"> <li>VIII. Latch</li> <li>IX. Overlay</li> <li>X. Accessory Slot</li> <li>XI. Tongue</li> <li>XII. Tongue Screws</li> <li>XIII. Shims</li> <li>XIV. 1/4- 20 Flat Head Screws</li> </ul>
--	---	---

**USING OVERLAY**

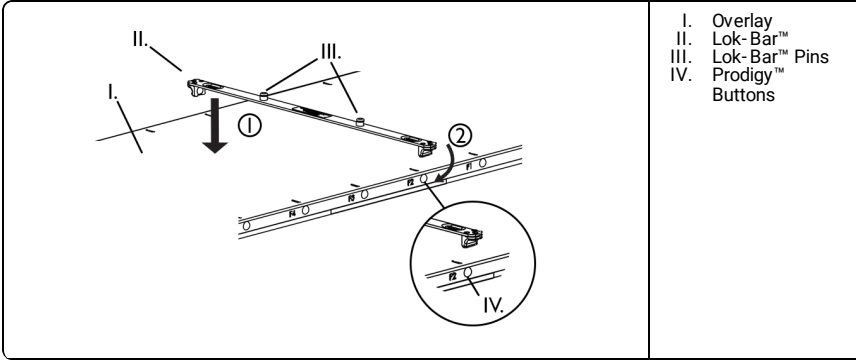**⚠ WARNING**


- Lok-Bar™ is a positioning device only, do not use to secure patient.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

- I. Overlay
- II. Lok-Bar™
- III. Lok-Bar™ Pins
- IV. Prodigy™ Buttons

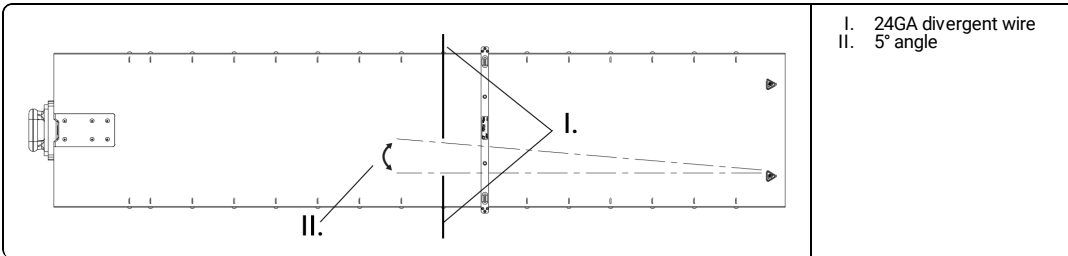
NOTE: Spacing between divergent wires (2) can be used as a reference for slice location when viewing multiple images containing axial slices.

- I. 24GA divergent wire
- II. 5° angle

**⚠ WARNING**

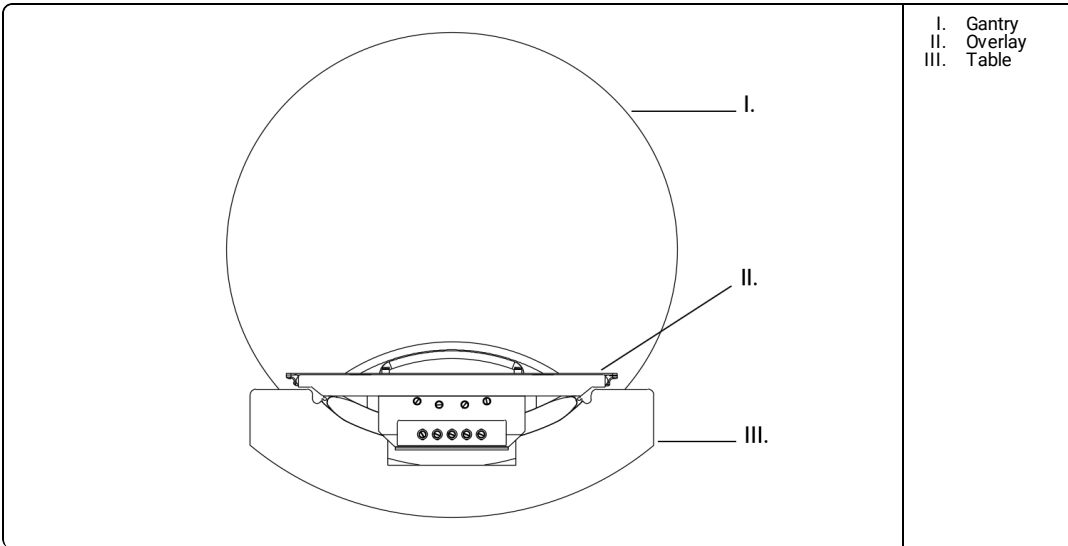- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.
- Potential pinch point occurs between overlay and gantry when scanning with the table in lowest position.
- Potential pinch point occurs between overlay and table when retracting overlay.

- I. Gantry
- II. Overlay
- III. Table

**REPROCESSING**

**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

**MAINTENANCE**

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. IPPS, LOK-BAR AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.