Thorawedge™

INSTRUCTIONS FOR USE (IFU)

CE

INTENDED USE

The device is indicated to aid in supporting and positioning adult and pediatric patients undergoing radiation therapy including electron, photon, and proton treatments. The device is also used during image acquisition to support treatment planning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- *Do not use if device appears damaged.*
- *Verify all angles of treatment and attenuation characteristics prior to treating patients.*
- *Do not exceed patient load of 330 lbs (150 kg).*
- *Do not reposition device with patient on it.*
- *Ensure patient remains still for duration of setup and treatment.*
- *Ensure device is secure prior to use.*

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

MRI SAFETY INFORMATION

MR Safe

- Device is MR safe.

USING THORAWEDGE™

1. Position device on table.

REPROCESSING

WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. Disinfect with a mild, non-abrasive cleanser.
2. Wipe dry with lint-free cloth or allow to air dry prior to reassembly and use.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

STORAGE

NOTE: Store device on a flat surface and cover with a cloth to prevent damage or soiling.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. THORAWEDGE IS A TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.