Multifix™

INSTRUCTIONS FOR USE (IFU)

CE

INTENDED USE

Device is intended to aid in patient positioning, mainly for diagnostics and radiotherapy treatment.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.


⚠ WARNING

- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not use if device appears damaged.
- Do not reposition device with patient on it.
- Do not use Lok-Bar™ to secure positioning devices.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at www.CQmedical.com.
- Verify patient position with completed setup sheet prior to treatment.
- Ensure device is secure prior to use.


NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

ASSEMBLING MULTIFIX™

1. Position Multifix™ along or perpendicular to couchtop.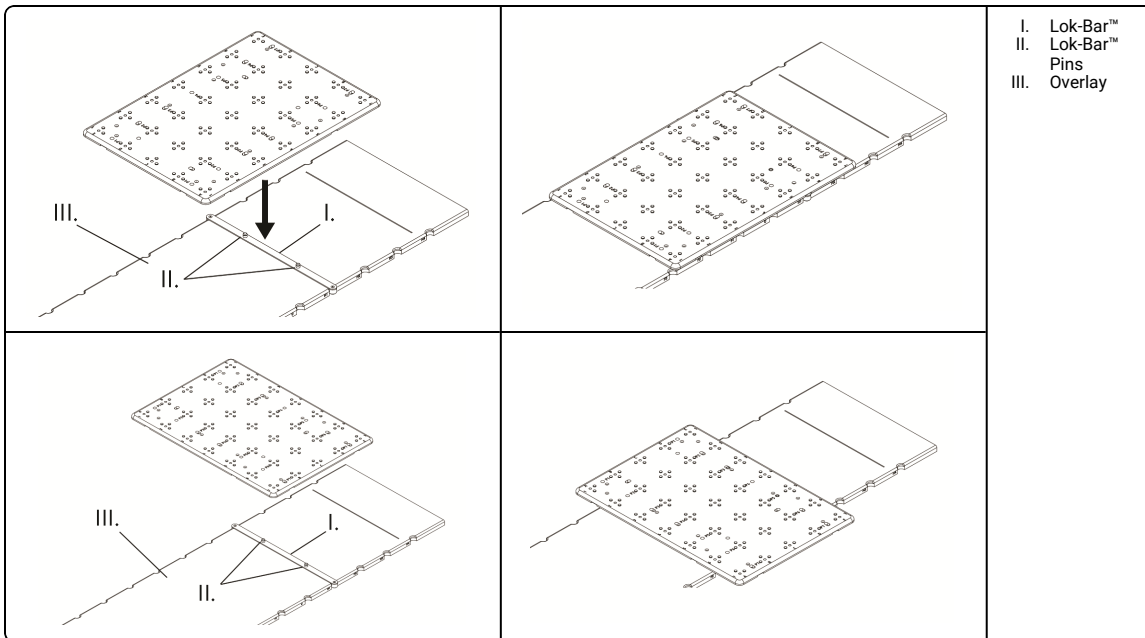

2. Ensure gantry indicator is positioned towards gantry.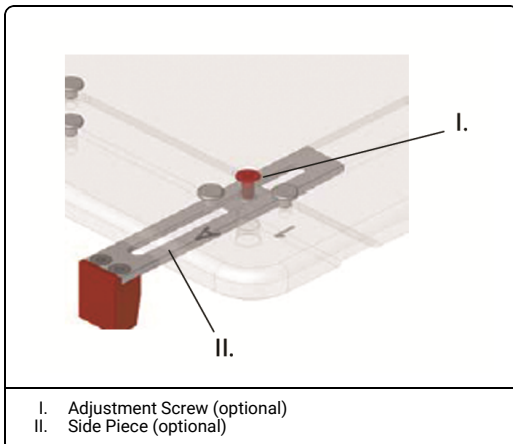

3. Loosen all adjustment screws located on sidepieces and rotate sidepiece to desired orientation.
4. Push all side pieces against table and tighten adjustment screws.


⚠ WARNING

- Ensure adjustment screws are tightened properly by verifying the device is secure.
- Ensure clearance is maintained between Multifix™ and all Linac or imaging equipment before using device.

USING MULTIFIX™

NOTE: See thermoplastic manual for Posicast® use.

1. With patient in desired treatment position, position profile with attached Posicast, above Multifix™ over desired attachment area.
2. Position profile over desired pivots and push quick fasteners downwards into profile.

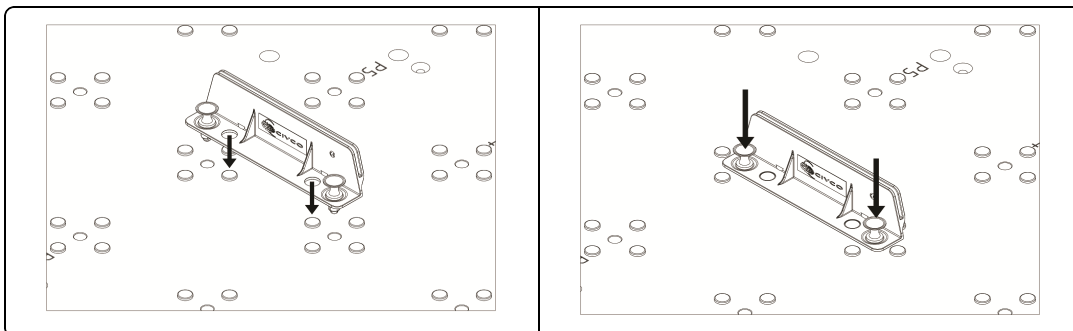NOTE: Pull quick fasteners upwards to release profile.

REPROCESSING**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. LOK-BAR AND MULTIFIX ARE TRADEMARKS OF MEDTEC LLC. POSICAST IS A REGISTERED TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC