Vac-Lok™ Vacuum Compressors

INSTRUCTIONS FOR USE (IFU)

INTENDED USE

Device is intended to inflate and/or deflate Vac-Lok™ cushions.


 **WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of safety hazard control to co-workers and themselves. To avoid safety hazards, wear all recommended protective equipment and follow safety control policies established by your facility.
- Routinely check vacuum compressors and power cords for signs of deterioration.
- Do not use if device appears damaged.
- Do not cover ventilation openings.
- Do not attempt to service device.

NOTE: Place female valve connector onto cushion valve to let air into Vac-Lok™ cushion without using pump.


SETTING UP SINGLE FUNCTION VAC-LOK™ VACUUM COMPRESSOR

1. Verify Vac-Lok™ Vacuum Compressor's ON/OFF switch is OFF and unit is unplugged from power source.
2. Remove tape from ports on compressor.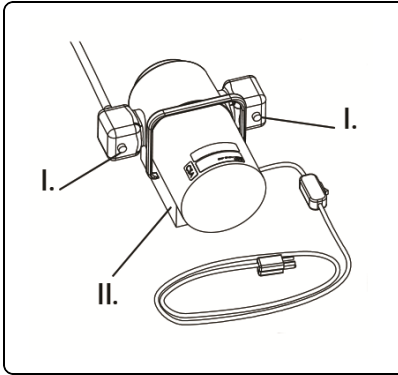

I. Pump Port
II. Pump/Compressor

3. Attach mufflers to ports.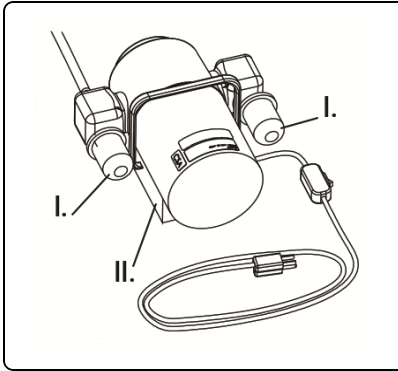


I. Muffler
II. Pump/Compressor

USING SINGLE FUNCTION VAC-LOK™ VACUUM COMPRESSOR

NOTE: Voltage requirements: MT-VL-1A: 110 V / 8 A 60 Hz

1. Verify Vac-Lok™ Vacuum compressor's ON/OFF switch is OFF.
2. Plug power cord into appropriate GFCI (ground fault circuit interrupter) receptacle.
3. Attach vacuum hose to cushion valve (*refer to Vac-Lok™ Cushion manual*).

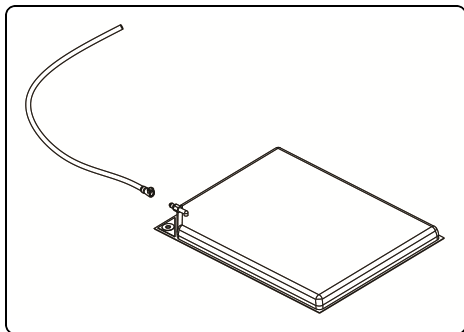4. Press ON/OFF switch until most air is drawn from cushion.
5. Verify vacuum's ON/OFF switch is OFF.
6. Disconnect vacuum hose from Vac-Lok™ cushion.

REPROCESSING

1. Clean device with a mild, non-abrasive cleaner as needed.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

DISPOSAL

WARNING

- *Device should be disposed of in manner appropriate for electronic equipment.*

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. VAC-LOK IS A TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
**MEDTEC LLC**