MR Lok-Bar™

INSTRUCTIONS FOR USE (IFU)

CQ MEDICAL™

CE

INTENDED USE

The device is intended to aid in support and positioning of patients during MR, radiological and other procedures.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- *Do not use if device appears damaged.*
- *Do not use Lok-Bar™ to secure positioning devices.*
- *Verify all angles of treatment and attenuation characteristics prior to treating patients.*
- *Only use CQ Medical compatible accessories.*

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

MRI SAFETY INFORMATION

MR Safe

- Device is MR safe.


USING Lok-Bar™

1. Snap Lok-Bar™ at appropriate longitudinal position.

WARNING

- *Ensure Lok-Bar™ is secure.*

2. Place positioning accessory onto Lok-Bar™ pins.

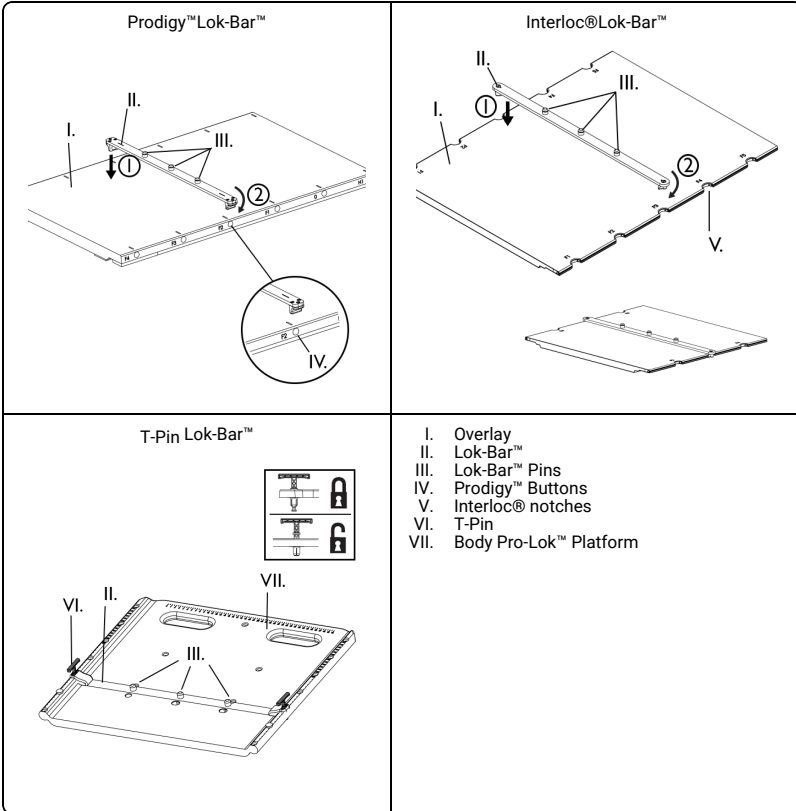REPROCESSING

⚠ WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Disinfect with a mild, non-abrasive cleanser.
2. Wipe dry with lint-free cloth or allow to air dry prior to reassembly and use.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2026 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. BODY PRO-LOK, LOK-BAR AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. INTERLOC IS A REGISTERED TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.