IPPS™ Overlay

An indexed patient positioning system for use with Siemens Magnetom Aera, Magnetom Avanto, Magnetom Avanto fit, Magnetom Espree, Magnetom Prisma, Magnetom Prisma fit, Magnetom Skyra, Magnetom Skyra fit, Magnetom Spectra, Magnetom Symphony A Tim System, Magnetom Trio A Tim System, Magnetom Verio, Magnetom Vida

REFERENCE GUIDE

CIVCO
Radiotherapy™

CE

INTENDED USE

The device is intended to aid in patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

- Do not use if device appears damaged.
- Use MR compatible accessories only.
- Ensure all accessories are securely attached in correct position prior to imaging.
- Ensure device is secure prior to use.
- Avoid scanning in first level mode.
- Avoid scanning sedated patients.
- Ensure patient is informed to use squeeze ball in case of heating sensation.

- NOTE:
- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
 - Do not exceed a patient weight of 550 lbs (250 kg).
 - Store overlay in upright position when not in use to prevent damage to overlay supports.

MRI SAFETY INFORMATION

- Device is MR safe.


MR Safe

USING OVERLAY

PRIOR TO USE: Refer to operating instructions provided with each RF Coil.


NOTE: Tools Required: 3/16" Allen Wrench.

1. Remove table pads and fixed head coil from table.
2. Attach overlay supports to overlay.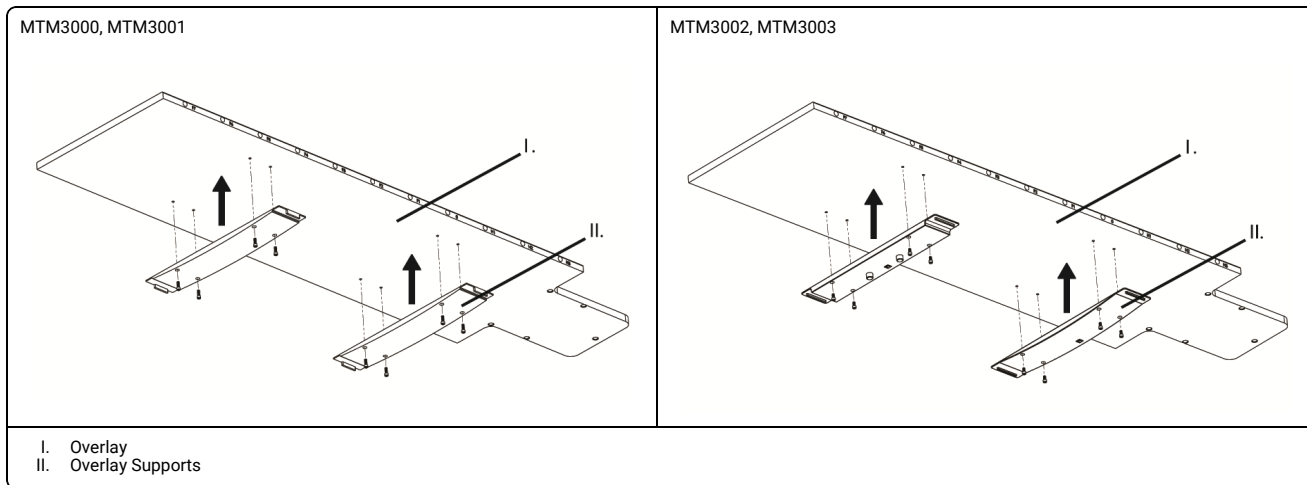


3. Assemble head support.

NOTE: Tools Required: 5/32" hex wrench.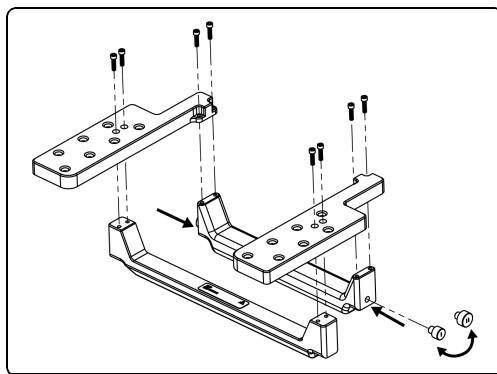


4. Place head bridge base at head end of table (if required.)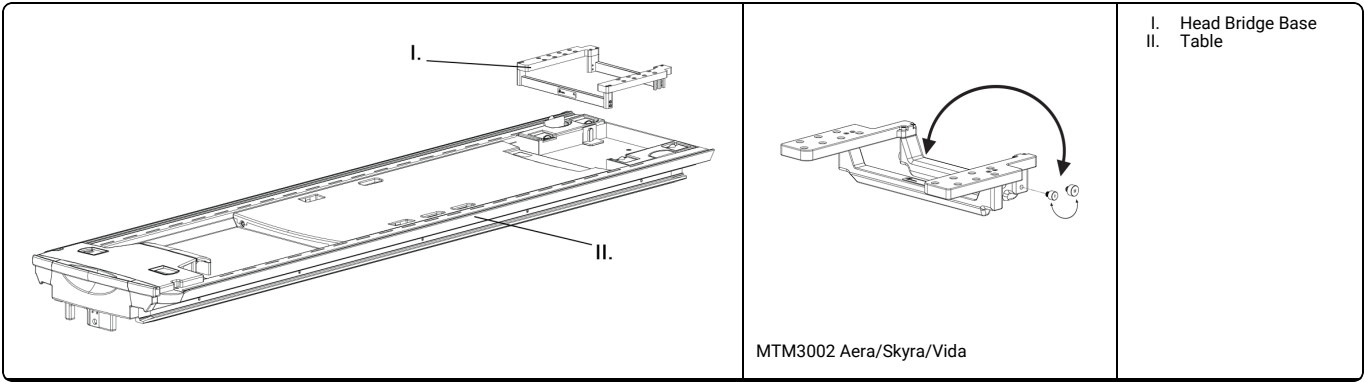


5. Set overlay on table in desired location, ensure overlay supports rest in slots on side of table.

⚠ WARNING


- Grasp overlay at head end and foot end, using two people, to prevent pinched fingers.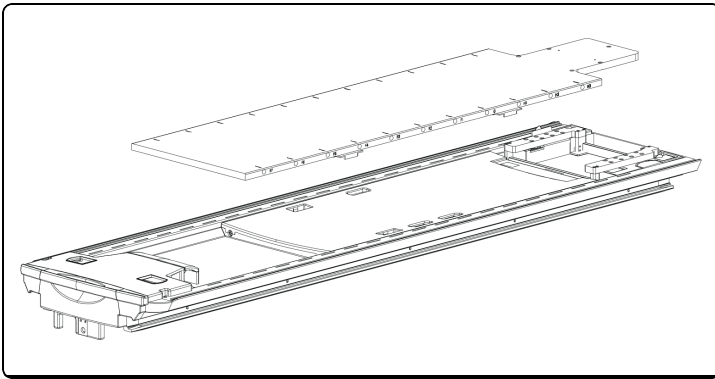

6. Plug in coils.
7. Position patient as desired.
8. Position coils and head bridge (if required), in desired location. Press down pins to lock in place.
9. Position coil and body bridges (if applicable), attaching clips into slots at side of table.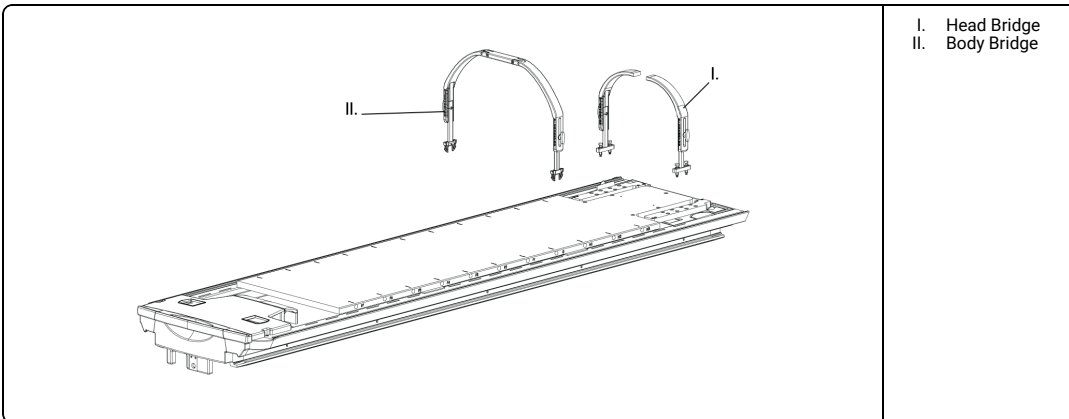


NOTE: Position coils between patient and bridges, using existing Velcro to secure coils to bridges.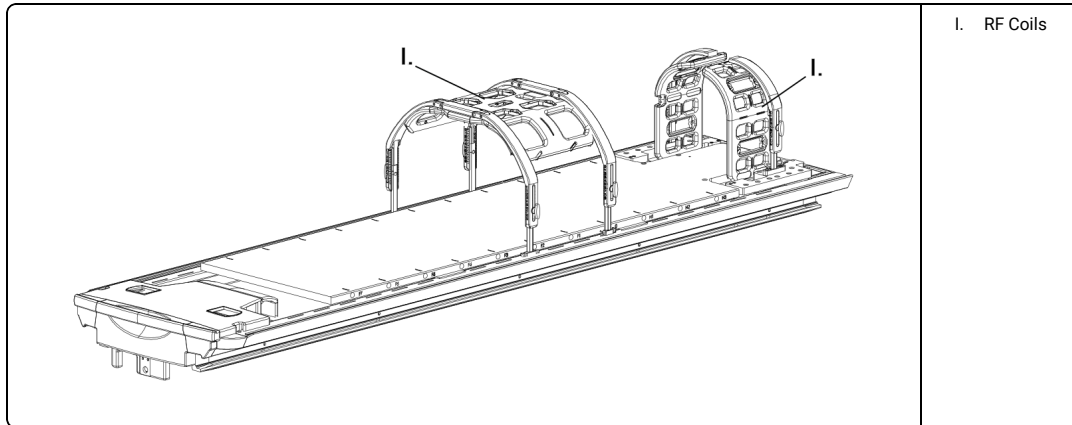

USING THERMOPLASTICS (OPTIONAL)

NOTE: Thermoplastic adapter is required for head, neck and shoulder thermoplastics.

1. Snap thermoplastic adapter to H3 button.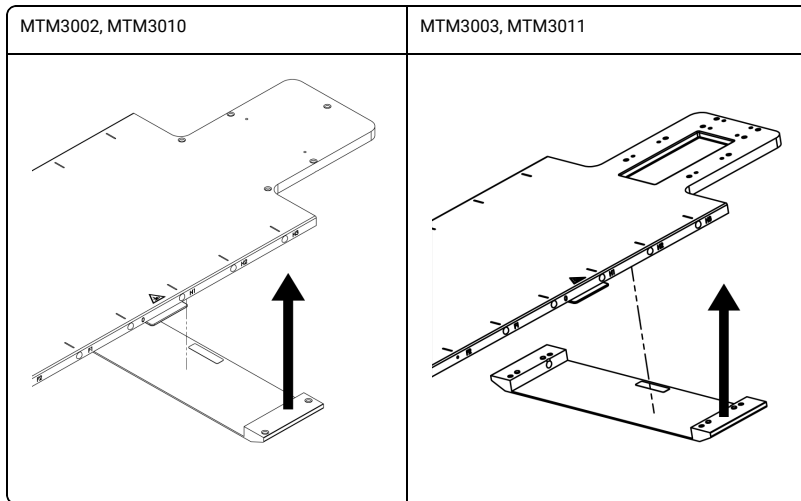


2. Position patient onto overlay for thermoplastic setup (refer to Thermoplastic instruction).
3. Form and attach thermoplastic mask (refer to Thermoplastic instruction).

USING Lok-Bar™

⚠ WARNING

- Do not use Lok-Bar™ to secure positioning devices.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

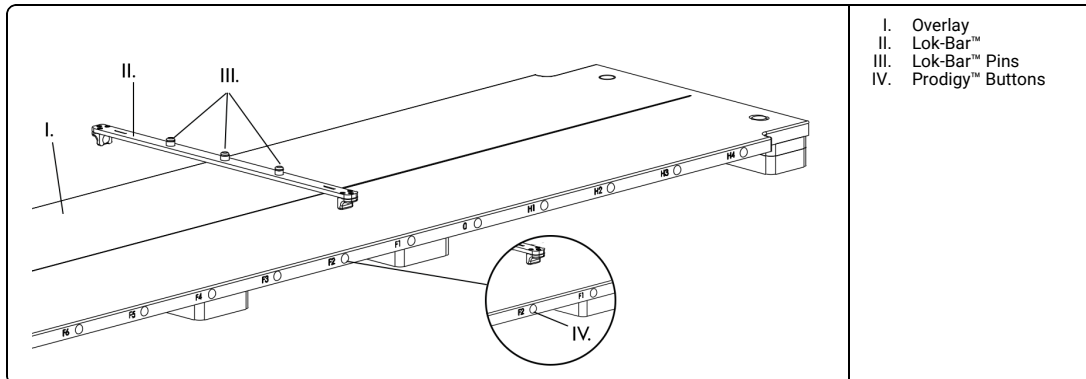⚠ WARNING

- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.

REPROCESSING **WARNING**

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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CIVCO

Radiotherapy™

 CIVCO RADIOTHERAPY
1401 8th Street SE
Orange City, IA 51041
United States
info@civcort.com

COPYRIGHT © 2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. IPPS, LOK-BAR AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.

WWW.CIVCORT.COM