Repair Kit

For use with Waterbath

INSTRUCTIONS FOR USE (IFU)

INTENDED USE

Device is intended to provide the materials and instruction for an approved repair of waterbaths.

**WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of safety hazard control to co-workers and themselves. To avoid safety hazards, wear all recommended protective equipment and follow safety control policies established by your facility.
- Consult your facilities manager before assembly and installation.
- Use only power cords provided.
- Do not use if device appears damaged.
- Hot water poses a scalding hazard.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

REPLACING POWER INLET, SWITCH, AND WIRES

**WARNING**

- Ensure waterbath is turned off, unplugged, cooled and drained before performing maintenance.

I. Power Inlet II. Switch III. Clips								
Waterbath								
Power Inlet A		<table><tr><td></td><td>Green/White</td></tr><tr><td>N:</td><td>White New</td></tr><tr><td>L:</td><td>Black New</td></tr></table>		Green/White	N:	White New	L:	Black New
	Green/White							
N:	White New							
L:	Black New							
Power Inlet B		<table><tr><td></td><td>Green/White</td></tr><tr><td>N:</td><td>White New</td></tr><tr><td>L:</td><td>Black New</td></tr></table>		Green/White	N:	White New	L:	Black New
	Green/White							
N:	White New							
L:	Black New							
Switch		4B: White New 5B: White Existing 1A: Black New 2A: Black Existing						

1. Remove power cord.
2. Remove fuse holder from installed power inlet to access clips (only required for Power Inlet A).
3. Using a screwdriver, pry out installed power inlet by depressing retaining clips.

4. Disconnect three existing wires from installed power inlet, ensuring green/white wire does not fall into waterbath case. Discard old power inlet.
5. Attach green/white wire to new power inlet terminal identified with ground symbol.


6. Attach new black wire (*provided*) to new power inlet terminal marked "L".
7. Attach new white wire (*provided*) to new power inlet terminal marked "N".

NOTE: Do not push new power inlet into mounting hole at this time.

8. Using a screwdriver, pry out old switch (*see image above*).
9. Disconnect existing black wire from terminal marked "2A" on old switch.
10. Disconnect existing white wire from terminal marked "5B" on old switch.
11. Leave existing black and white wires already attached to old switch terminals "1A" and "4B".
12. Gently pull new black and white wires (*attached in steps 6 & 7*) through waterbath case until wire ends extend out of mounting hole for the switch.

NOTE: Care should be taken to avoid pulling opposite end of new wires loose from new power inlet terminals.

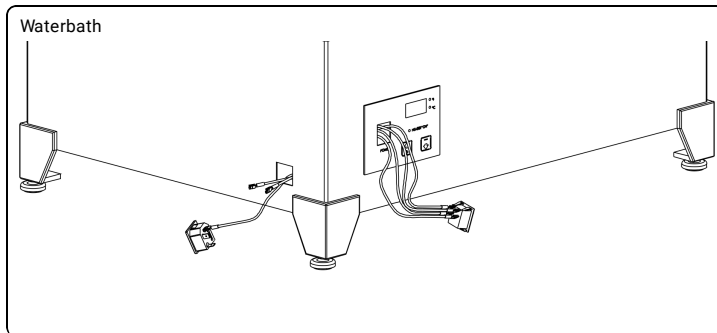13. Connect new black wire to terminal marked "1A" on new switch.
14. Connect new white wire to terminal marked "4B" on new switch.
15. Connect old white wire to terminal marked "5B". The opposite end of this white wire has always been connected to main waterbath.
16. Connect old black wire to terminal marked "2A". The opposite end of this black wire has always been connected to main waterbath.
17. Push new power inlet into mounting hole with fuse holder in downward position, ensuring clips engage and power inlet is fully seated.
18. Push new switch into mounting hole of waterbath case with "1" or "ON" portion in upward position, ensuring clips engage and switch is fully seated.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC