Universal Couchtop™ PET/CT Overlay

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INTENDED USE

The device is intended to aid in patient positioning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING


- Do not use if device appears damaged.
- Ensure extension is in locked position by lifting gently on superior end.
- Do not exceed patient weight of 440 lbs. (200 kg.).

- NOTE:
- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
 - Overlay weight: 20 lbs (9.1 kg). For total table load rating, including accessories, refer to your system's manual.

ATTACHING DOCKING STATION TO CRADLE

NOTE: Tools Required: 5/16" hex wrench and level (*not provided*).

1. Remove all accessories from cradle.
2. Fasten docking station loosely using screws and washers (*provided*).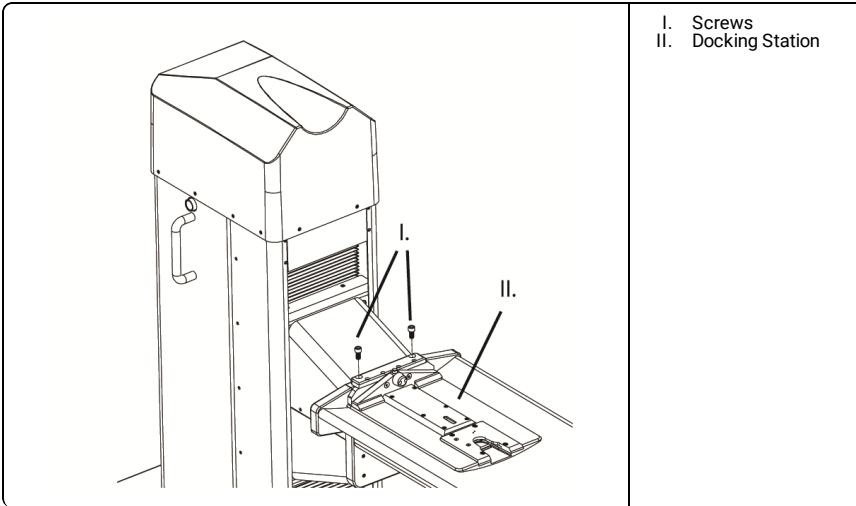


ADJUSTING OVERLAY (IF APPLICABLE)

NOTE: If adjustment is necessary use the following options or combination of options.


DOCKING STATION ADJUSTMENT

NOTE: Tools Required: 3/32" and 5/32" hex wrench and level (*not provided*).


1. Loosen screws holding king pin guide cover.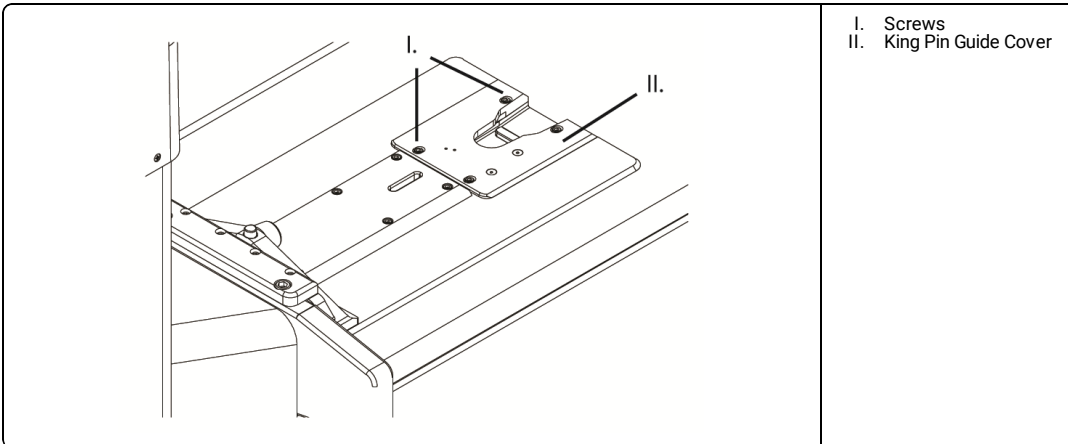

2. Slide overlay into docking station, ensuring locating cone fits securely into end of overlay and king pin fits properly into locked position of king pin guide cover.

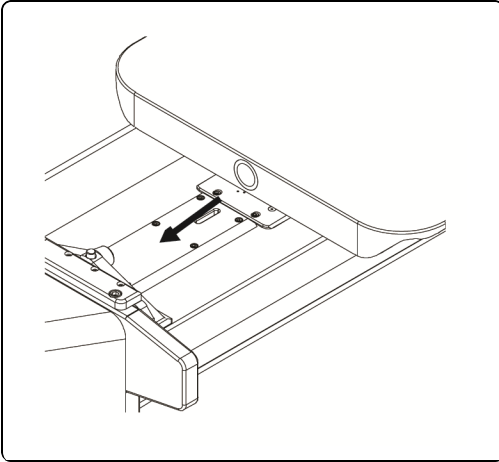
3. Note and mark position of king pin guide cover.
 4. Remove overlay.
 5. Secure king pin guide cover by tightening screws.
 6. Loosen release mechanism adjustment screws and adjust release mechanism so red release button sticks out of end plate by approximately .25".
 7. Ensure latch catch is fully engaged with "U" shaped latch mechanism and tighten all screws.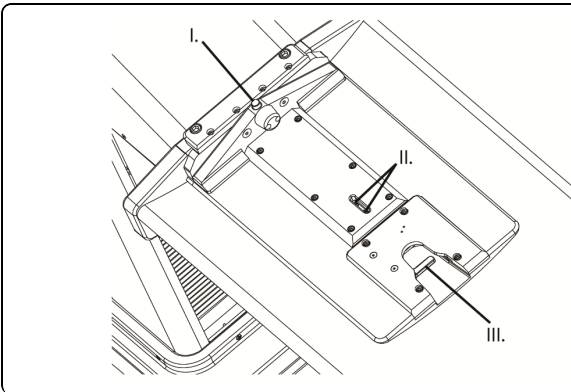

- I. Red Release Button
 II. Adjustment Screws
 III. "U" Shaped Latch Mechanism

8. Install locking device to each side with screws (provided).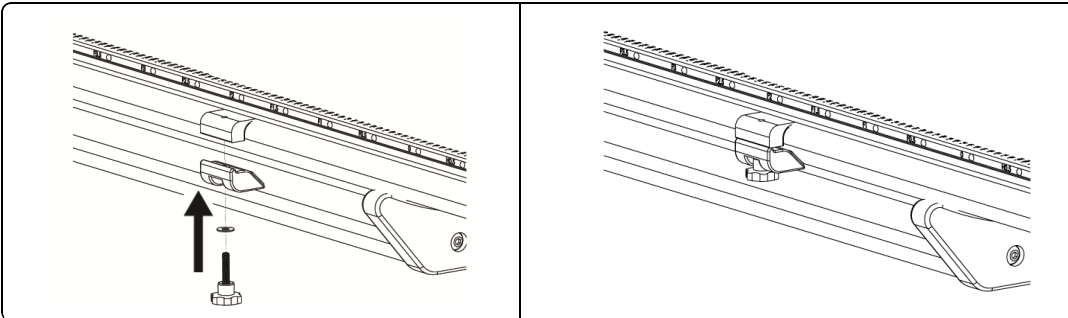

9. Slide overlay into docking station, ensuring locating cone fits securely into end of overlay and king pin fits and locks into king pin guide cover.
 10. Adjust to desired position.

NOTE:

- Verify locking mechanism releases properly.
- Ensure docking station location allows for easy installation and removal of overlay.
- Ensure overlay is level to within 1° laterally and longitudinally.


11. Once overlay is locked properly, tighten screws securing docking station to cradle.

LEVELING ADJUSTMENT (IF APPLICABLE)

NOTE: If adjustment is necessary use the following options or combination of options.

OPTION A

1. Remove king pin mounting plate and adjust by adding or removing shims.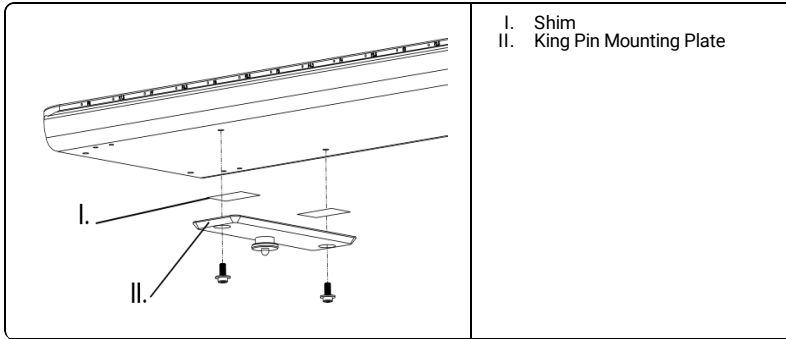


- I. Shim
- II. King Pin Mounting Plate

OPTION B

1. Loosen socket cap screw to allow adjustment of support.
2. Adjust set screws as needed.

NOTE: Set screws can be adjusted individually to allow for lateral leveling and vertical adjustment of overlay.

3. Tighten socket cap screws to lock adjustment in place.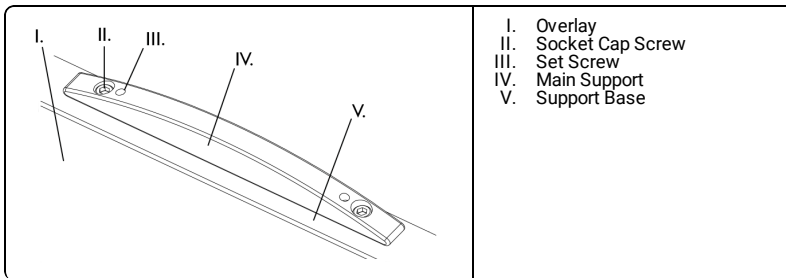


- I. Overlay
- II. Socket Cap Screw
- III. Set Screw
- IV. Main Support
- V. Support Base

OPTION C

1. Remove socket cap screws.
2. Add/remove spacer plates or shims as needed.

NOTE: Spacers and shims can be adjusted individually to match adjustment of main support.

3. Insert and tighten socket cap screws, attaching short supports to overlay.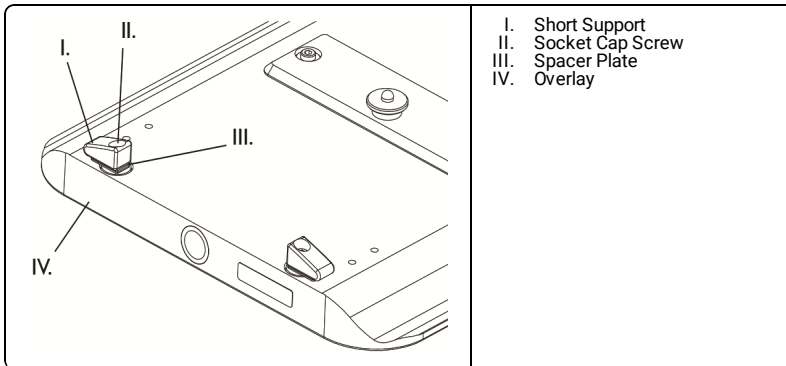


- I. Short Support
- II. Socket Cap Screw
- III. Spacer Plate
- IV. Overlay


USING Lok-Bar™

⚠ WARNING
 • Do not use Lok-Bar™ to secure positioning devices.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.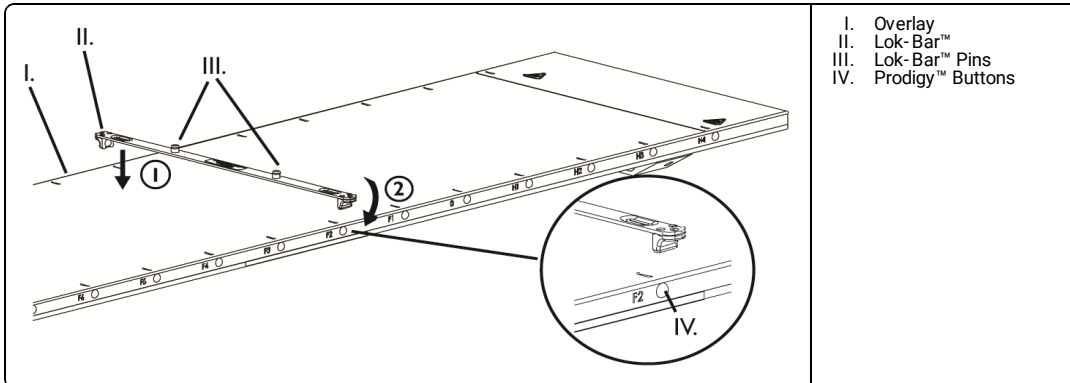

⚠ WARNING
 • Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.
 • Potential pinch point occurs between overlay and gantry when scanning with the table in lowest position.

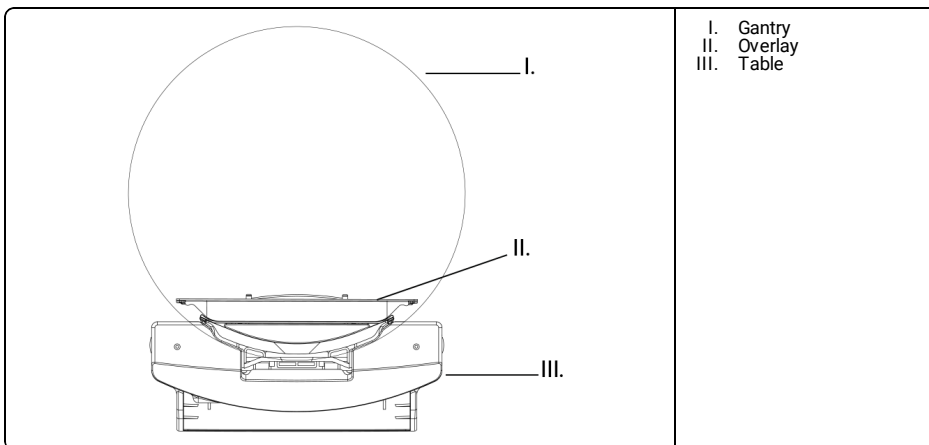REPROCESSING

⚠ WARNING
 • Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. LOK-BAR, PRODIGY AND UNIVERSAL COUCHTOP ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC