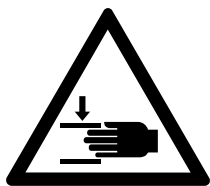
Positilt™

INSTRUCTIONS FOR USE (IFU)

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INTENDED USE

Device is intended to incline the patients head to optimize the radiotherapy treatment of the head or neck.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

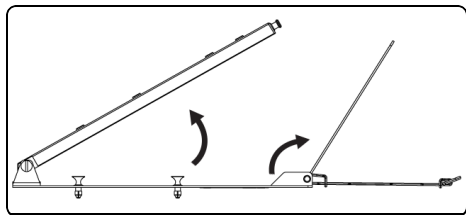
⚠ WARNING

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not reposition device with patient on it.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at www.CQmedical.com.
- Verify patient position with completed setup sheet prior to treatment.
- Do not allow patient to reposition themselves.
- Only use compatible CQ Medical accessories in combination with this device.
- Ensure device is secure prior to use.


NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

USING POSITILT™


1. Lift upper plate and angle plate.

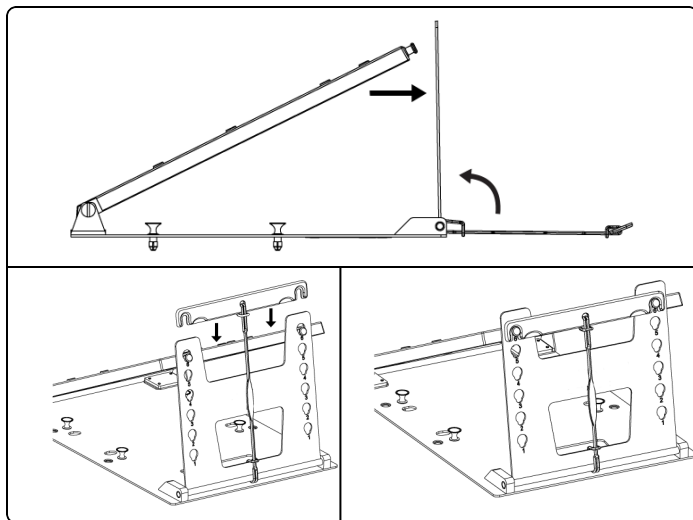2. Lower upper plate into desired angle.

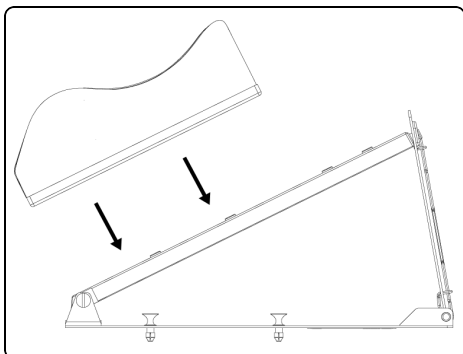3. Ensure baseplate is fully seated into indexing support plate.
4. Place desired headrest onto Positilt™.

REPROCESSING

WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. Clean device with a mild, non-abrasive cleaner as needed.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. POSITILT IS A TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC