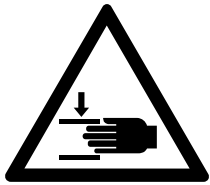
Universal Couchtop™ CT Overlay

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INTENDED USE

The device is intended to support and aid in positioning a patient during radiologic and other medical procedures.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- Do not use if device appears damaged.
- Ensure extension is in locked position by lifting gently on superior end.

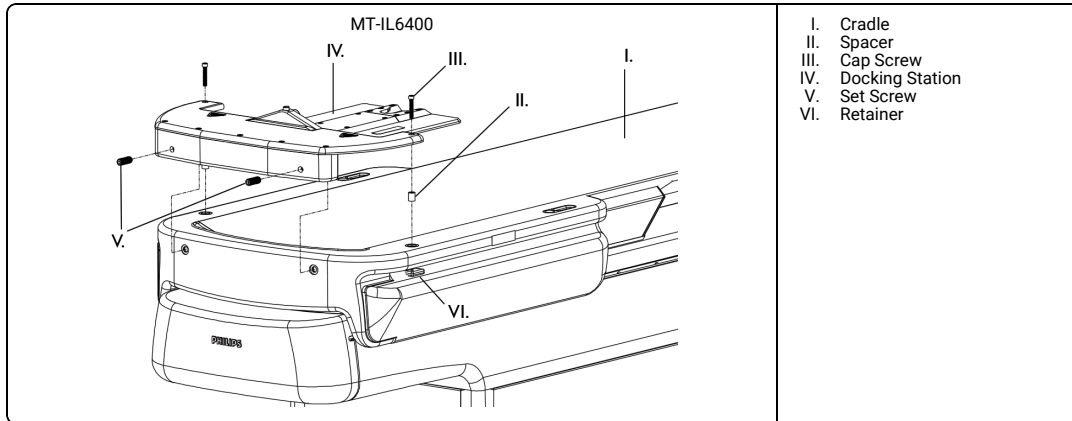
NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

Overlay weight: 20 lbs (9.1 kg). For total table load rating, including accessories, refer to your system's manual.

ATTACHING DOCKING STATION TO CRADLE


NOTE: Tools Required: 5/32", 3/16", 1/4", and 1/16" hex wrenches and level (*not provided*).

1. Remove all accessories from cradle.
2. Fasten docking station using caps screws (*provided*), spacers and set screws (*provided*).

ATTACHING OVERLAY

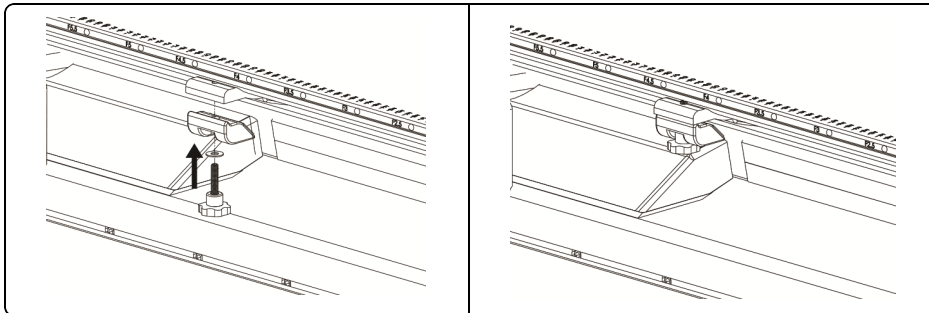1. Install locking device to each side with screws (*provided*).
2. Slide overlay into docking station, ensuring locating cone fits securely into end of overlay and king pin fits properly into locked position of king pin guide cover.

3. Adjust to desired position.
- NOTE: Verify locking mechanism releases properly.
- Ensure docking station location allows for easy installation and removal of overlay.
 - Ensure overlay is level to within 1" laterally and longitudinally.
4. Once overlay is locked properly, tighten screws securing docking station to cradle.

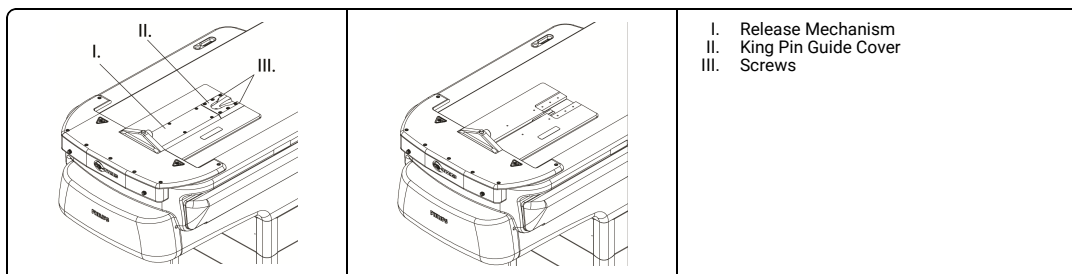
ADJUSTING OVERLAY (IF APPLICABLE)

NOTE: If adjustment is necessary use the following options or combination of options.


DOCKING STATION ADJUSTMENT

NOTE: Tools Required: 3/32", 5/32" and M8 hex wrenches and level (*not provided*).


1. Remove release mechanism cover and king pin guide cover.

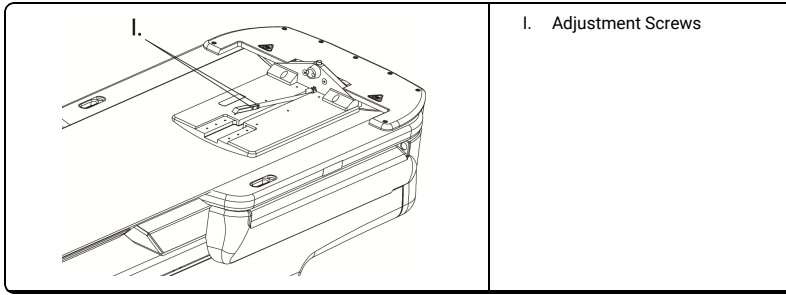
2. Loosen release mechanism adjustment screws, adjust release mechanism until fully extended and retighten screws.

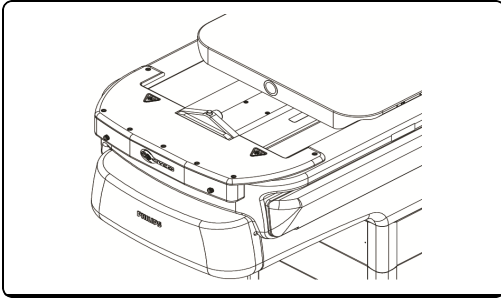3. Fasten king pin guide cover loosely using screws.
 4. Slide overlay into docking station, ensuring locating cone fits securely into end of overlay and king pin fits properly into locked position of king pin guide cover.

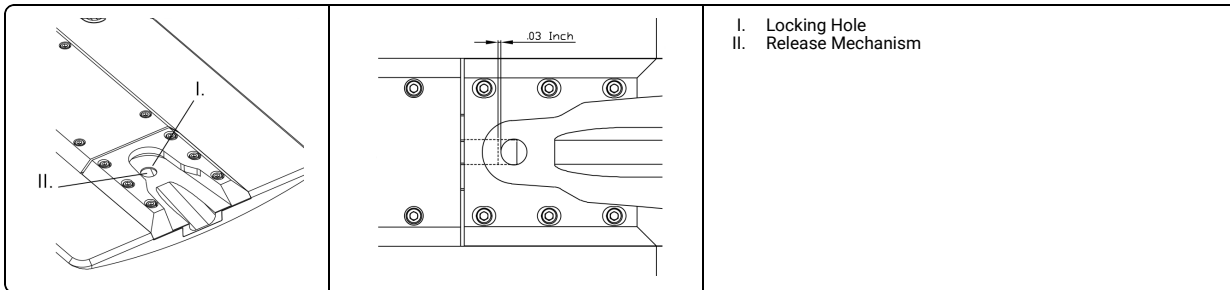5. Remove overlay.
 6. Note and mark position of king pin guide cover.
 7. Loosen release mechanism adjustment screws and adjust release mechanism until wedge end of release mechanism aligns with rear end of locking hole in king pin guide cover.
 NOTE: Verify all adjustment screws are tight, ensuring king pin guide cover is secured in marked position.


8. Fasten release mechanism cover onto docking station.
 9. Slide overlay into docking station, ensuring locating cone fits securely into end of overlay and king pin fits properly into locked position of king pin guide cover.
 NOTE:
 - Verify locking mechanism releases properly.
 - Ensure docking station location allows for easy installation and removal of overlay.
 - Ensure overlay is level to within 1° laterally and longitudinally.
 10. Once overlay is locked properly, tighten screws securing docking station to cradle.

LEVELING ADJUSTMENT (IF APPLICABLE)

NOTE: If adjustment is necessary use the following options or combination of options.

OPTION A

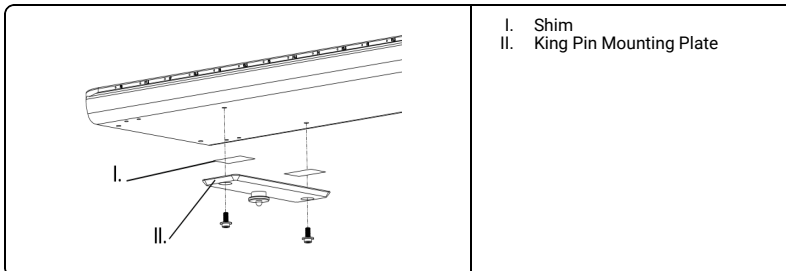
1. Remove king pin mounting plate and adjust by adding or removing shims.

OPTION B


1. Loosen socket cap screw to allow adjustment of support.
 2. Adjust set screws as needed.
 NOTE: Set screws can be adjusted individually to allow for lateral leveling and vertical adjustment of overlay.

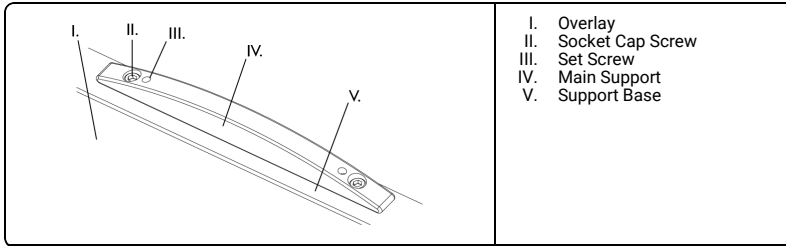3. Tighten socket cap screws to lock adjustment in place.

OPTION C

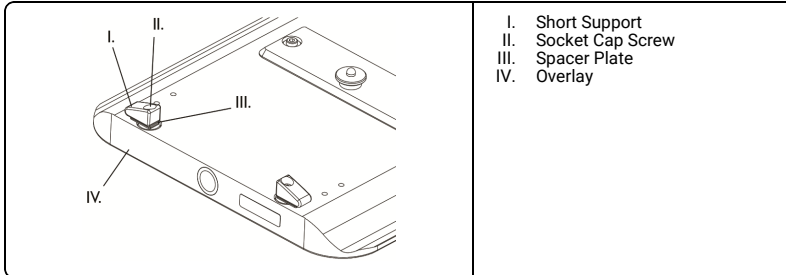1. Remove socket cap screws.
 2. Add/remove spacer plates or shims as needed.
- NOTE: Spacers and shims can be adjusted individually to match adjustment of main support.
3. Insert and tighten socket cap screws, attaching short supports to overlay.


ATTACHING PHANTOM MOUNT

1. Slide phantom mount onto superior end of couchtop assembly.
2. Latch both levers to secure mount.

WARNING

- Ensure device is secure prior to use.
- Do not apply a load exceeding 33 lbs (15 kg).

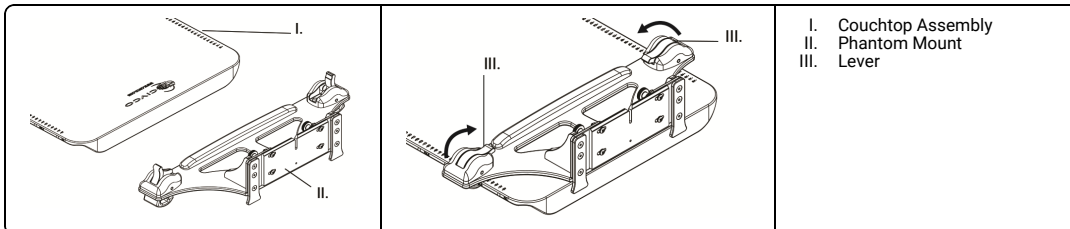NOTE: Prior to patient positioning and imaging, remove phantom mount.


USING Lok-Bar™

WARNING


- Do not use Lok-Bar™ to secure positioning devices.

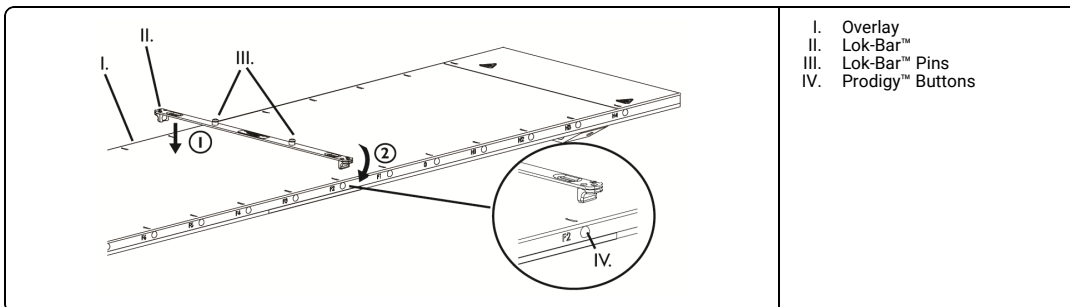1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

 WARNING

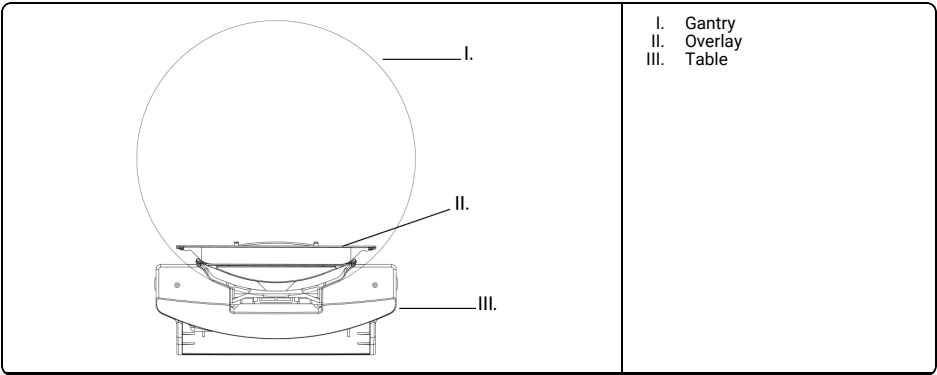- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.
- Potential pinch point occurs between overlay and gantry when scanning with the table in lowest position.

REPROCESSING

 WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of loose parts, damage, and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. LOK-BAR, PRODIGY AND UNIVERSAL COUCHTOP ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC