Posicast® Reloadable Thermoplastics

REFERENCE GUIDE

CIVCO
Radiotherapy™

CE

INTENDED USE

Device is intended for patient fixation.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

- Do not use if device appears damaged.
- Device should be used for a single patient throughout setup and treatment cycle.
- Ensure thermoplastic is properly aligned over patient before forming.
- Form thermoplastic to avoid contact with tracheotomy tube.
- Cutting portals in thermoplastic may reduce structural integrity.
- Do not sterilize thermoplastic.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.


Thermoplastic Type	Waterbath Temperature	Heating Time
Posicast® Thermoplastic	70°C (158°F)	1 min

USING THERMOPLASTICS

⚠ WARNING


- Ensure baseplate is secure on treatment platform.

1. Setup waterbath with 2–3" of water and heat (refer to table for appropriate temperature).
2. Insert thermoplastic into reloadable frame or profile (if applicable).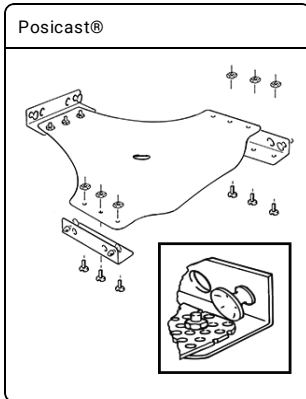

3. Place thermoplastic into bath for directed time.

NOTE: • Improper heating time may cause difficult application.
• Posicast® Thermoplastic with carbon fiber reloadable profiles: Do not submerge profiles.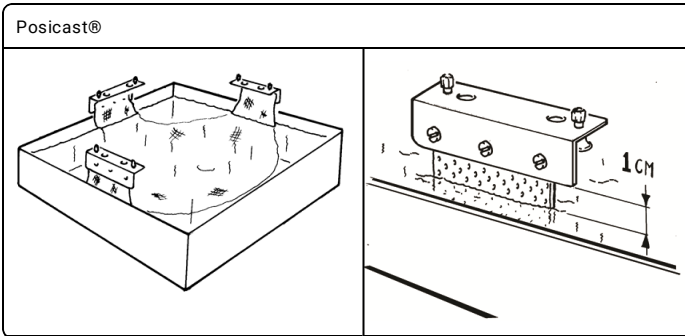


4. Remove thermoplastic from waterbath.
5. Quickly remove excess water from thermoplastic (*less than five seconds*).
6. Position thermoplastic over treatment area. Gently pull down to appropriate attachment base (*refer to attachment base instruction*).

NOTE: Overstretching of Posicast® material may affect rigidity of mask.

7. Use all available quickfasteners to latch thermoplastic to baseplate.
8. Gently mold material to contours of treatment area.
9. Allow thermoplastic to cool completely before releasing from attachment base.

RELEASING QUICKFASTENERS FOR POSICAST®

1. Press profile in the middle with one hand. Maintain this pressure through steps 1-4.
2. Push first quickfastener with other hand.
3. Pull first quickfastener "CLICK" should be audible.
4. Push and pull second quickfastener.
5. Release pressure from profile. Pull the profile from the baseplate in vertical direction.
6. Release all profiles one by one as described above.

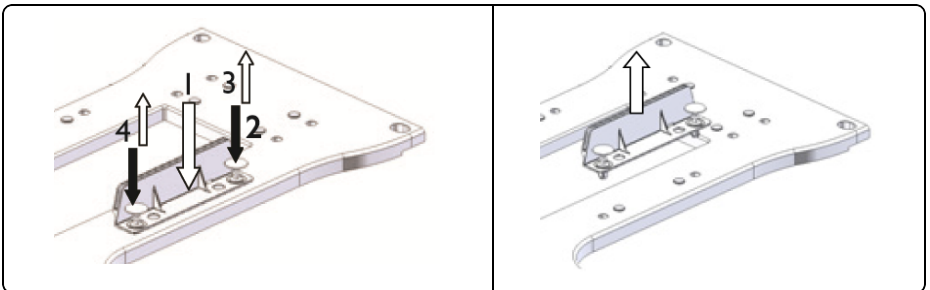NOTE: If it is difficult to remove the quickfasteners, use a toothpick or a cotton swab and carefully remove any residue in the quickfastener holes.

REPROCESSING

WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Reloadable frame: wipe with common germicidal or antiseptic wipe, such as alcohol.
2. Clean thermoplastic as needed with mildly warm water and soap.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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Radiotherapy™

 CIVCO RADIOTHERAPY

1401 8th Street SE
Orange City, IA 51041
United States

800.842.8688, +1 712.737.8688

info@civcort.com

COPYRIGHT © 2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVCO RADIOTHERAPY AND CIVCO RT ARE TRADEMARKS OF MEDTEC LLC. HIPFIX AND UNIFRAME ARE REGISTERED TRADEMARKS OF MEDTEC LLC. IMRT REINFORCED THERMOPLASTIC, TYPE-S AND ZENTEC ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.

WWW.CIVCORT.COM