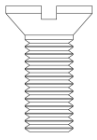
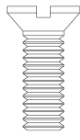
# SBRT Rails-Only Demonstration Kit

---

REFERENCE GUIDE

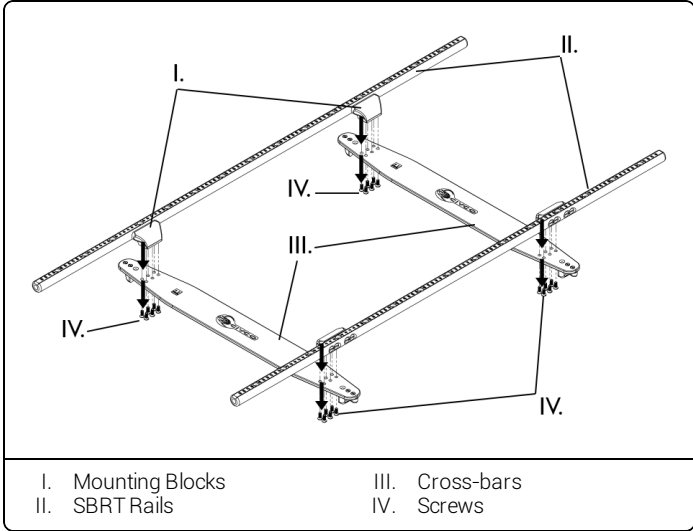
**CIVCO**

 <p>#10-24 x 1/2" for mounting cross-bar assembly to rails</p>	 <p>#10-32 x 3/4" for assembling Interloc® cross-bar</p>	 <p>#8-32 x 1/2" for assembling Prodigy™ cross-bar</p>
---	---	---

DISASSEMBLING RAILS-ONLY UNIT

1. Remove #10-24 x 1/2" screws (24) from mounting blocks on SBRT rail assembly.
2. Separate cross-bars from rails.

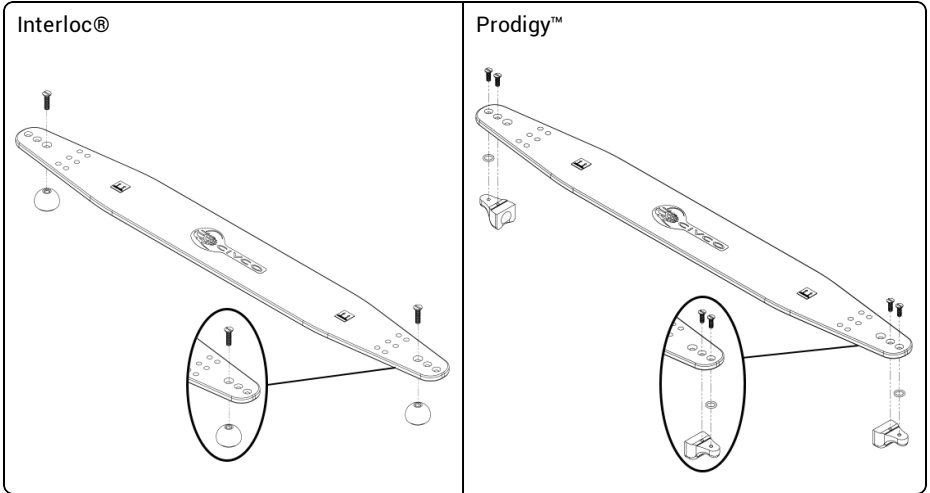
ASSEMBLING INTERLOC® / PRODIGY™ CROSS-BARS

**⚠ WARNING**

- Using incorrect screw to attach ends could result in device damage and/or patient mistreatment.

- Use appropriate screws to assemble cross-bar, ensure screws are in the correct locations (*refer to illustrations below*).
  - Interloc®: #10-32 x 3/4" screws
  - Prodigy™: #8-32 x 1/2" screws

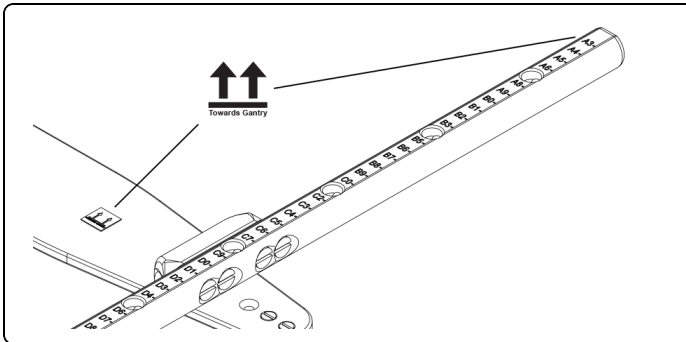
### ASSEMBLING RAILS-ONLY UNIT

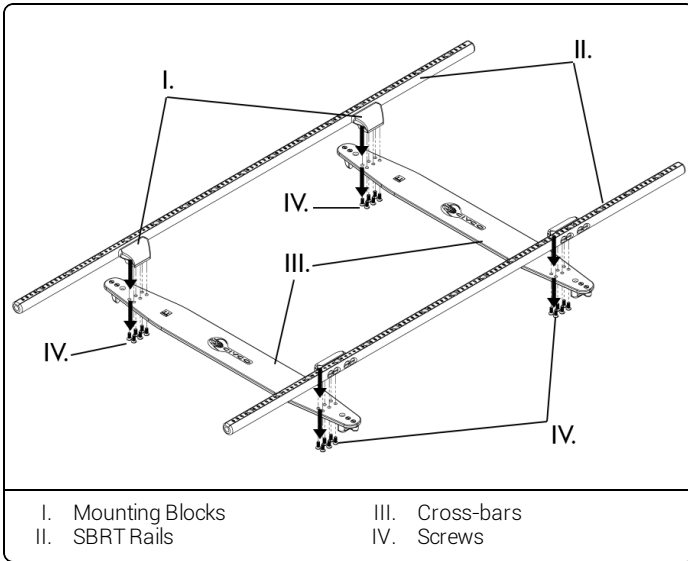
#### ⚠ WARNING

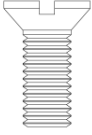
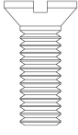
- Failure to assemble rails with proper screws or proper orientation could result in device damage and/or patient mistreatment.

- Ensure Towards Gantry labels on both cross-bars are aligned correctly with rail indexing (*refer to illustrations below*).

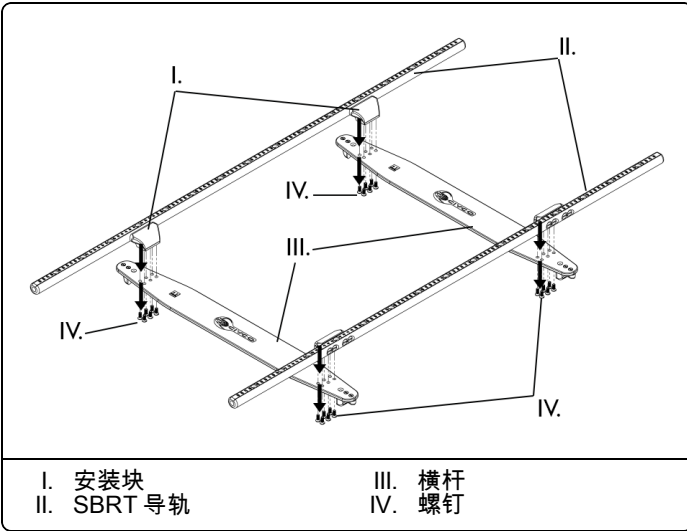
2. Attach desired cross-bars using #10-24 x 1/2" screws (24) into mounting blocks on SBRT rail assembly.

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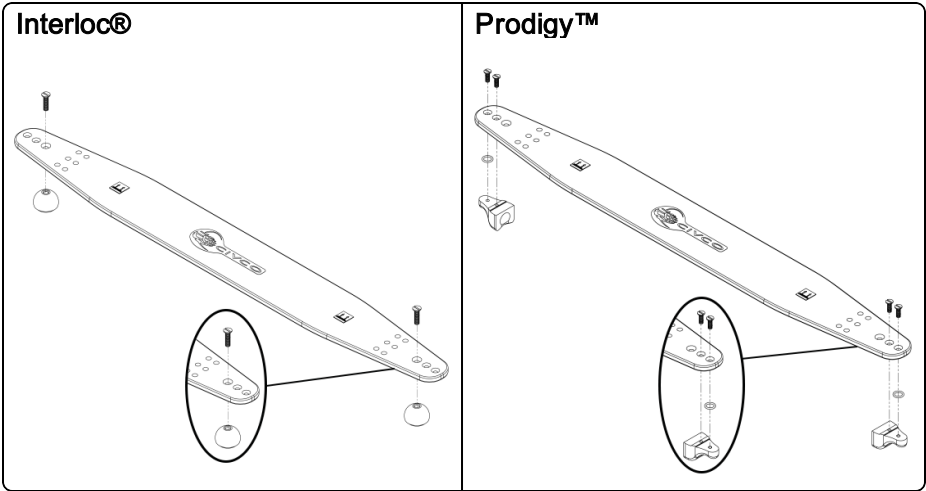
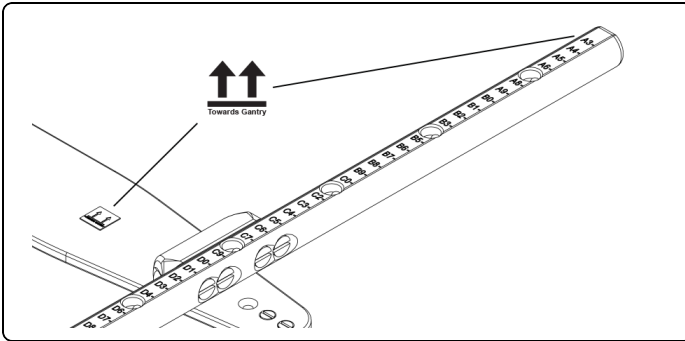
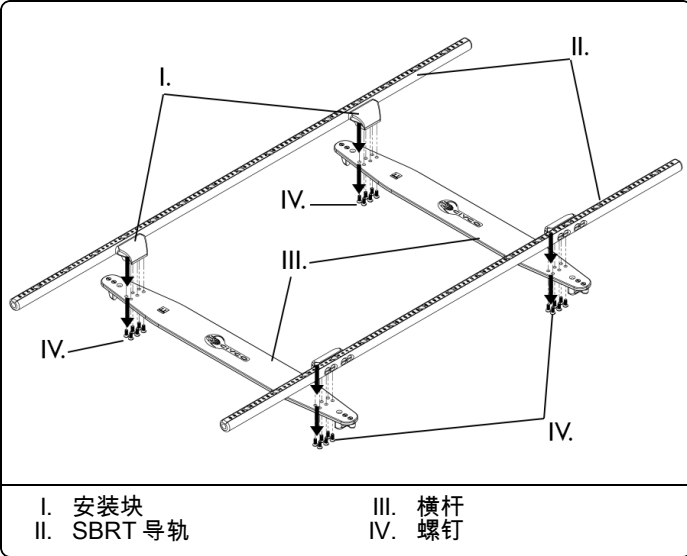
**备案人名称:** CIVCO Medical Solutions

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**备案人注册地址:** 1401 8th St. SE, Orange City, IA, 51041, United States

**生产地址:** 1401 8th St. SE, Orange City, IA, 51041, United States

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[info@civcort.com](mailto:info@civcort.com)

---

COPYRIGHT © 2019 ALL RIGHTS RESERVED. CIVCO, EXACT AND INTERLOC ARE REGISTERED TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. LOK-BAR AND PRODIGY ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.

[WWW.CIVCORT.COM](http://WWW.CIVCORT.COM)