Universal Couchtop Long Extension

For use with Elekta Precise, Elekta Synergy®, Siemens TXT, Siemens ZXT,

Siemens Definition MPT, Varian Exact®, Varian TrueBeam™ and CQ Medical Protura™ systems

INSTRUCTIONS FOR USE (IFU)

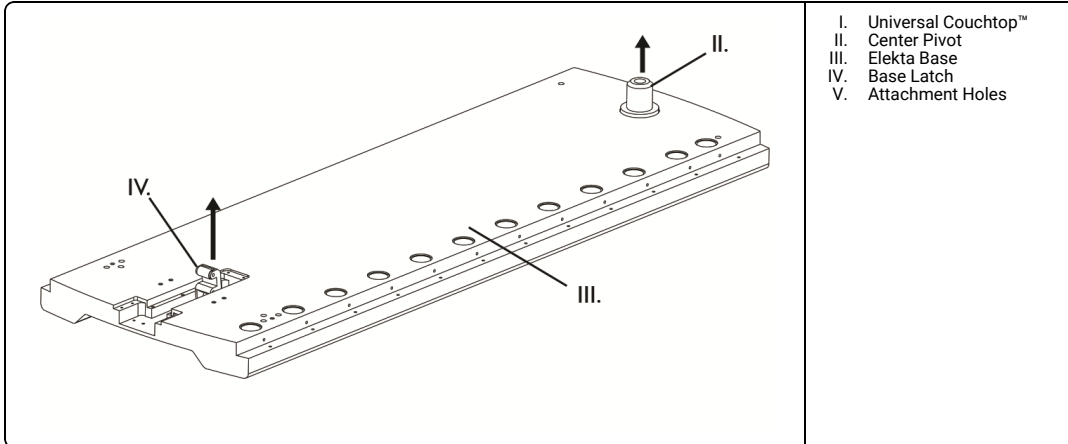
ATTACHING UNIVERSAL COUCHTOP™ TO ELEKTA PRECISE AND SYNERGY**⚠ WARNING**

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NOTE:


- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
- Tools Required: 6mm Hex Wrench and leveling tool.

1. Remove center pivot and latch.

2. Place couchtop on table and align attachment holes (6).

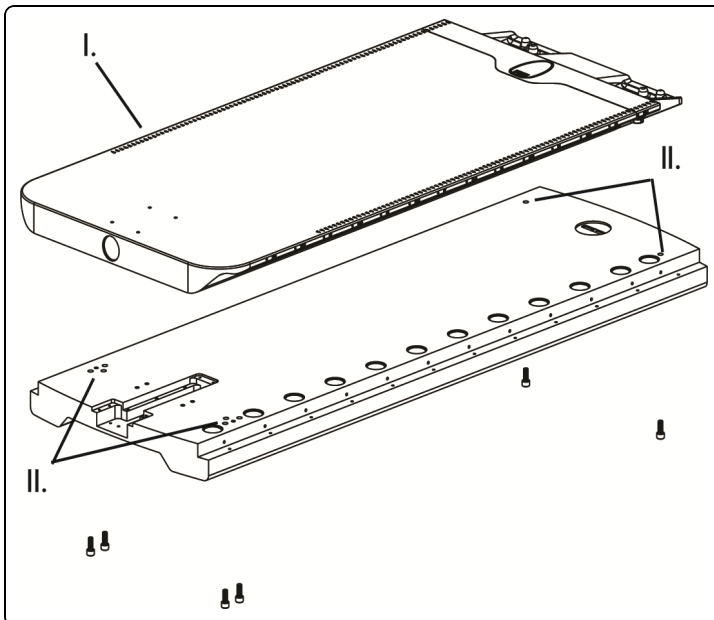3. Attach couchtop to table using hardware (provided). Torque bolts to 21 N-m (15.5 lb-ft).

4. Ensure junction of couchtop is level (refer to Leveling Junction section).

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ATTACHING UNIVERSAL COUCHTOP™ TO SIEMENS TXT AND CQ MEDICAL PROTURA™

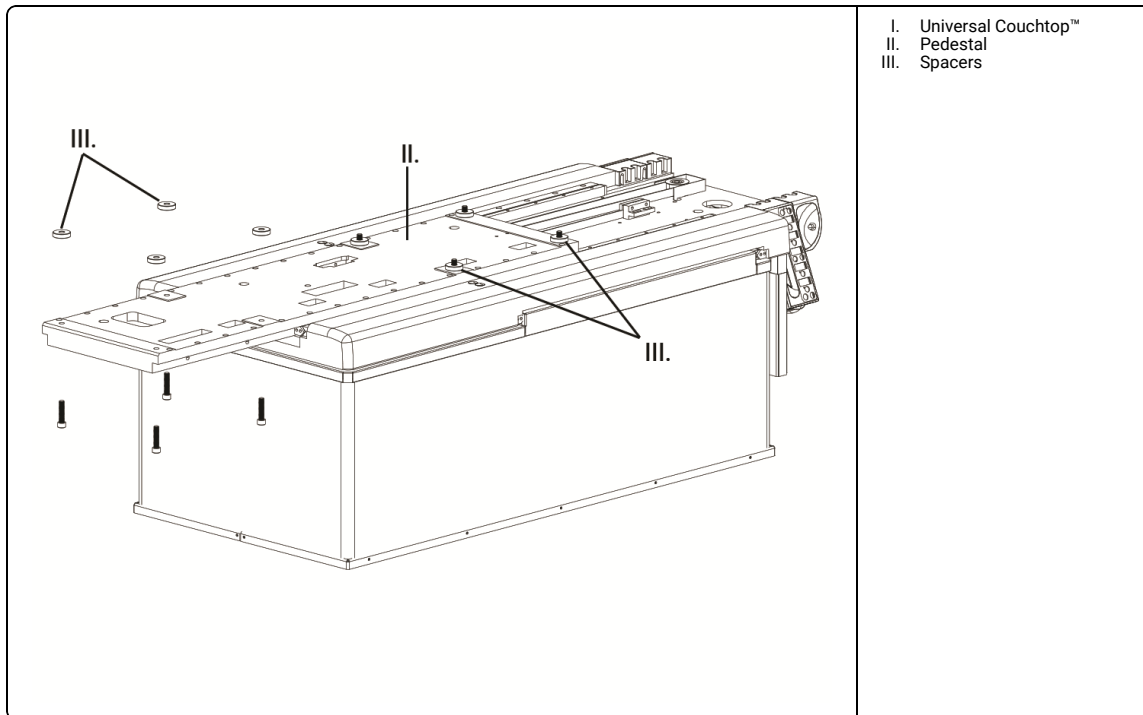
 WARNING

- Do not use if device appears damaged.


NOTE:

- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
- Tools Required: 8mm Hex Wrench, screwdriver and leveling tool.
- Remove existing top.
- Two (2) thinner spacers are for use on the superior mounting holes.
- Use hardware provided with Protura™ for installation on Protura™ unit.
- Universal Couchtop™ mounts to Protura™ using the same eight (8) mounting locations, screws come up through top plate of Protura™ instead of Siemens TXT pedestal.

1. Place spacers on pedestal at attachment locations.

2. Attach couchtop to table using hardware (provided). Torque bolts to 35 N·m (25.8 lb-ft).

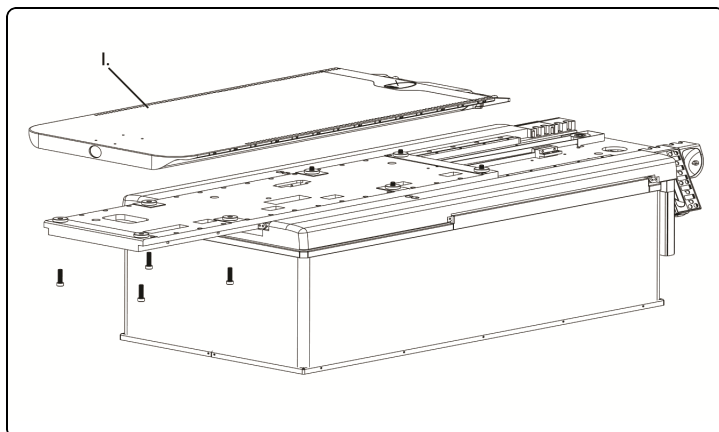
3. Ensure junction of couchtop is level (refer to Leveling Junction section).

NOTE: If couchtop is not level, remove couchtop and shim accordingly until level is achieved.

ATTACHING UNIVERSAL COUCHTOP™ TO SIEMENS ZXT

- NOTE:
- Tools Required: 5mm and 8mm Hex Wrenches, screwdriver and leveling tool.
 - Remove head extension, half panels, grid, center panel and all other accessories.
 - Two (2) thinner spacers are for use on the superior mounting holes.

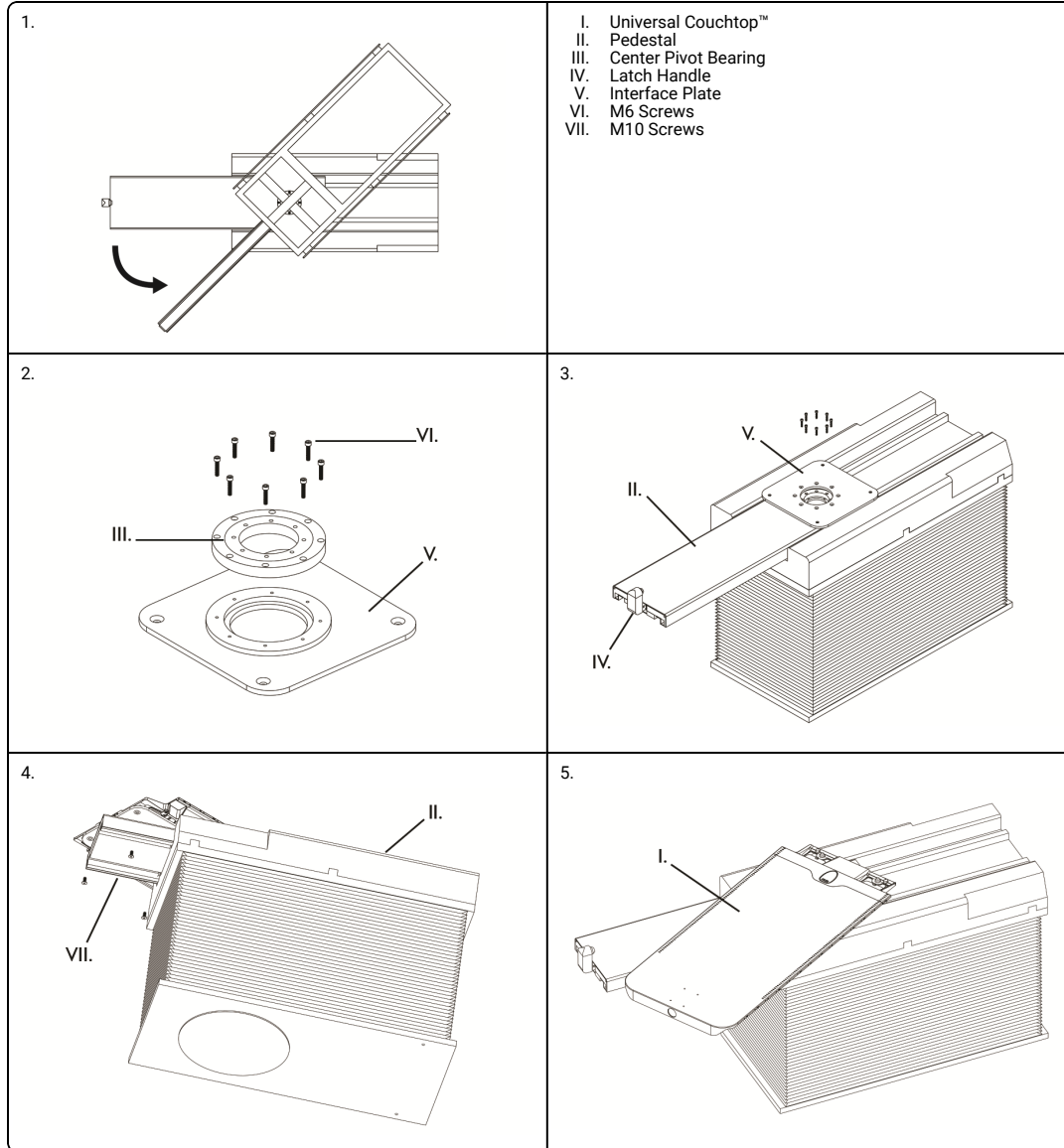1. Rotate table 45° to expose bolts and lift steel frame off of table.
2. Remove center pivot bearing from frame and refasten on interface plate.
3. Attach interface plate to table using hardware (*provided*). Torque bolts to 8.7 N·m (6.4 lb·ft).
4. Attach interface plate to couchtop using hardware (*provided*). Torque bolts to 35 N·m (25.8 lb·ft).
5. Rotate and lock couchtop onto table using latch handle.
6. Ensure junction of couchtop is level (*refer to Leveling Junction section*).

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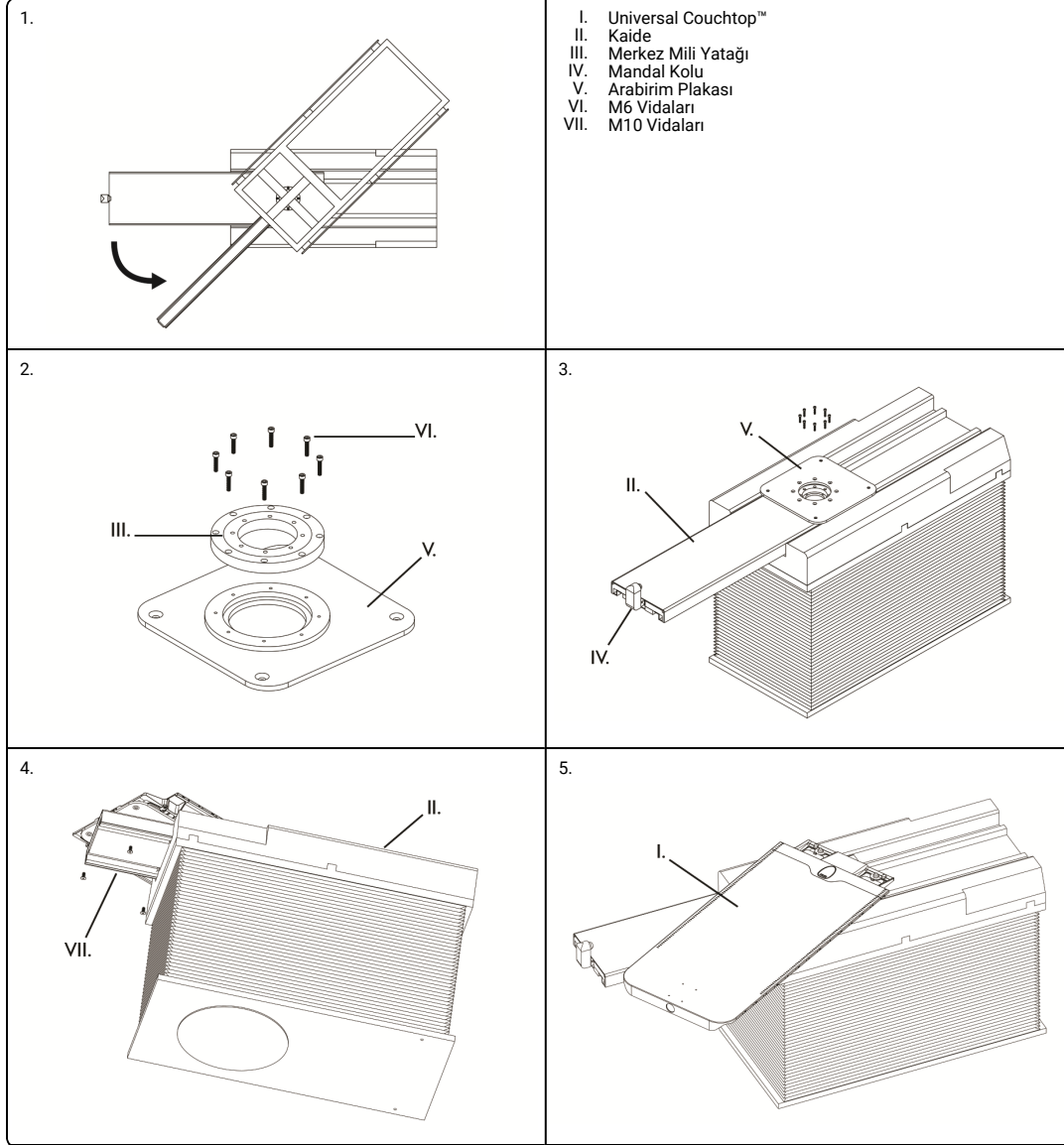
ATTACHING UNIVERSAL COUCHTOP™ TO VARIAN EXACT

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
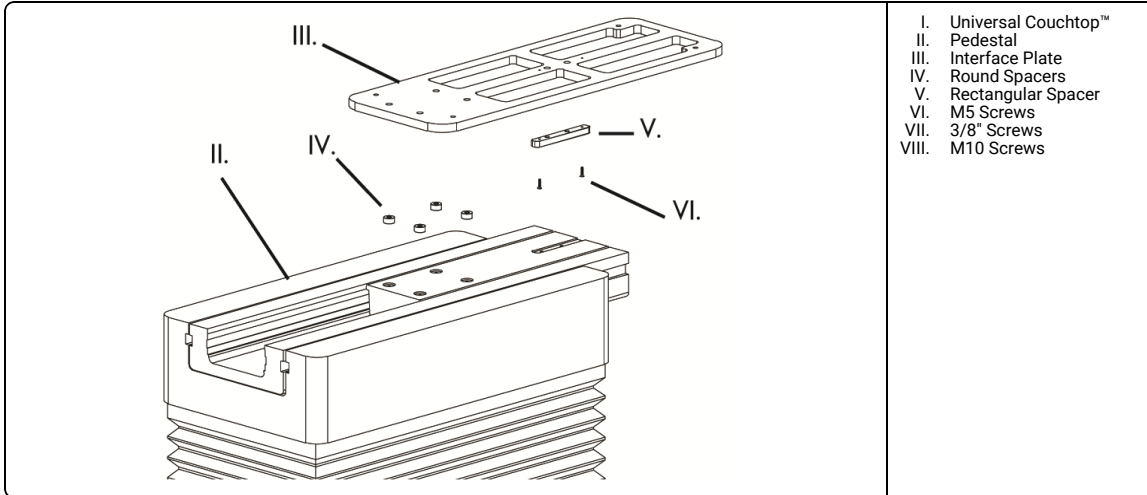
NOTE:

- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
- Tools Required: 5/16", 4mm and 8mm Hex Wrenches and leveling tool.
- Two (2) thinner spacers are for use on the superior mounting holes.


- Remove existing table and components, including frame.
- Attach rectangular spacer to interface plate using hardware (*provided*) and place round spacers onto pedestal.

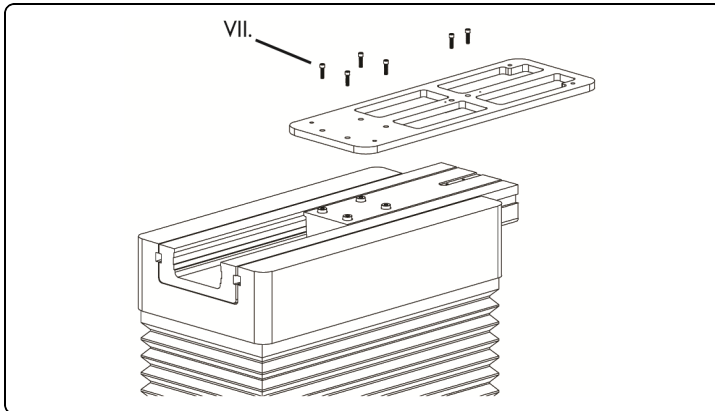- Attach interface plate to pedestal using hardware (*provided*). Torque bolts to 26.7 N-m (19.7 lb-ft).

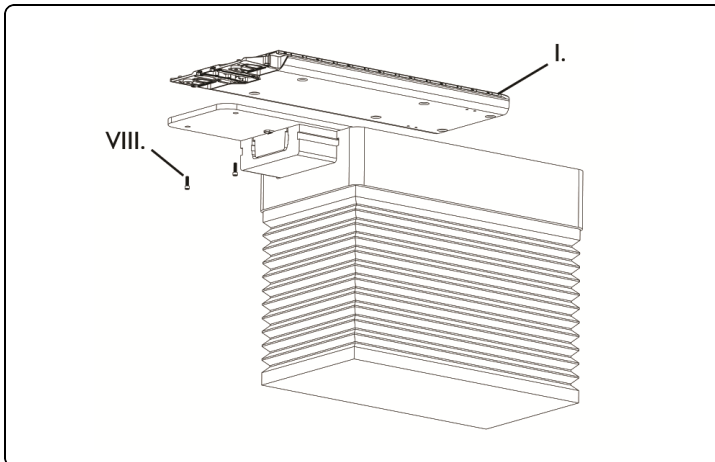- Attach couchtop to table using hardware (*provided*). Torque bolts to 35 N-m (25.8 lb-ft).

- Ensure junction of couchtop is level (*refer to Leveling Junction section*).

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ATTACHING UNIVERSAL COUCHTOP™ TO SIEMENS DEFINITION MPT

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
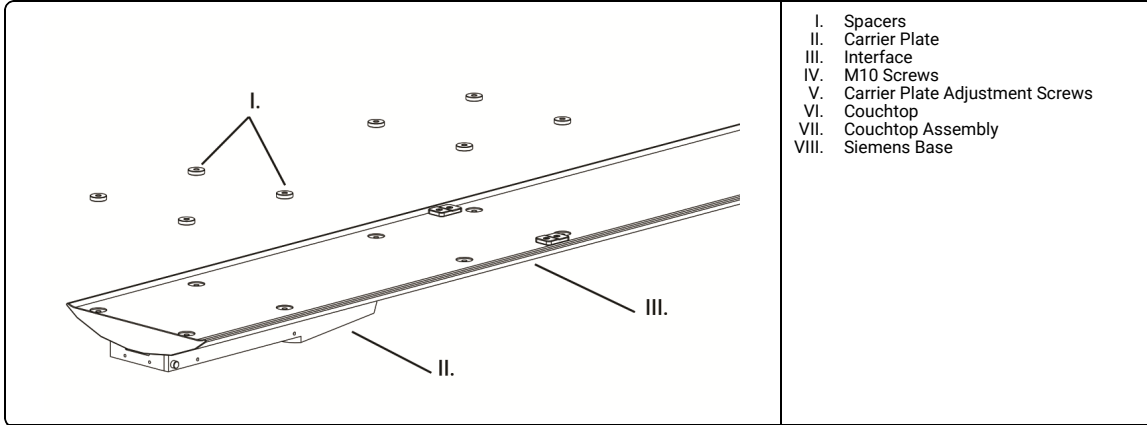
NOTE:

- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
- Tools Required: 8mm Hex Wrenches and leveling tool.
- Remove existing top.


1. Place spacers on interface at attachment locations.

2. Attach couchtop to interface using hardware (provided). Torque bolts to 35 N·m (25.8 lb·ft).

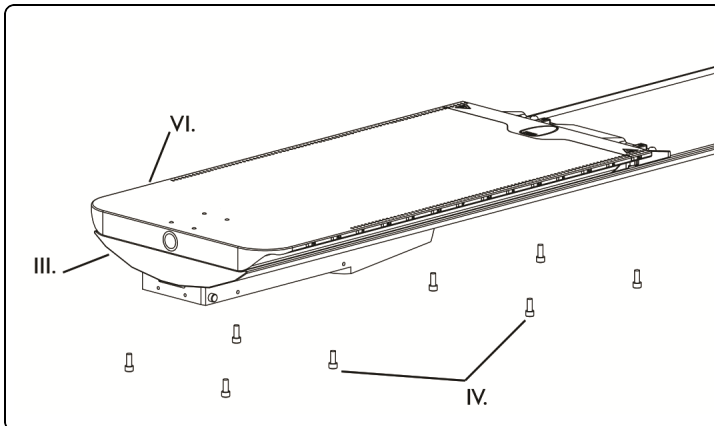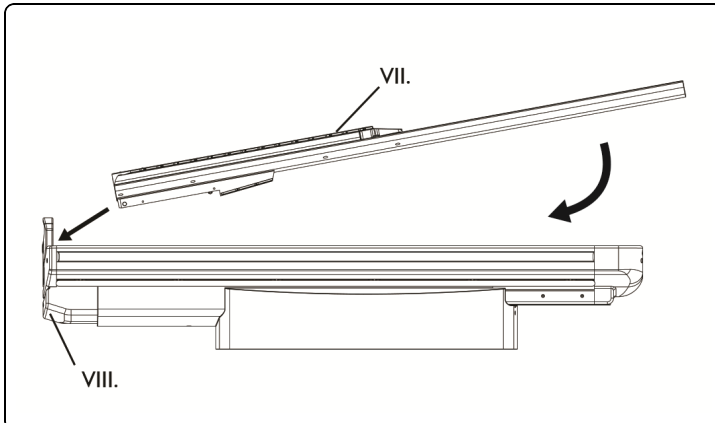
3. Place couchtop assembly onto Siemens base.

4. Ensure couchtop assembly fits properly on MPT. Verify couchtop is secure laterally. If lateral play is present, adjust carrier plate adjustment screws as needed to remove lateral play.

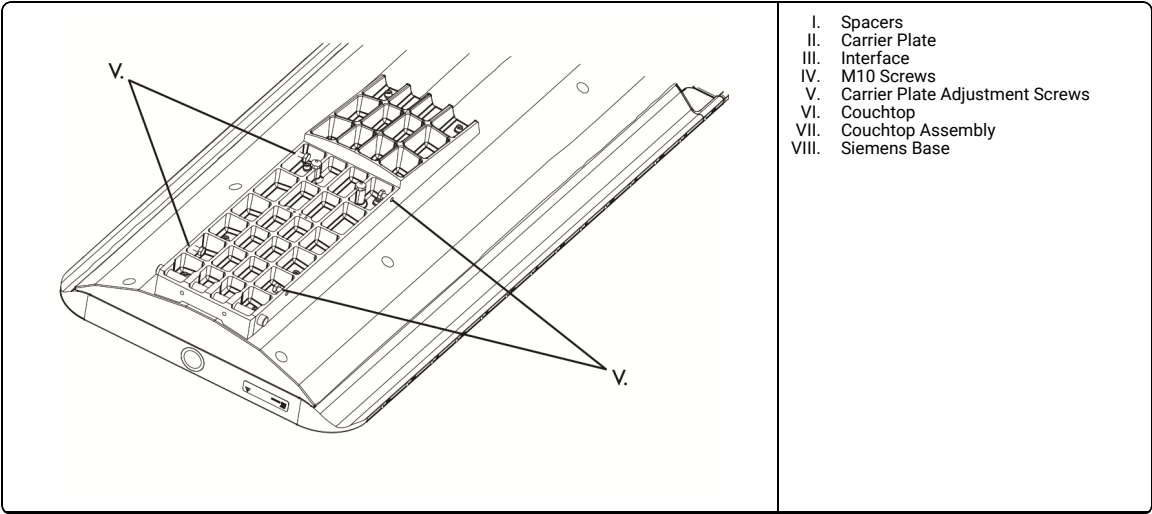NOTE: Screws can be used to make minor lateral and angular (yaw) adjustments if necessary.

5. Ensure junction of couchtop is level (refer to *Leveling Junction* section).
NOTE: If couchtop is not level, remove couchtop and shim at spacer locations between interface and couchtop accordingly until level is achieved.

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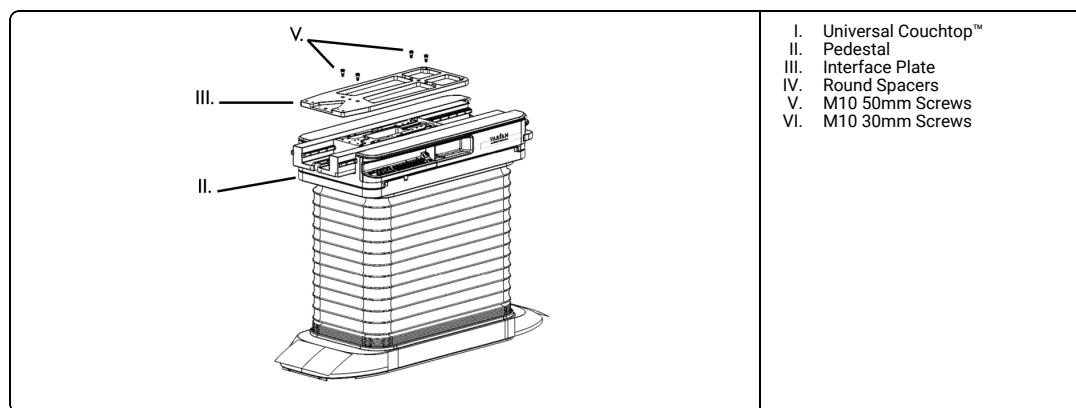
ATTACHING UNIVERSAL COUCHTOP™ TO VARIAN TRUEBEAM™ 4 DEGREES OF FREEDOM BASE

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

- Varian service representative must verify LaserGuard collision detection is installed and functioning and TrueBeam™ system software is v2.7+.
- Varian service representative will remove the existing couchtop.
- Attach interface plate to pedestal using hardware (*provided*). Torque bolts to 35 N·m (25.8 lb·ft).

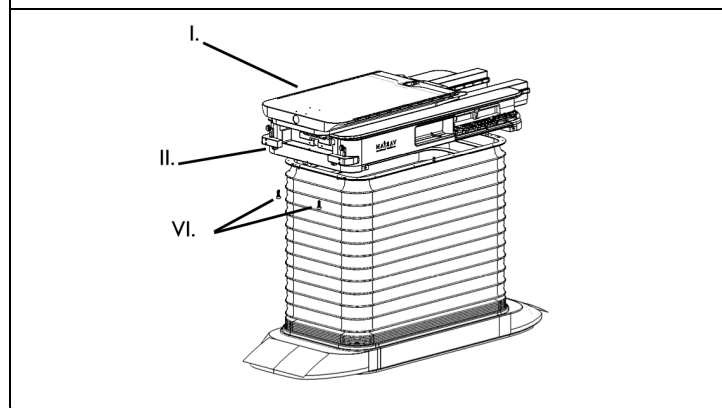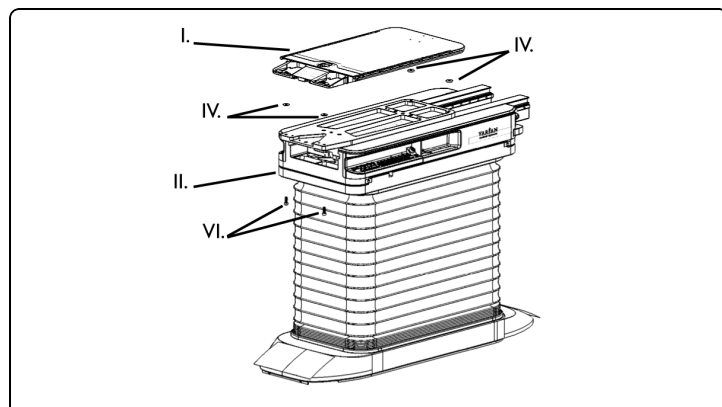- Attach couchtop to interface using hardware (*provided*). Torque bolts to 35 N·m (25.8 lb·ft).

- Ensure junction of couchtop is level (*refer to Leveling Junction section*).
- Varian service representative will configure TrueBeam™ system administration (*encrypted HASP access required*).
- Varian service representative will re-calibrate the couch (*encrypted HASP access required*) and perform Installation Product Acceptance with customer before TrueBeam™ can be returned to clinical use.

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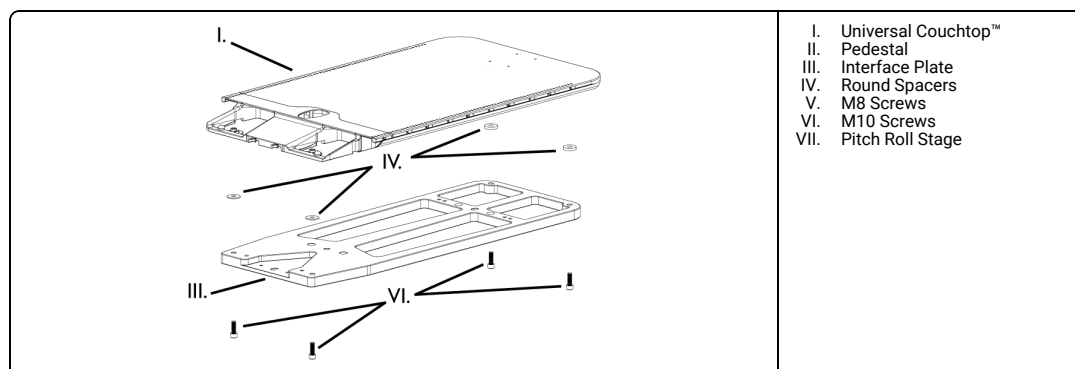
ATTACHING UNIVERSAL COUCHTOP™ TO VARIAN TRUEBEAM™ 6 DEGREES OF FREEDOM BASE

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

- NOTE:
- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
 - When requesting a Varian service representative for UCT installation, refer to Varian procedure: STB-PS-518 Install CQ Medical Universal Couchtop™ (UCT) on TrueBeam™ .
 - Tools Required: 6mm and 8mm Hex Wrenches and leveling tool.
 - Two (2) thinner spacers are for use on the superior mounting holes.
 - UCT Base Weight: 14 lbs (6.4 kg).
 - Interface Plate Weight: 27 lbs (12.3 kg).

- Varian service representative must verify LaserGuard collision detection is installed and functioning and TrueBeam™ system software is v2.7+.
- Varian service representative will remove the existing couchtop.
- Attach couchtop to interface using hardware (*provided*). Torque bolts to 35 N·m (25.8 lb·ft).

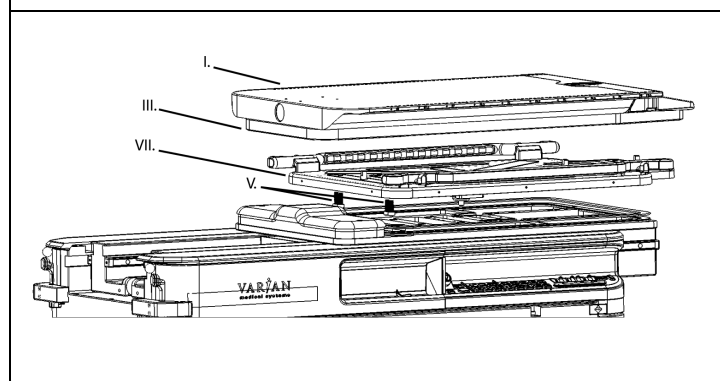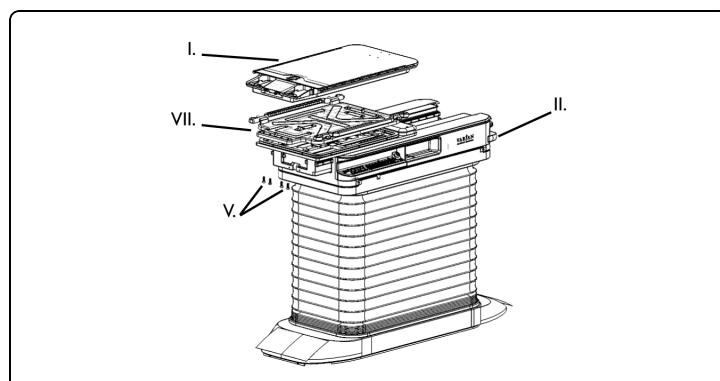- Remove rubber gasket from bottom of interface plate (*rubber gasket only required for 4 degrees of Freedom Base*).
- Lower bellows from pitch roll stage for access to inferior mounting holes.
- Attach interface plate and couchtop to pedestal using hardware (*provided*). Torque bolts to 25 N·m (18.4 lb·ft).

- Varian service representative will configure TrueBeam™ system administration (*encrypted HASP access required*).
- Varian service representative will re-calibrate the couch (*encrypted HASP access required*) and perform Installation Product Acceptance with customer before TrueBeam™ can be returned to clinical use.

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

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

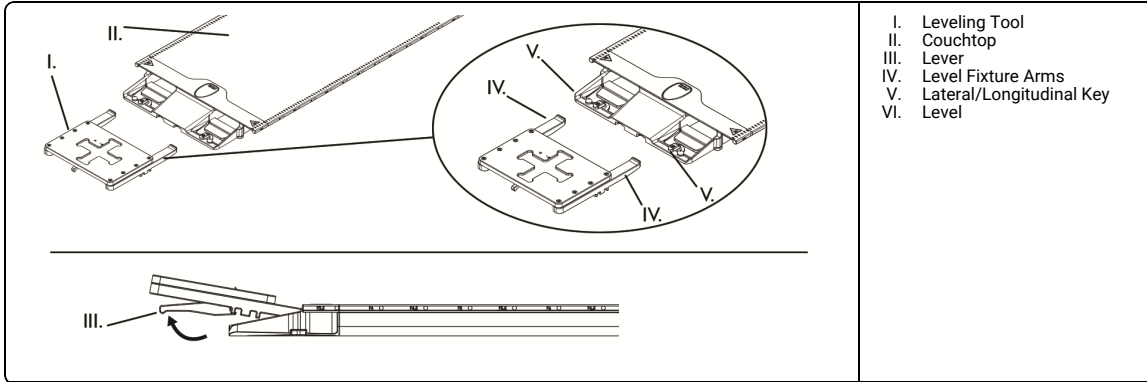- Place leveling tool on couchtop while holding lever underneath.

- Ensure Level Fixture Arms are placed over the lateral/longitudinal key.
- Once leveling tool is in place release lever.
- Place level (provided) into recess on leveling tool.
- Ensure the bubble falls between lines (indicated on image) to verify the junction is level.
- Check level in both longitudinal and lateral directions.

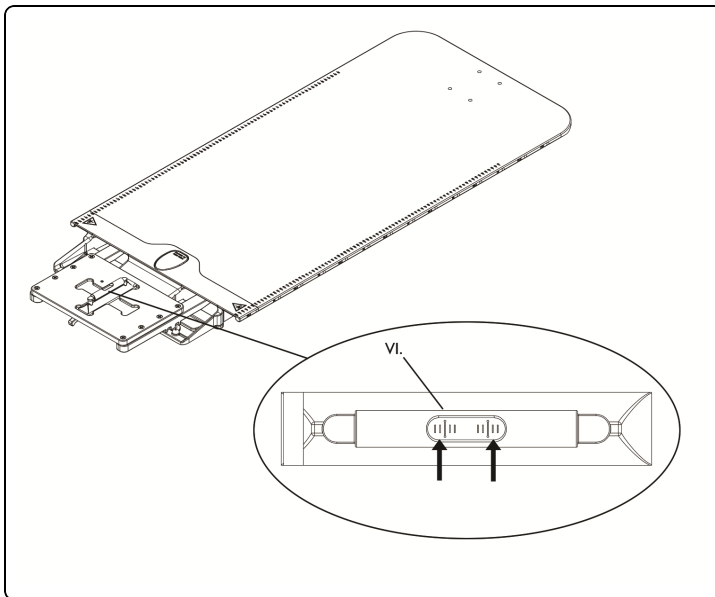NOTE: If Junction is not level, remove couchtop and shim accordingly until level.

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COPYRIGHT © 2021 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. IPPS, LOK-BAR, PRODIGY, AND TYPE-S ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC