Universal Couchtop™ Long Extension

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INTENDED USE

The device is intended to support and aid in positioning a patient during radiologic and other medical procedures.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

**WARNING**

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not apply a load exceeding 484 lbs (220 kg).
- Phantom Mount: Do not apply a load exceeding 66 lbs (30kg).
- Ensure extension is in locked position by lifting gently on superior end.

NOTE: • If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.


 • Avoid scanning in junction region of couchtop, artifacting may occur (refer to extension user manual).

 • Couchtop must be used in conjunction with an extension when imaging patients (refer to extension user manual).

ATTACHING AND REMOVING COUCHTOP

NOTE: Use two people to attach/remove couchtop from base.

1. Place couchtop assembly onto Siemens base.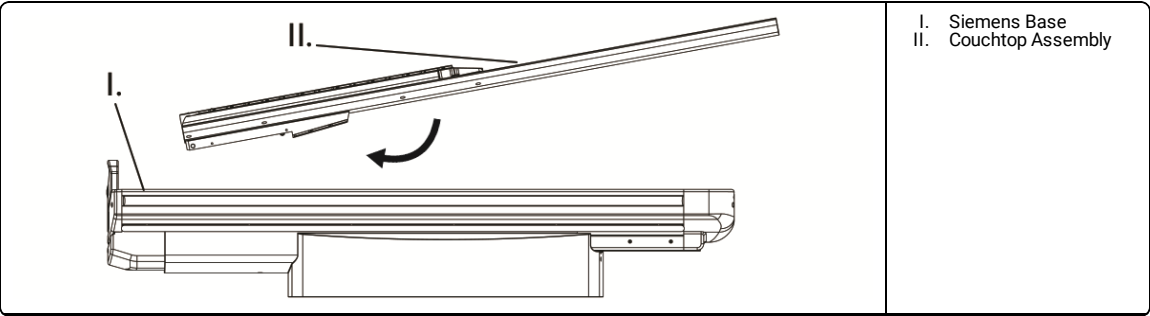

2. Remove couchtop assembly from base.


ATTACHING PHANTOM MOUNT

1. Slide phantom mount onto superior end of couchtop assembly.
2. Latch both levers to secure mount.

**WARNING**


- Ensure device is secure prior to use.

NOTE: Prior to patient positioning and imaging, remove phantom mount.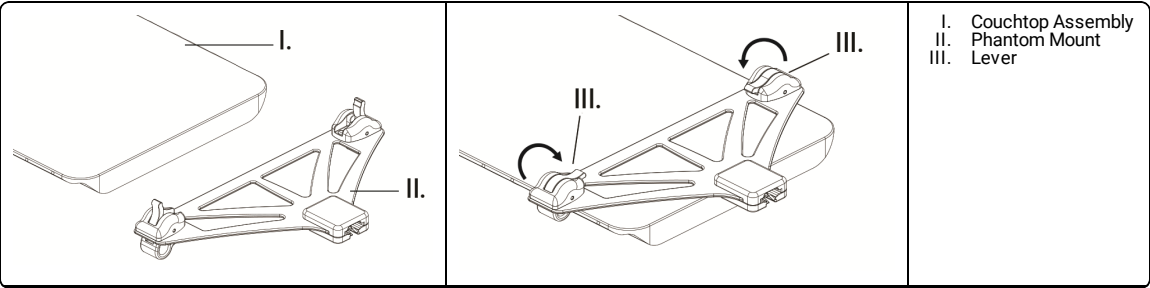

**WARNING**

- Avoid pinch points when table is in motion by ensuring patient does not grasp couchtop sides.
- Potential pinch point occurs between couchtop and gantry when scanning with the table in lowest position.

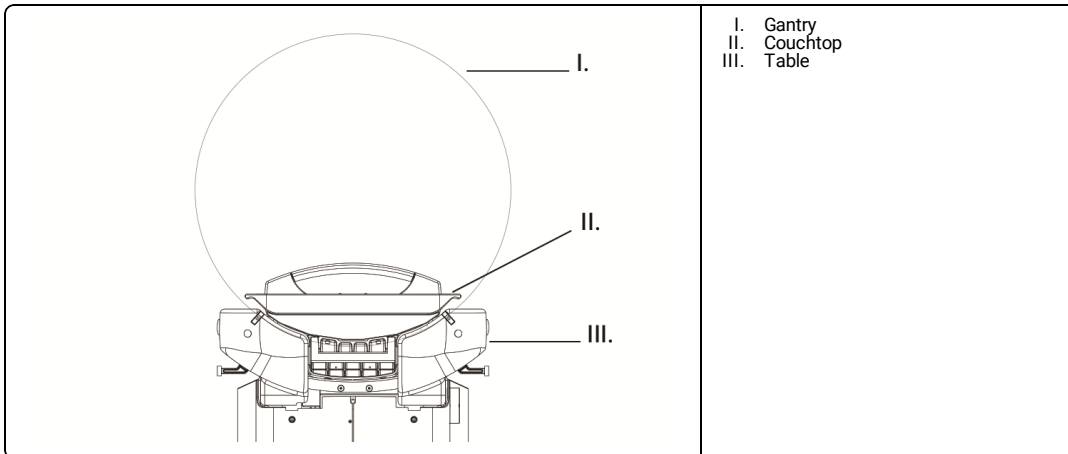**REPROCESSING****WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. PRODIGY AND UNIVERSAL COUCHTOP ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC