ONEPlatform™

Body Pro-Lok™ System

INSTRUCTIONS FOR USE (IFU)

Towards Gantry

Towards Gantry

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Kızağa Doğru

INTENDED USE

The device is intended to aid in support and positioning of patients during MR, radiological and other procedures.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Ensure all Body Pro-Lok™ accessories are properly attached using Lok-Bars or bridge latching mechanisms.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at www.CQmedical.com.
- Ensure patient is set up in correct position before beginning treatment.
- Ensure patient remains still for duration of setup and treatment.
- Ensure clearance is maintained between Body Pro-Lok™ system and all linac or imaging equipment before using system.
- Do not exceed patient weight of 500 lbs (226.8 kg).
- Do not use Lok-Bar™ to secure devices.
- For more than one platform, ensure all platform lateral scale indexing markers align. If platforms have opposite orientation, contact your CQ Medical representative.
- Do not attempt to move or carry platform by accessory bridges.

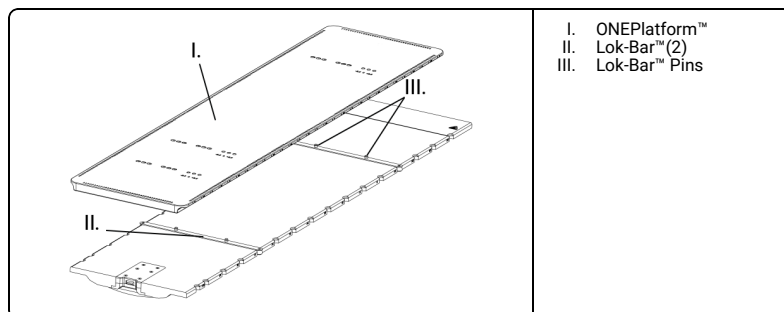
NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

MRI SAFETY INFORMATION

MR Safe

- Device is MR safe.

1. Place ONEPlatform™ onto Lok-Bar™ pins.

**REPROCESSING****⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. BODY PRO-LOK, LOK-BAR, ONEPLATFORM ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.