# Body Pro-Lok™ Bridges

Body Pro-Lok™ System

---

REFERENCE GUIDE

**CIVCO**

CE

**Towards Gantry**

Towards Gantry

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Naar rijbrug

Pöytää kohti

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Kızağa Doğru

## INTENDED USE

The device is intended to aid in support and positioning of patients during MR, radiological and other procedures.

## CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

### ⚠ WARNING

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Ensure all Body Pro-Lok™ accessories are properly attached using T-pin Lok-Bars or bridge latching mechanisms.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at [WWW.CIVCORT.COM](http://WWW.CIVCORT.COM).
- Verify patient position with completed setup sheet prior to treatment.
- Ensure patient remains still for duration of setup and treatment.
- Do not use Lok-Bar™ to secure positioning devices.
- Monitor patient constantly when using accessories to ensure patient comfort.
- Ensure clearance is maintained between Body Pro-Lok™ system and all linac or imaging equipment before using system.
- Do not attempt to move or carry platform by accessory bridges.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

## MRI SAFETY INFORMATION

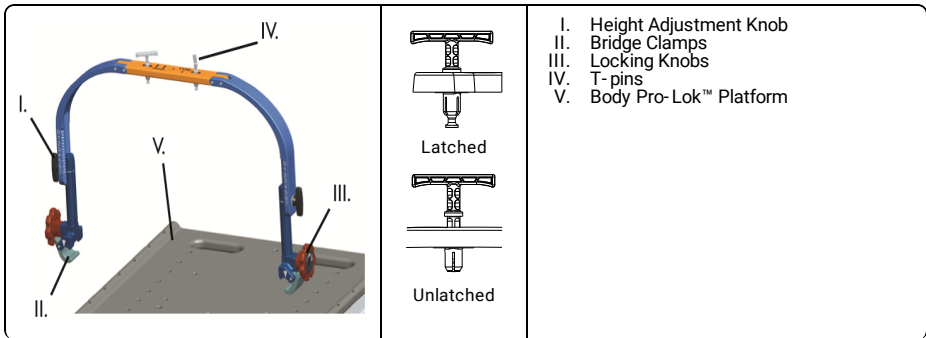
- Device is MR safe.

MR Safe

1. Loosen height adjustment knobs.
2. Raise Body Pro-Lok™ Bridge height adjustment to highest position and tighten.
3. Turn Body Pro-Lok™ Bridge locking knobs to unlatched position.
4. Place bridge clamps onto Body Pro-Lok™ Platform.
5. Turn Body Pro-Lok™ Bridge locking knobs to latched position, ensuring bridge is securely affixed.
6. Adjust bridge height to desired position.

NOTE: Note index marks for desired position.

## REPROCESSING

### ⚠ WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

## MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2020 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. BODY PRO-LOK AND LOK-BAR ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.