# C-Qual™ Breastboard

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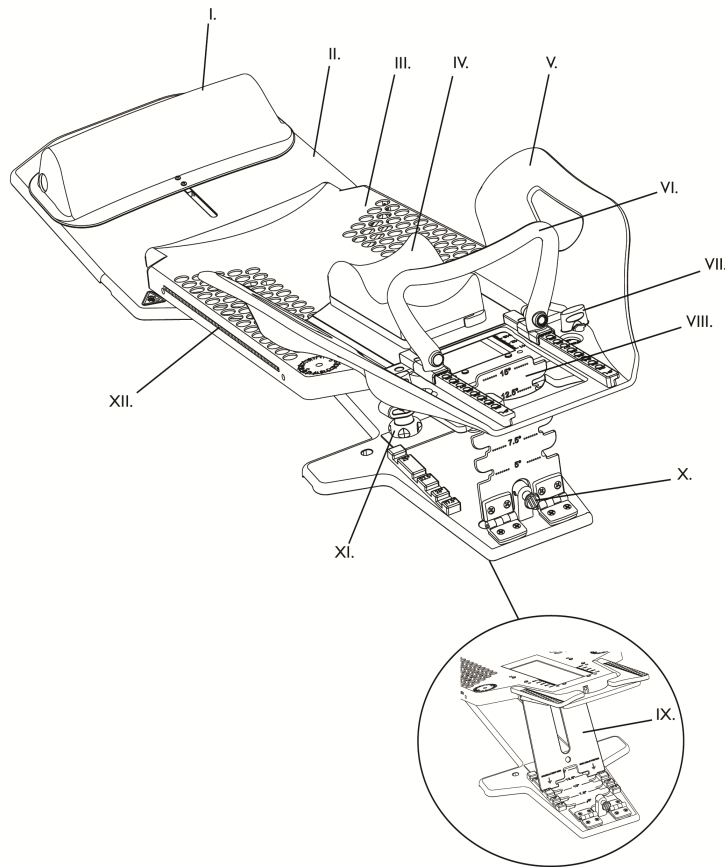
**INTENDED USE**

The device is intended to support and aid in positioning a patient during radiologic and other medical procedures.

**CAUTION**

Federal (United States) law restricts this device to sale by or on the order of a physician.

**⚠ WARNING**

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Ensure all accessories are attached securely prior to treatment.
- Do not allow patient to reposition themselves.
- Ensure patient remains still for duration of setup and treatment.
- Lok-Bar™ is a positioning device only, do not use to secure patient.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at [www.CQmedical.com](http://www.CQmedical.com).
- Verify patient position with completed setup sheet prior to treatment.
- Do not exceed patient weight of 330 lbs. (150 kg).
- Do not exceed 200 lbs (91 kg) on wings.
- To prevent damage, grasp both the tilting board and lower board when handling.
- Do not reposition device with patient on it.
- Verification image should be taken to confirm patient position prior to treatment.
- Do not allow patient to reposition using hand grip as these are not intended to be load bearing.
- Only use CQ Medical compatible accessories.
- This product is manufactured with carbon fiber, and rough handling may cause damage resulting in fibers coming off. If this happens, stop using the product to prevent fibers from getting in contact with the patient's or user's skin.

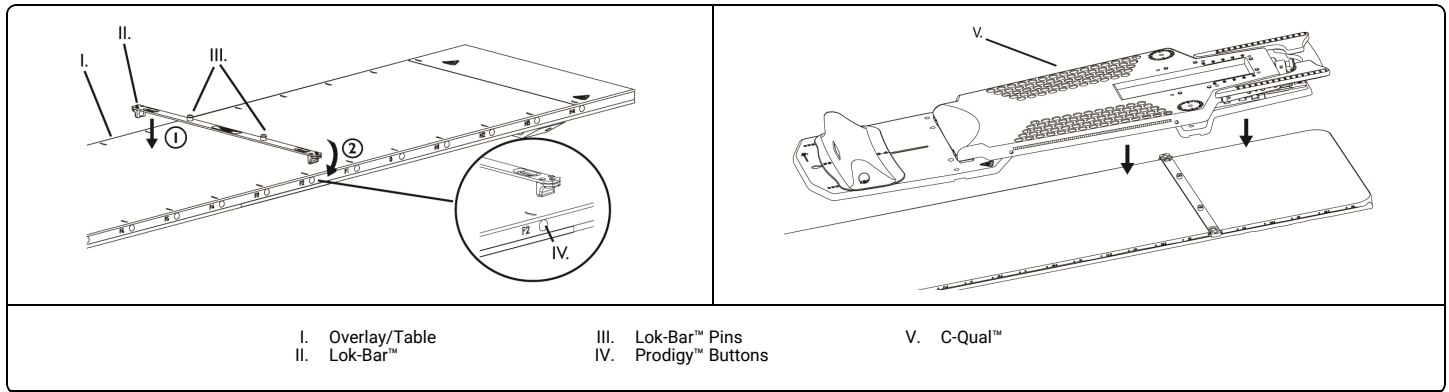
NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

**ATTACHING C-QUAL™ TO TABLE**

**⚠ WARNING**


- Lok-Bar™ is a positioning device only, do not use to secure patient.

1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place C-Qual™ onto Lok-Bar™ pins carefully to prevent damage (refer to Lok-Bar™ instruction). Ensure C-Qual™ and Lok-Bar™ are secure.

**ATTACHING MONARCH™ OVERHEAD ARM POSITIONER AND HEAD SUPPORT TO C-QUAL™**

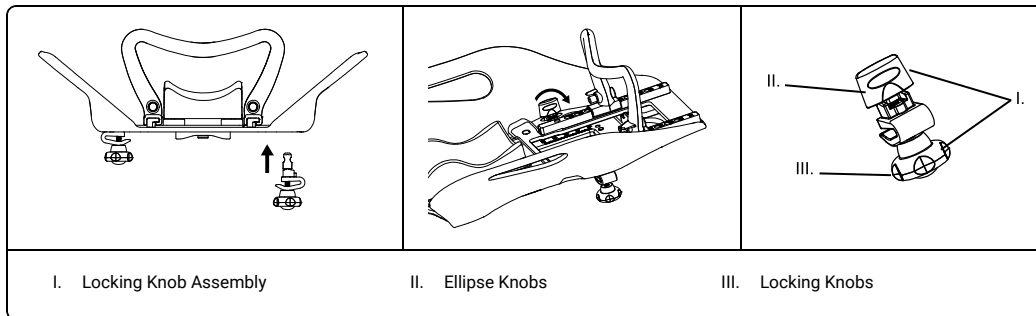
1. Insert locking knob assemblies as shown.
2. Turn ellipse knobs 90 degrees to secure.

NOTE: Ensure Monarch™ locking knobs are not overtightened to allow attachment.

3. With Upper Board in locked position, slide Monarch™ onto C-Qual™.
4. Secure locking knobs.

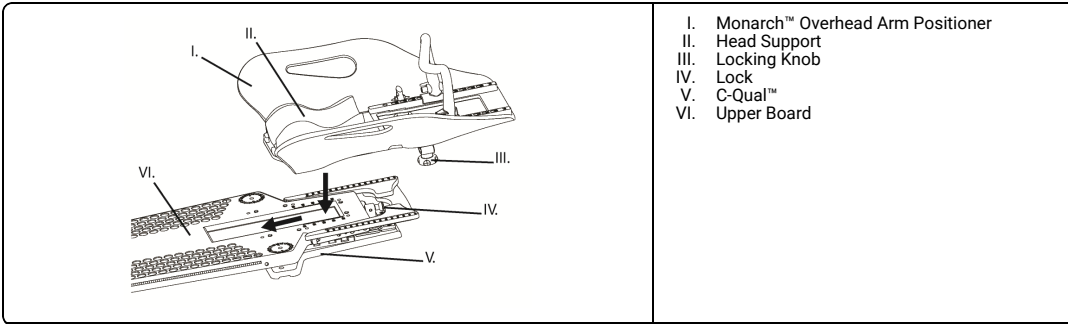
5. Place desired head support into recessed area of Monarch™.  
 NOTE: Ensure to not overtighten locking knobs.

- I. Monarch™ Overhead Arm Positioner
- II. Head Support
- III. Locking Knob
- IV. Lock
- V. C-Qual™
- VI. Upper Board

**ADJUSTING C-QUAL™**

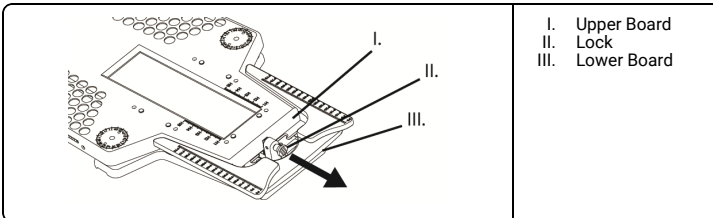1. Pull lock to unlock upper board from lower board.  
 NOTE: Monarch™ not shown.


- I. Upper Board
- II. Lock
- III. Lower Board

2. Position upper board.


**⚠ WARNING**

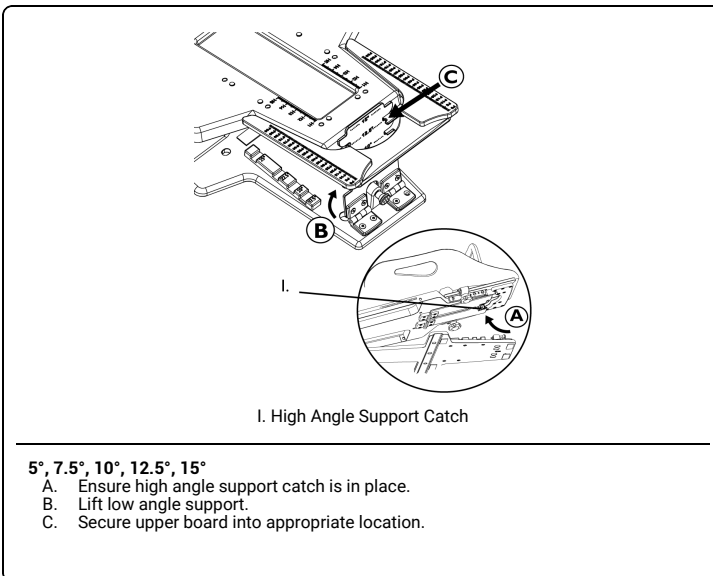- Ensure upper board is fixed properly before placing patient.

NOTE: Monarch™ not shown.

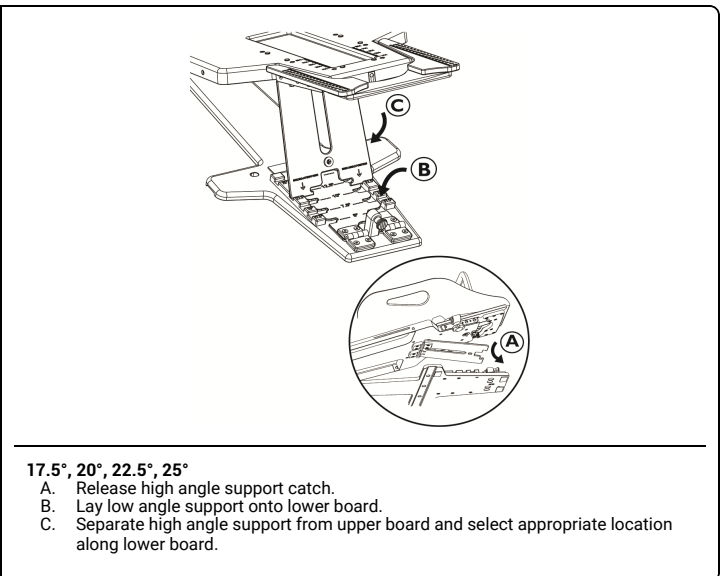
- 5°, 7.5°, 10°, 12.5°, 15°**  
 A. Ensure high angle support catch is in place.  
 B. Lift low angle support.  
 C. Secure upper board into appropriate location.


- 17.5°, 20°, 22.5°, 25°**  
 A. Release high angle support catch.  
 B. Lay low angle support onto lower board.  
 C. Separate high angle support from upper board and select appropriate location along lower board.

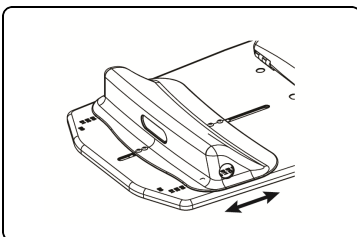3. Slide or lift on bottom stop to achieve appropriate location (B0-B10).


4. Head and breast thermoplastics can be attached to C-Qual™ by inserting into attachment points (See thermoplastics instruction).
5. Adjust Monarch™ as needed.
- 6.

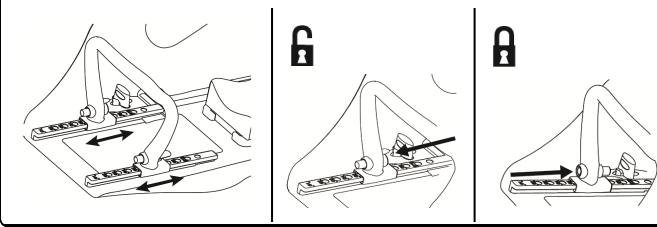Adjust hand grip as needed.

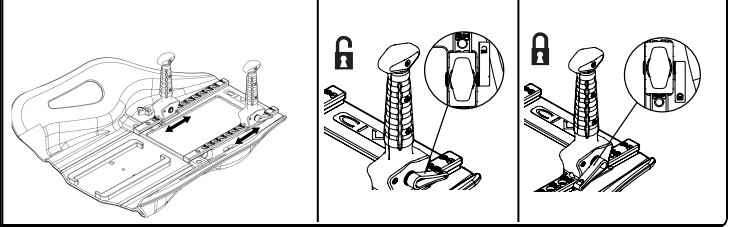Adjust Monarch™ Individual Hand Grips as needed.

NOTE: To use head and neck thermoplastic remove Monarch™ prior to attachment.

**REMOVING MONARCH™ OVERHEAD ARM POSITIONER FROM C-QUAL™**

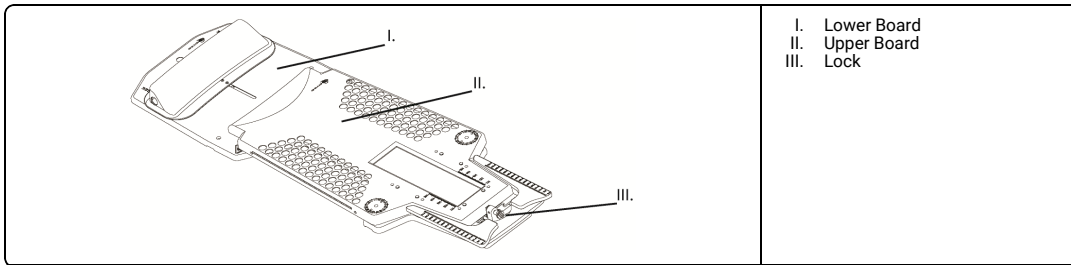
NOTE: Prior to removing Monarch™, lock upper board to lower board.

1. Loosen locking knobs on Monarch™.
2. Slide Monarch™ clear of locking brackets and lift off of C-Qual™.

**REMOVING C-QUAL™ FROM COUCHTOP**


NOTE: If Monarch™ is attached during removal of C-Qual™ to couchtop, ensure locking knobs on Monarch™ are securely tightened.

1. Lock upper board to lower board and lift to remove.


**ATTACHING MONARCH™ OVERHEAD ARM POSITIONER TO TABLE WITH LOK-BAR™**

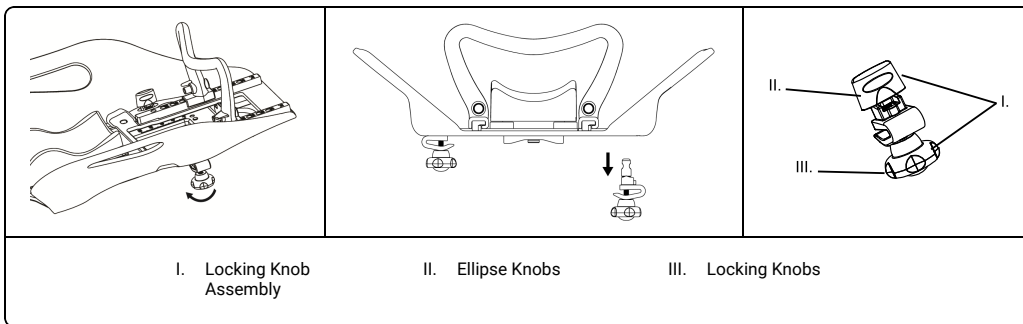
1. Completely loosen locking knobs on Monarch™.
2. Pull up on ellipse knobs and turn 90 degrees.
3. Remove locking knobs assembly.

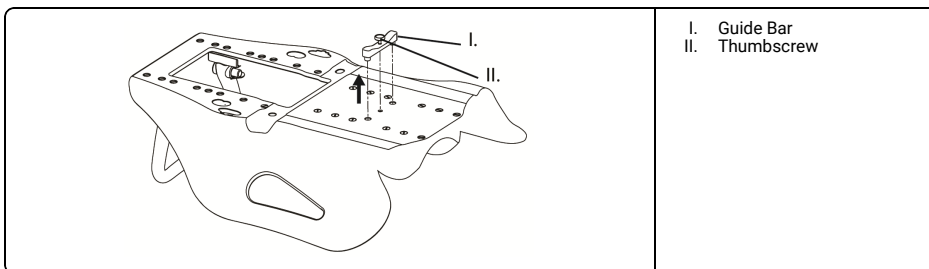
4. Remove guide bar assembly.

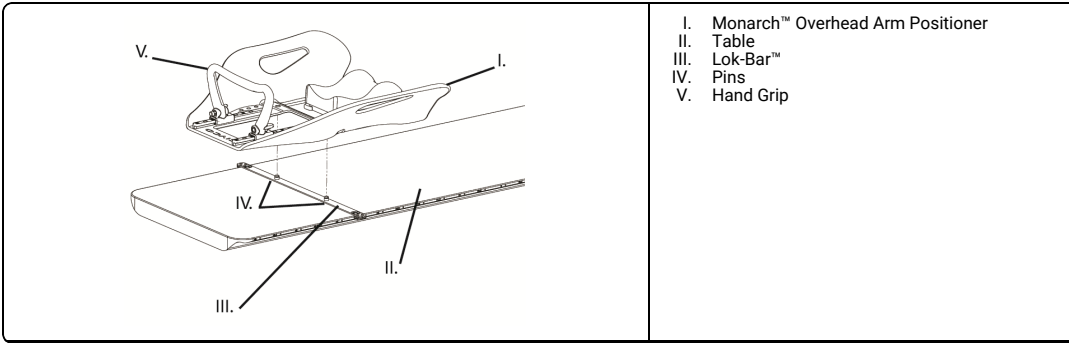5. Place Monarch™ onto Lok-Bar™ pins (refer to Lok-Bar™ instruction).

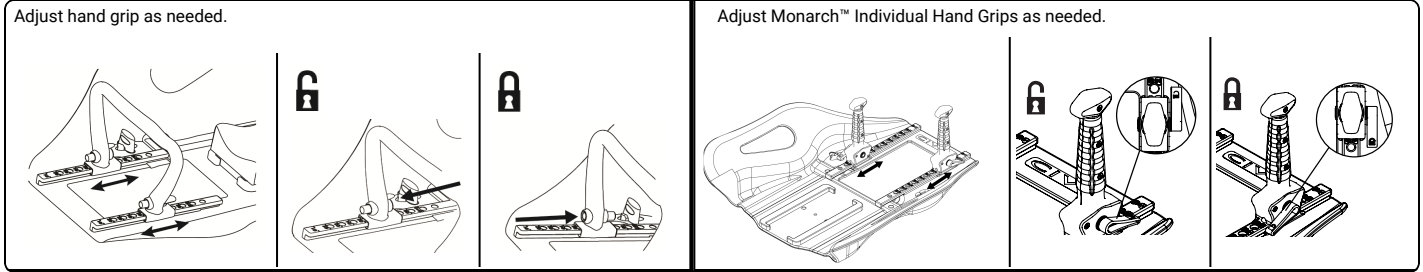6.

**REPROCESSING**

**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.
- Inspect device for damage such as cracks or breakage. If damage is evident, discontinue use and contact CQ Medical for further instructions as needed.

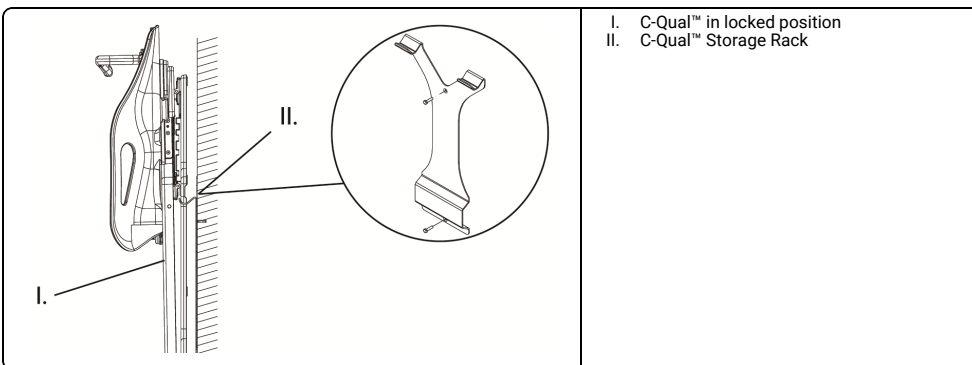
1. Clean device with a mild, non-abrasive cleaner as needed.

**MAINTENANCE**

NOTE: Inspect device prior to use for signs of damage and general wear.

**STORAGE**

- NOTE:
- Ensure Monarch™ Overhead Arm Positioner is securely attached to C-Qual™.
  - Wrap device in clean soft cloth to prevent scratching. Store in cool, dry place.
  - Before C-Qual™ storage holder installation, consult your facility's manager.
  - Device should be stored on C-Qual™ storage rack.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. C-QUAL, LOK-BAR AND MONARCH ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC