Solstice™ Thermoplastic Frame

For use with Solstat™ Immobilization System and Solstice™ SRS Immobilization System

INSTRUCTIONS FOR USE (IFU)

CQ MEDICAL™

CE

INDICATIONS FOR USE / INTENDED USE

- Devices are used to aid in the support and positioning of patients during an MRI.
- The intended use of this device is for fixation and repositioning of the patient during radiotherapy and diagnostics.

OPEN FACE THERMOPLASTIC ONLY:

The device is indicated to position and/or immobilize adult and pediatric patients undergoing radiation therapy of the head, brain, and neck, including Surface Guided Radiation Therapy (SGRT) and electron, photon, and proton treatments. The device is also used during image acquisition, including Computed Tomography (CT) Magnetic Resonance (MR) imaging, to support treatment planning.

SOLSTICE™ SHOULDER THERMOPLASTIC:

The device is indicated to aid in supporting, positioning, and/or immobilization of adult and pediatric patients undergoing radiation therapy of the head, brain, neck, and spine including radiosurgery and electron, photon, and proton treatments. The device is also used during image acquisition to support treatment planning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

MRI SAFETY INFORMATION

- Devices are MR Safe.

MR Safe

⚠ WARNING

- Do not use if device appears damaged.
- Device should be used for a single patient throughout setup and treatment cycle.
- Ensure thermoplastic is properly aligned over patient before forming.
- Ensure nose mouth portal is aligned during molding to allow patient to breathe from nose and mouth during future use.
- Ensure patient is comfortable during molding process to ensure fit is comfortable during future use.
- Device can be used on a patient with a tracheotomy tube without interfering with the function of the tracheotomy tube. Avoid contact with tracheotomy tube when forming thermoplastic.
- Cutting thermoplastic masks may reduce the rigidity of the device. If cutting is necessary, ensure rigidity of mask is maintained by adding thermoplastic strips. Thermoplastic strips are not included with masks but can be purchased by contacting your CQ Medical representative.
- Verify all angles of treatment, attenuation characteristics and WET values prior to treating patients.
- Ensure thermoplastic is properly attached using clamps, pins or quick fasteners.
- Ensure device is properly assembled and secure.
- Thermoplastic material may cause skin irritation in individuals with sensitive skin.
- Do not sterilize thermoplastic.


NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

USING THERMOPLASTICS**⚠ WARNING**

- Ensure latches are in unlocked position.

Thermoplastic Type	Waterbath Temperature	Heating Time
Solstice™ Head Only Thermoplastic	75°C (165°F)	5 min
Solstice™ Shoulder Thermoplastic	75°C (165°F)	7–9 min


OPTIONAL: To prevent possibility of hair adhering to thermoplastic during forming, use hair cover provided.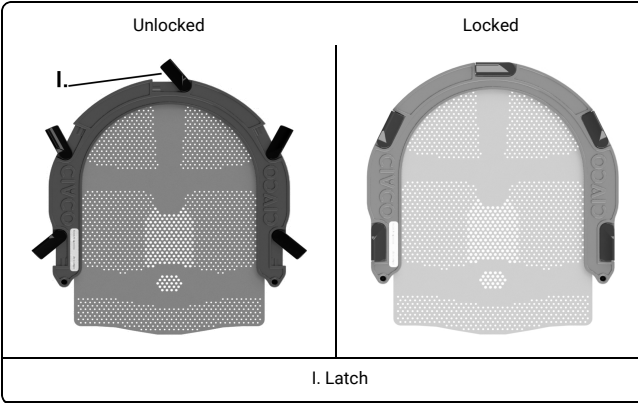

1. Setup waterbath with 2–3" of water and heat (*refer to table for appropriate temperature*).
2. Place thermoplastic into bath for directed time.

NOTE: Improper heating time may cause difficult application.

3. Remove thermoplastic from waterbath.
4. Quickly remove excess water from thermoplastic (*less than five seconds*).
5. Position thermoplastic over device.
6. Gently pull frame down to top edge of bowl, aligning mask to capture forehead and chin.
7. Lock all latches to device to secure thermoplastic.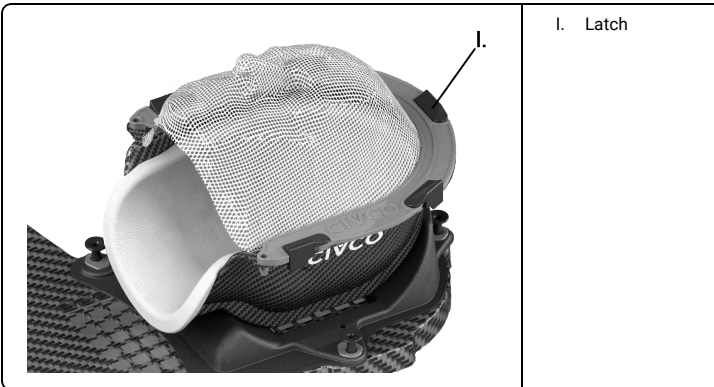


NOTE: *For Open Face Only* - Ensure desired anatomical locations are free of thermoplastic material as required by camera system region of interest.

8. Gently mold material to contours.
9. Allow thermoplastic to cool completely before releasing device.


USING SOLSTICE™ SHOULDER THERMOPLASTIC

- NOTE:**
- Use of an immobilization device which aids shoulder depression, such as patient handles, is recommended when using Solstice™ Shoulder Thermoplastic.
 - While pulling mask ensure thermoplastic is not folded incorrectly under itself.
 - Overstretching of thermoplastic material may affect rigidity of mask. After pulling thermoplastic mask, ensure rigidity of mask is maintained.

1. Follow steps 1-5 under *Using Thermoplastics*.
2. Gently pull sides of frame outward and down to attachment base.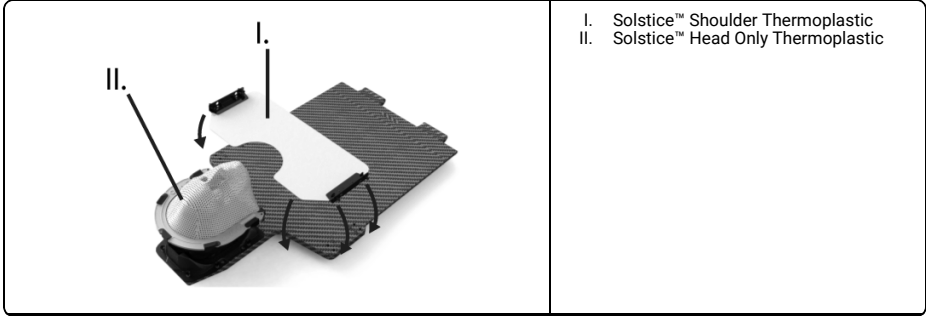

3. Use all available pins to latch thermoplastic to attachment base.

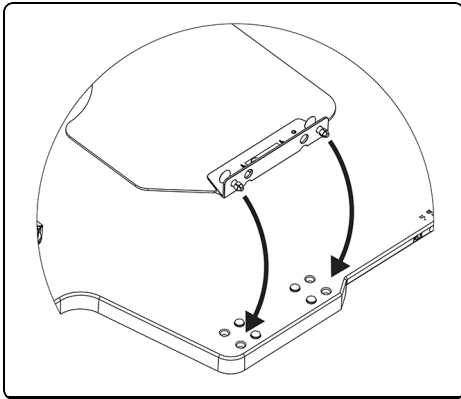4. Gently mold material to contours of treatment area.
5. Allow thermoplastic to cool completely before releasing from attachment base.

REPROCESSING**⚠ WARNING**

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

1. If necessary, clean surface of thermoplastic and/or frames and pins by removing gross visual contaminants with alcohol or warm soapy water. If gross visual contamination cannot be removed repeat cleaning steps and if necessary, discard device.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. SOLSTAT AND SOLSTICE ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
**MEDTEC LLC**