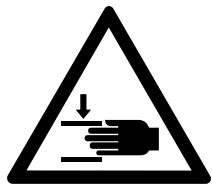
Universal Couchtop™ CT Overlay

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INTENDED USE

The device is indicated to aid in supporting and positioning adult and pediatric patients undergoing radiation therapy including electron, photon, and proton treatments. The device is also used during image acquisition to support treatment planning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

 WARNING

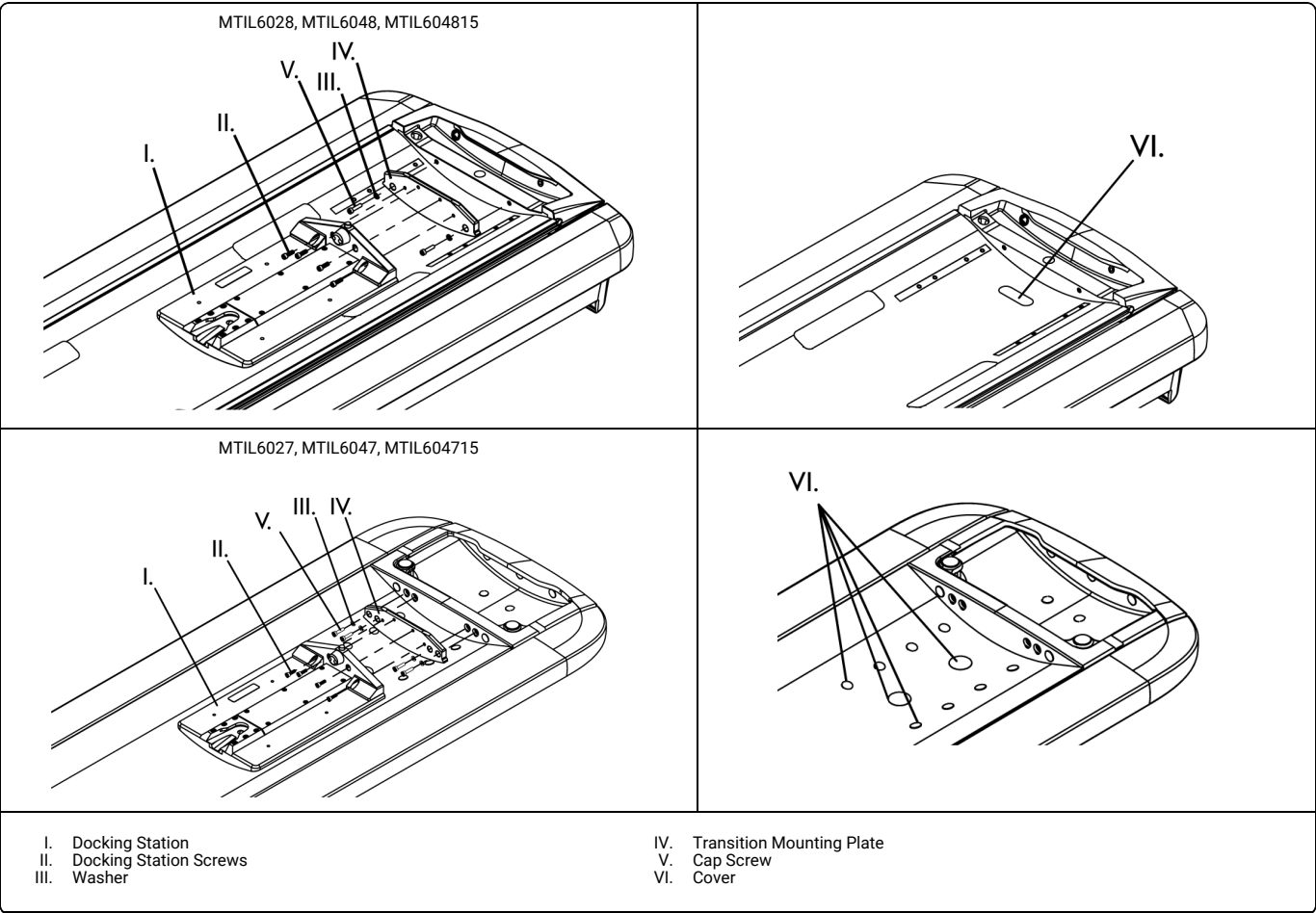
- Do not use if device appears damaged.
- MTIL6027, MTIL6028: Do not apply a load exceeding 444 lbs (200 kg).
- MTIL6047, MTIL6048: Do not apply a load exceeding 484 lbs (220 kg).
- Prior to use, ensure overlay is secure.

- NOTE:
- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
 - Maximum clear scanning area MTIL6027: 50.00" (1270mm)
 - Maximum clear scanning area MTIL6028: 48.70" (1237mm)
 - Maximum clear scanning area MTIL6047 47.05" (1195mm)
 - Maximum clear scanning area MTIL6048: 47.05" (1195mm)

ATTACHING DOCKING STATION TO CRADLE

NOTE: Tools Required: 3/16" hex wrench, 10mm hex wrench and level (not provided).

1. Remove all accessories from cradle.
2. If necessary, remove small cover to avoid interference during docking station attachment.
3. Fasten docking station using screws (provided) and transition mounting plate (provided).


ADJUSTING OVERLAY (IF APPLICABLE)

NOTE: If adjustment is necessary use the following options or combination of options.


DOCKING STATION ADJUSTMENT

NOTE: Tools Required: 3/32" and 5/32" hex wrench and level (not provided).

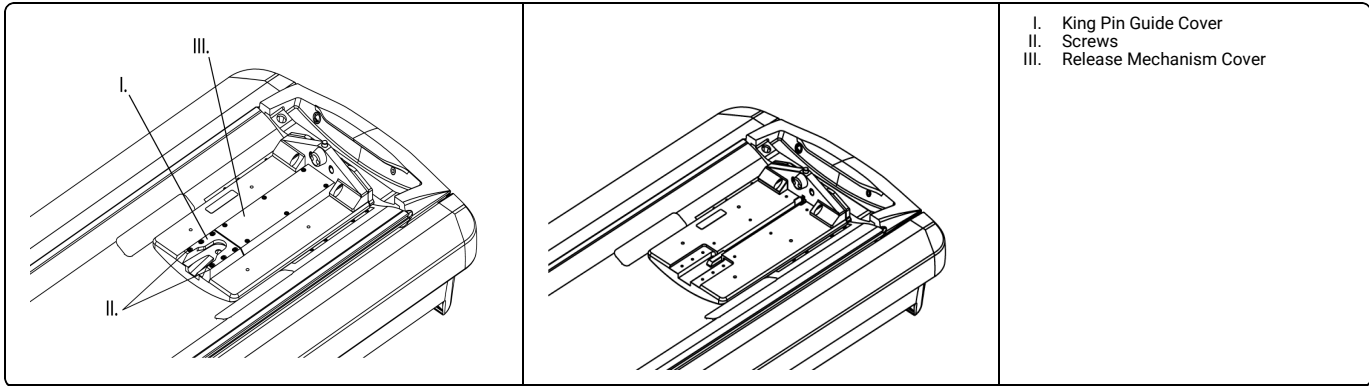1. Remove release mechanism cover and king pin guide cover.

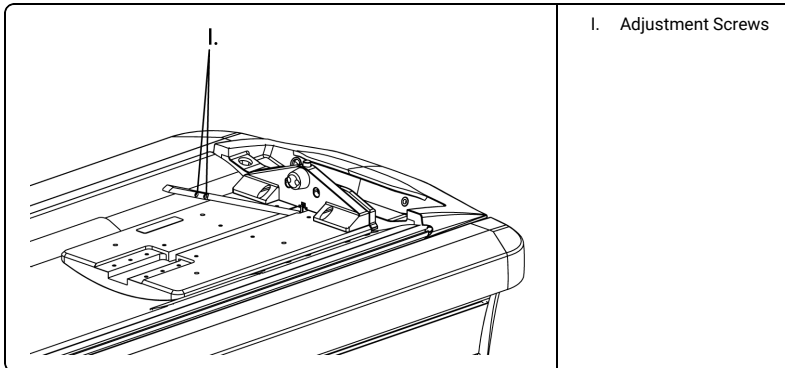
2. Loosen release mechanism adjustment screws, adjust release mechanism until fully extended and retighten screws.

3. Fasten king pin guide cover loosely using screws.

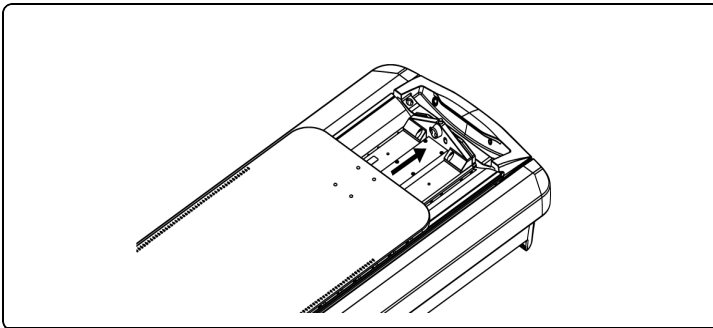4. Slide overlay into docking station, ensuring locating cone fits securely into end of overlay and king pin fits properly into locked position of king pin guide cover.


5. Note and mark position of king pin guide cover.

6. Remove overlay.

7. Loosen release mechanism adjustment screws and adjust release mechanism until wedge end of release mechanism aligns with rear end of locking hole in king pin guide cover.

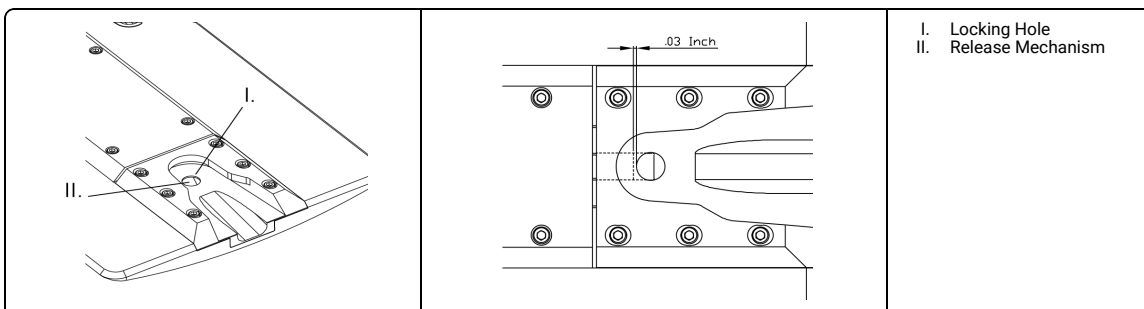NOTE: Verify all adjustment screws are tight, ensuring king pin guide cover is secured in marked position.

8. Fasten release mechanism cover onto docking station.

9. Slide overlay into docking station, ensuring locating cone fits securely into end of overlay and king pin fits and locks into king pin guide cover.

NOTE:

- Verify locking mechanism releases properly.
- Ensure docking station location allows for easy installation and removal of overlay.
- Ensure overlay is level to within 1° laterally and longitudinally.

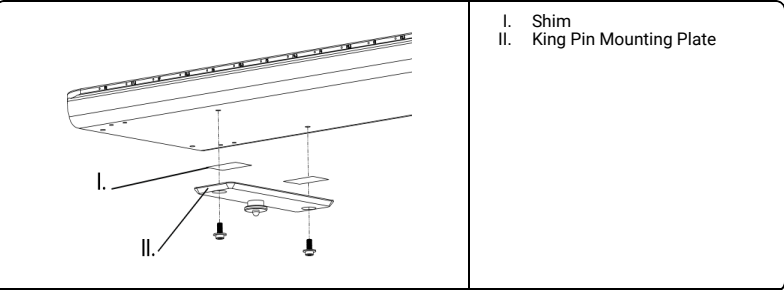
10. Once overlay is locked properly, tighten screws securing docking station to cradle.

LEVELING ADJUSTMENT (IF APPLICABLE)

NOTE: If adjustment is necessary use the following options or combination of options.


OPTION A

1. Remove king pin mounting plate and adjust by adding or removing shims.


OPTION B

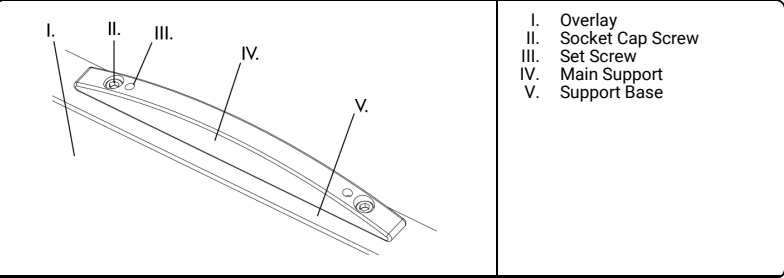1. Loosen socket cap screw to allow adjustment of support.
2. Adjust set screws as needed.
NOTE: Set screws can be adjusted individually to allow for lateral leveling and vertical adjustment of overlay.
3. Tighten socket cap screws to lock adjustment in place.

OPTION C

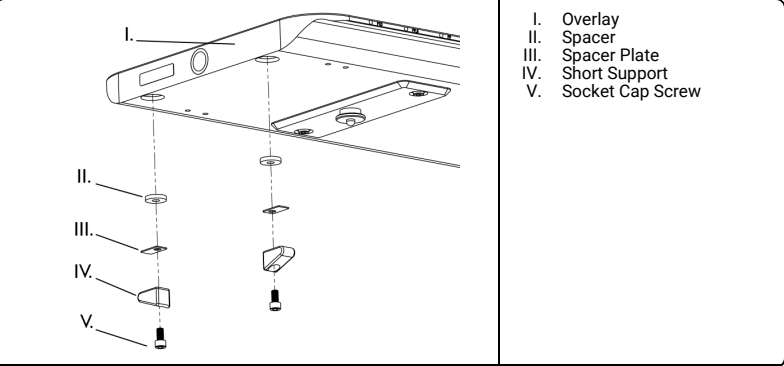
1. Remove socket cap screws.
2. Add/remove spacer plates or shims as needed.
NOTE: Spacers and shims can be adjusted individually to match adjustment of main support.
3. Insert and tighten socket cap screws, attaching short supports to overlay.


STABILIZING LATERAL MOVEMENT OF OVERLAY (IF APPLICABLE)

NOTE: If additional lateral stabilization is necessary use the following options to secure overlay.

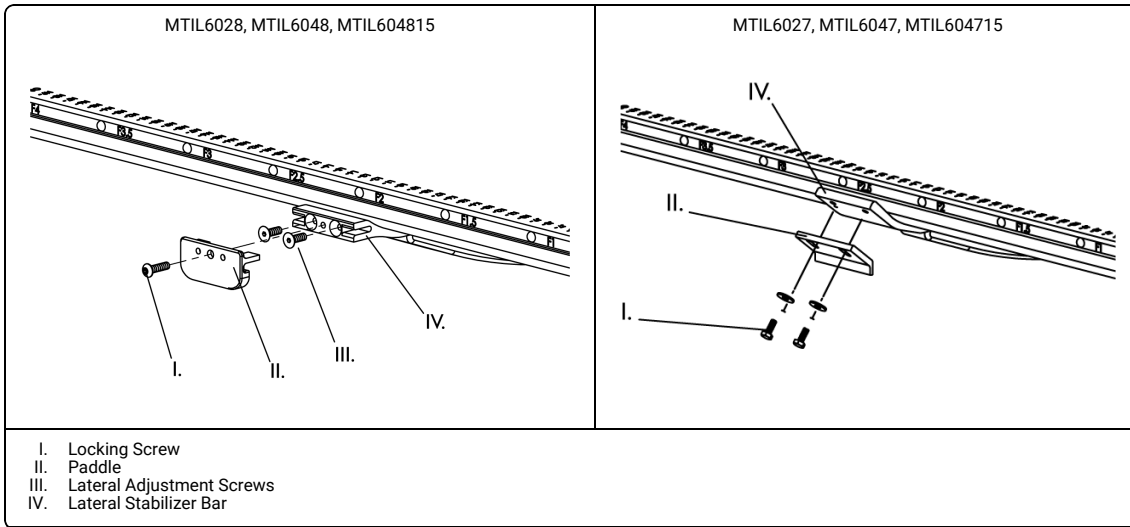1. Attach paddles to lateral stabilizer bar using locking screws.

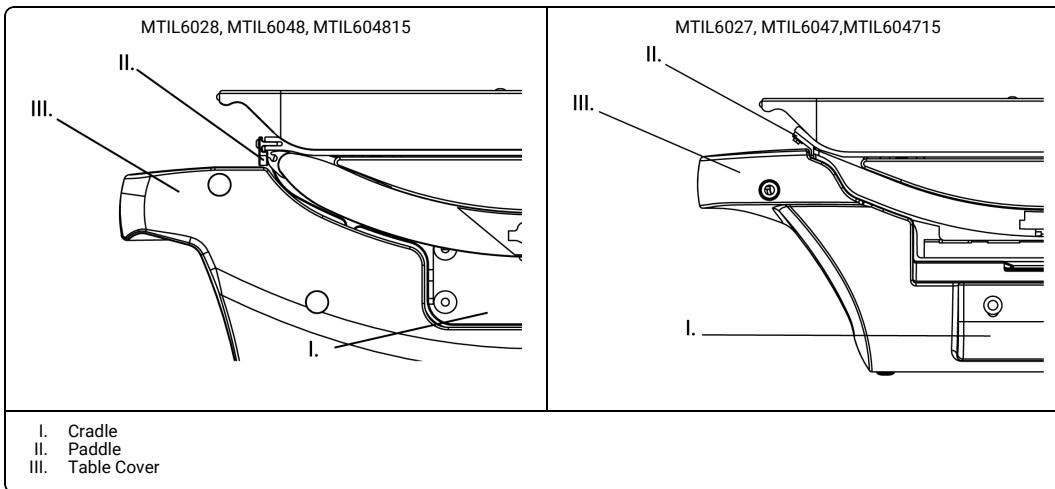2. Align couch overlay with cradle.
3. Slide paddles inward until contact is made with cradle.
4. Loosen lateral adjustment screws until contact is made with surface of paddle.
5. Tighten locking screws to secure.


NOTE: Ensure clearance between paddles and table covers when cradle is extended.

USING Lok-Bar™ **WARNING**

- Do not use Lok-Bar™ to secure positioning devices.

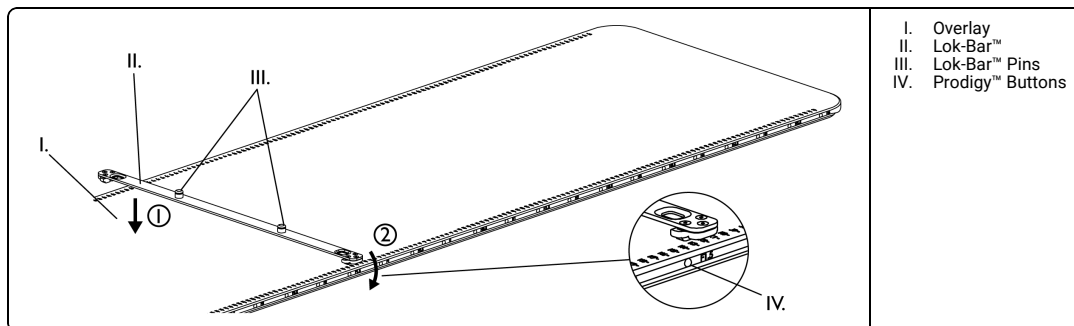1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

 **WARNING**

- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.

REPROCESSING **WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. LOK-BAR AND PRODIGY ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS.
MEDTEC LLC