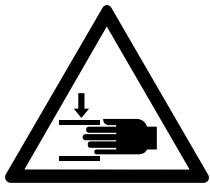
Universal Couchtop™ MRI Overlay

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INTENDED USE

Devices are used to aid in the support and positioning of patients during an MRI.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

- Do not use if device appears damaged.
- Ensure accessories are securely attached in correct position prior to imaging.
- Ensure device is secure prior to use.
- Store overlay in upright position when not in use to prevent damage to overlay supports.
- Use MR compatible accessories only.
- Grasp overlay at head end and foot end, using two people, to prevent pinched fingers.

- NOTE:
- If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
 - MTM3303 (GEHC 5917534-2), UCT MRI Overlay GEM: Overlay covers table IV pole. Use external IV pole if necessary.
 - UCT MRI Overlay GEM weight: 23.5 lbs (10.7 kg). For total table load rating, including accessories, refer to your system's manual.
 - MTM3304 (GEHC 5917537-2), UCT MRI Overlay Comfort Plus weight: 24.5 lbs (11.1 kg). For total table load rating, including accessories, refer to your system's manual.
 - MTM3305 (GEHC 5917536-2), UCT MRI Overlay eXpress Detachable weight: 24 lbs (10.9 kg). For total table load rating, including accessories, refer to your system's manual.

MRI SAFETY INFORMATION

MR Safe

- Device is MR safe.

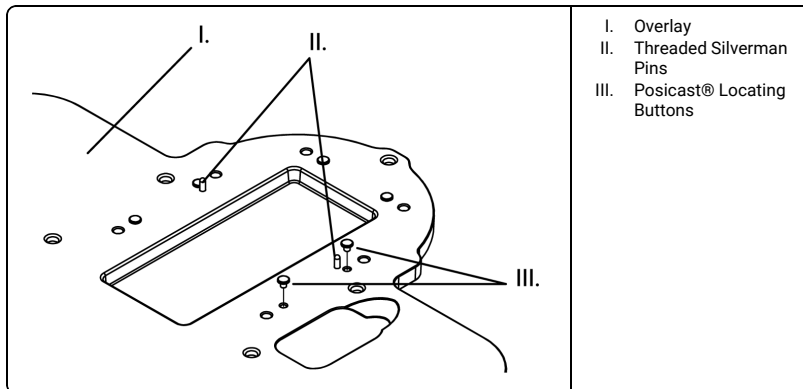
USING THERMOPLASTICS

PRIOR TO USE: Refer to instructions provided with each thermoplastic mask.

NOTE: Device is compatible with CQ MedicalPosicast® thermoplastics and CQ MedicalType-S™ thermoplastics (refer to Thermoplastic instruction).


UPDATE OVERLAY THERMOPLASTIC CAPABILITY

1. To use Type-S™ thermoplastic, ensure threaded Silverman pins are installed in overlay and Posicast® locating buttons are removed from overlay.
2. To use Posicast® thermoplastic, ensure Posicast® locating buttons are installed in overlay and threaded Silverman pins are removed from overlay.

THREADED SILVERMAN PINS

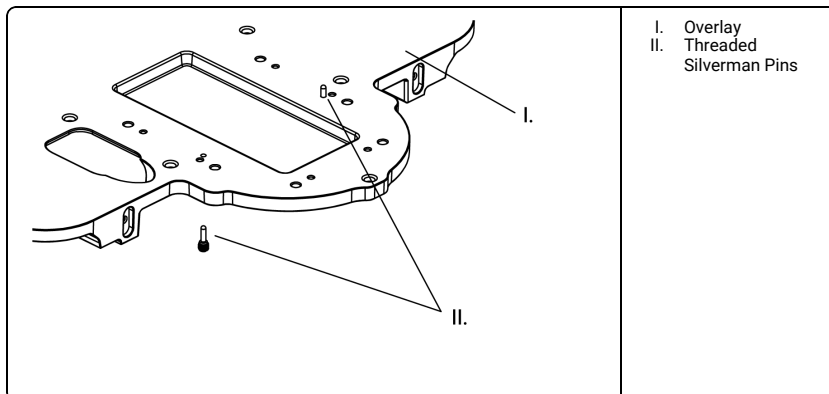1. To install or uninstall threaded Silverman pins, remove overlay from tabletop.
2. Locate threaded Silverman pins on underside of overlay and install or remove using flat head screwdriver.


3. Reattach overlay to tabletop.

ATTACHING OVERLAY TO PATIENT TABLE



PRIOR TO USE:

- Before using this product, refer to operating instructions provided with 32ch AIR Open Coil Suite.
- Before using this product, refer to operating instructions provided with GE MRI scanner.

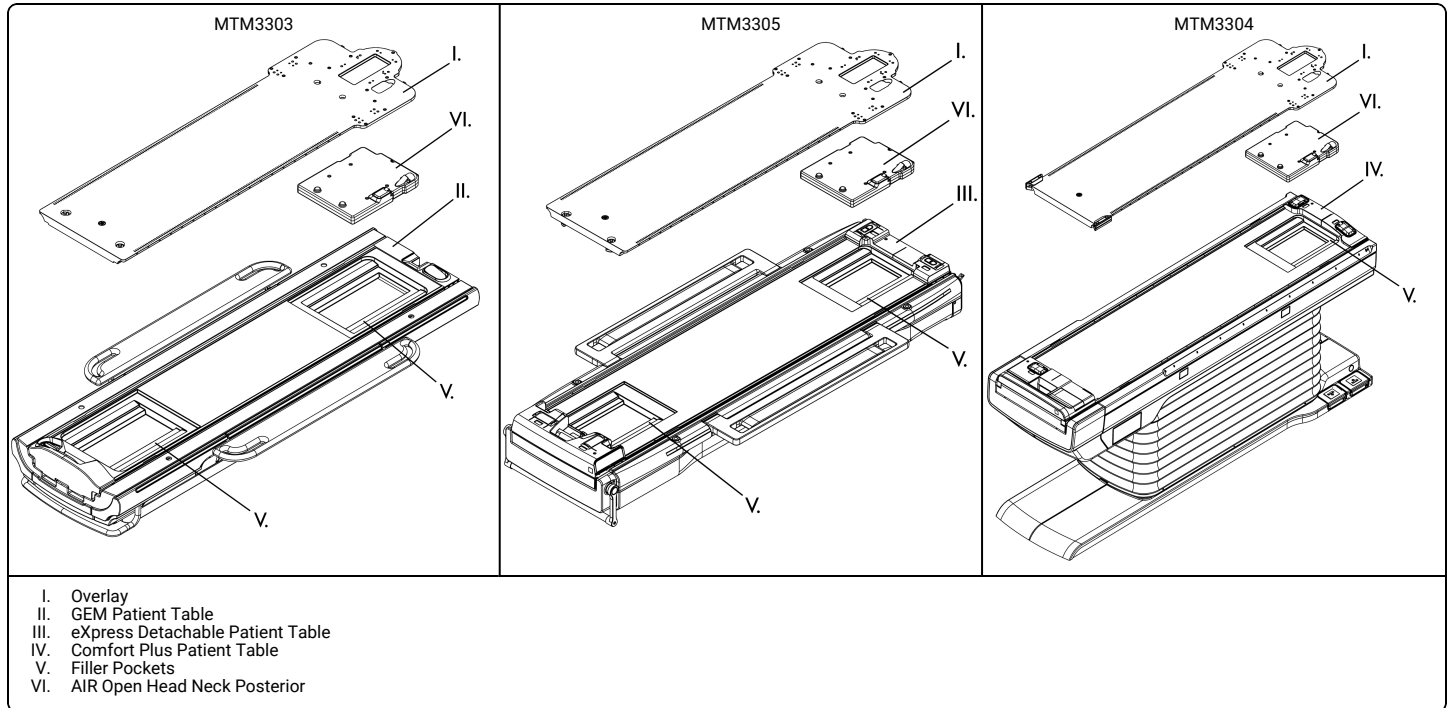1. Remove table pads and fillers from table.

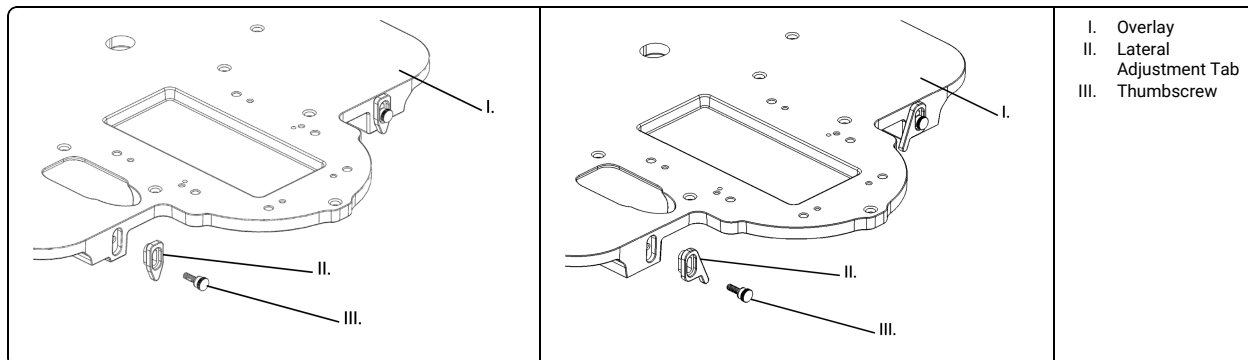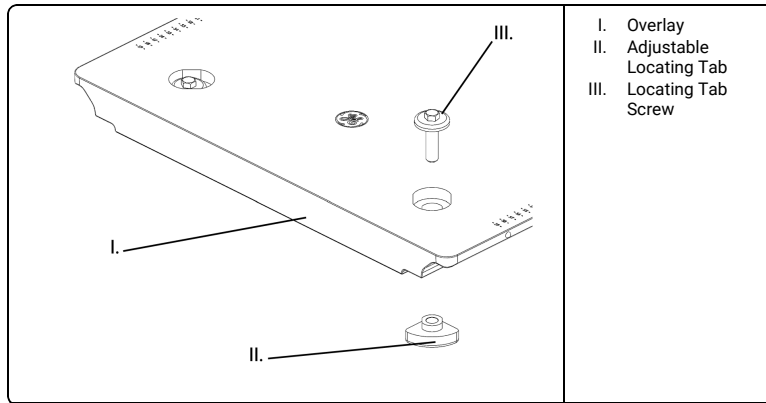
2. Ensure AIR Open Head Neck Posterior is properly installed in superior filler pocket (*refer to manufacturer's instruction*).
3. Ensure overlay properly locates on AIR Open Head Neck Posterior index posts (*refer to manufacturer's instruction*).
4. Ensure adjustable locating tabs and lateral adjustment tabs are inside filler pockets.

5. For GEM and eXpress Patient Tables: Adjust all adjustable locating tabs to be against filler pocket corners and tighten in place so overlay does not move.
6. Adjust all lateral adjustment tabs to be against filler pocket sides and tighten in place so overlay does not move.

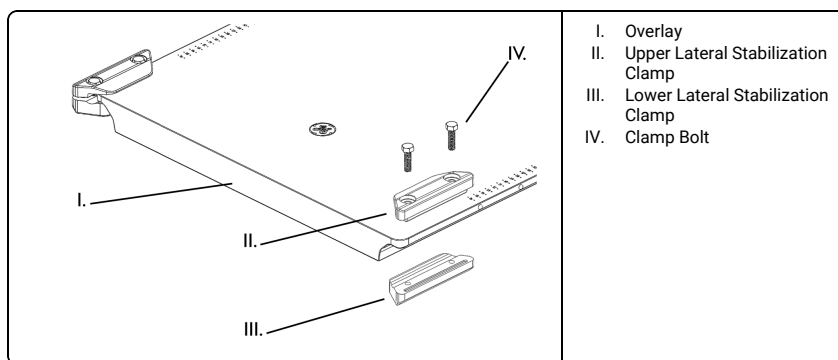7. For Comfort PlusPatient Table: Adjust all lateral stabilization clamps against inferior table housing and tighten in place so overlay does not move.

USING OVERLAY

1. Remove table pads and fillers from table.
2. Install AIR Open Head Neck Posterior.
3. Place overlay on cradle, ensure overlay is securely positioned.
4. Position patient as desired.
5. Position coils in desired location (*refer to manufacturer's instruction*).
6. Plug in coils (*refer to manufacturer's instruction*).

⚠ WARNING

- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides. Ensure operator keeps hands off table when overlay is in motion.

REPROCESSING

⚠ WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. If necessary, clean surface by removing visual contamination with common germicidal wipe, such as alcohol. If visual contamination cannot be removed repeat cleaning steps and if necessary, discard device.

MAINTENANCE

- NOTE:
- Inspect device prior to use for signs of damage and general wear.
 - Contact CQ Medical for replacement parts and accessories at [Info@CQ Medical.com](mailto:Info@CQMedical.com)

DESCRIPTION	GEHC PART NUMBER	GEHC REV NUMBER	CQ MEDICAL PART NUMBER (ORDERABLE)
Universal Couchtop™ MRI Overlay - GE GEM	M7021RT, 5917534-2	*	MTM3303
Universal Couchtop™ MRI Overlay - GE Comfort Plus	M7023RT, 5917537-2	*	MTM3304
Universal Couchtop™ MRI Overlay - GE eXpress Detachable	M7022RT, 5917536-2	*	MTM3305

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. TYPE-S AND UNIVERSAL COUCHTOP ARE TRADEMARKS OF MEDTEC LLC. POSICAST IS A REGISTERED TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
GE Medical Systems Turkey Ltd. Sti.
MEDTEC LLC