Elekta Unity Lok-Bar™

INSTRUCTIONS FOR USE (IFU)

INTENDED USE

The device is intended to support and aid in positioning a patient during radiologic and other medical procedures.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

WARNING

- Do not use if device appears damaged.
- Do not use Lok-Bar™ to secure positioning devices.
- Ensure Lok-Bar™ is only used on compatible equipment.
- Ensure Lok-Bar™ is securely attached to mating equipment.

NOTE: If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

MRI SAFETY INFORMATION


- Device is MR safe.

MR Safe

USING Lok-Bar™


1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.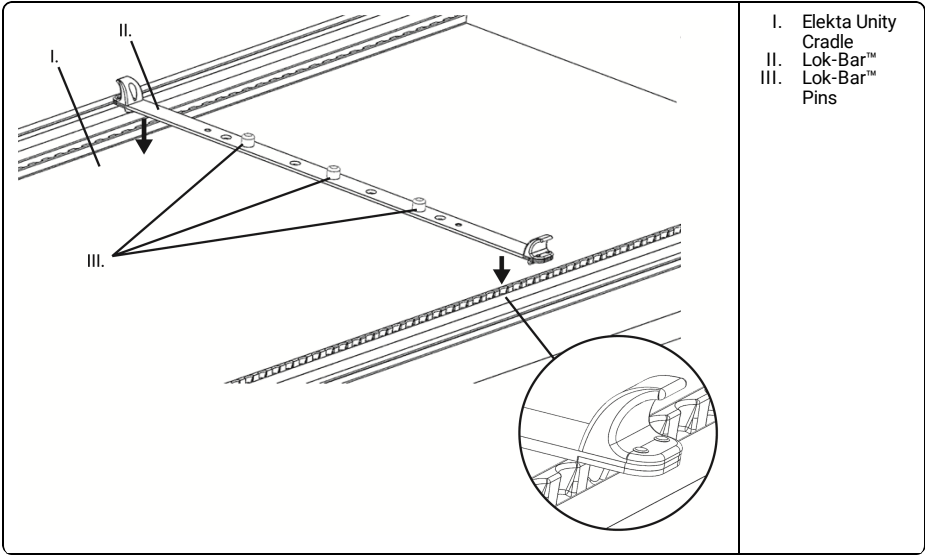

USING QA Lok-Bar™

NOTE: Elekta Conical Locators not included with QA Lok-Bar™.

1. Attach Elekta Conical Locators to QA Lok-Bar™.
2. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
3. Place Elekta QA Phantom onto Lok-Bar™.

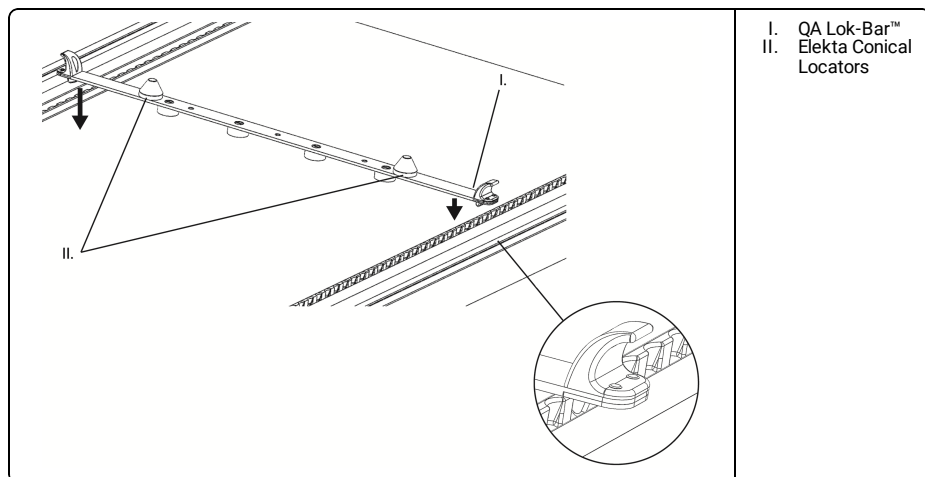REPROCESSING**WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. If necessary, clean surface by removing visual contamination with common germicidal wipe, such as alcohol. If visual contamination cannot be removed repeat cleaning steps and if necessary, discard device.
2. Wipe dry with lint-free cloth or allow to air dry prior to reassembly and use.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. LOK-BAR IS A TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC