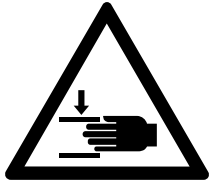
Universal Couchtop™ CT Overlay

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INTENDED USE

MTIL6029, MTIL6049: The device is indicated to aid in supporting and positioning adult and pediatric patients undergoing radiation therapy including electron, photon, and proton treatments. The device is also used during image acquisition to support treatment planning.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

**WARNING**

- Do not use if device appears damaged.
- Ensure device is secure prior to use.
- Avoid pinch points.
- MTIL6029: Do not apply a load exceeding 473 lbs (215 kg).
- MTIL6049: Do not apply a load exceeding 460 lbs (209 kg).
- Ensure accessories are securely attached in correct position prior to treatment and/or imaging.

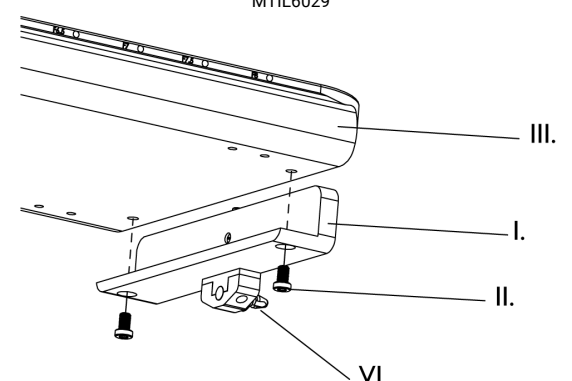
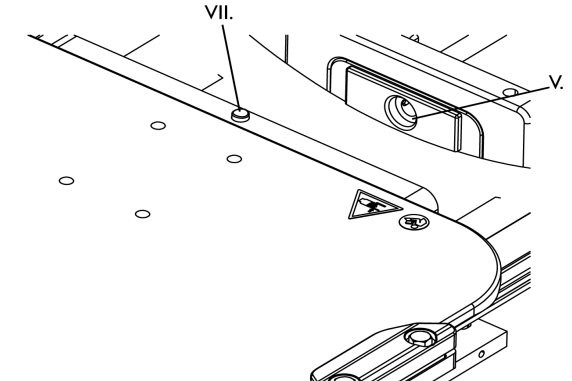
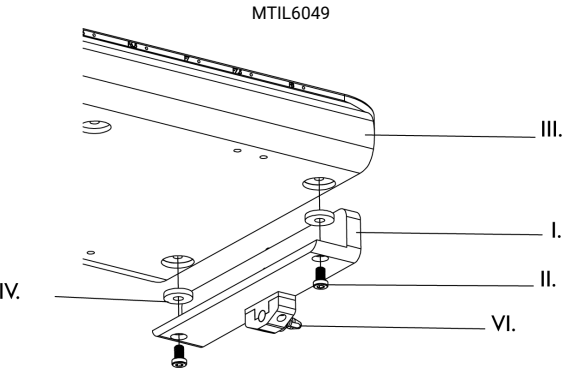
NOTE: • If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

 • Prior to use, ensure overlay is locked in position by gently lifting superior end.

ATTACHING INTERFACE PLATES TO CRADLE

NOTE: 7mm hex key wrench (provided).

1. Remove all accessories from cradle.
2. Fasten interface plate onto bottom of overlay using cap screws (provided).
3. Place overlay on cradle, ensuring top surface is level.
4. Suppress and hold push button and slide docking pin into mounting hole.
5. Release push button to lock into place.

MTIL6029 	
MTIL6049 	
I. Interface Plate II. Cap Screw III. Overlay IV. Spacer V. Mounting Hole VI. Docking Pin VII. Push Button	

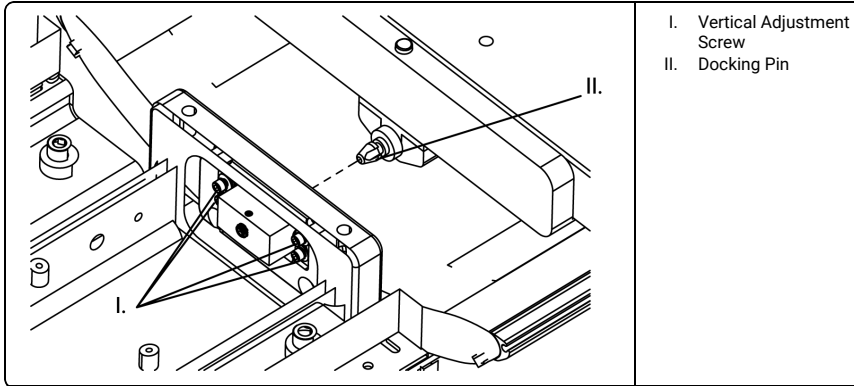
ADJUSTING OVERLAY (IF APPLICABLE)

NOTE: If adjustment is necessary use the following options or combination of options.


CRADLE MOUNTING HOLE ADJUSTMENT

NOTE: Tools Required (*not provided*): 5/32" hex key wrench, 3/16" hex key and level..

1. Remove cover from Siemens Biograph mCT system. Contact Siemens Installation Engineer for assistance removing cover, if needed.
2. Loosen vertical adjustment screws and slide mounting hole up or down to align with docking pin.

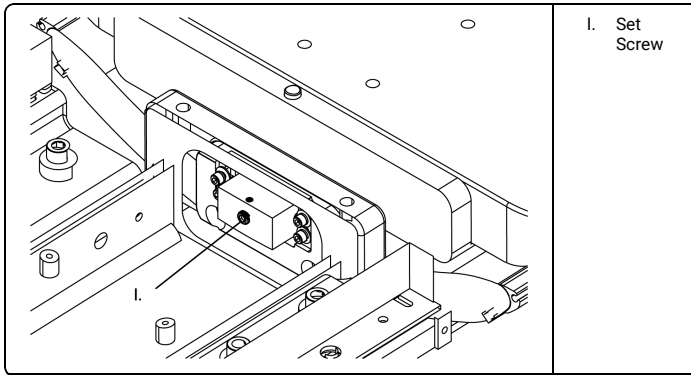3. Reinsert overlay to mounting hole.
4. Adjust set screw to contact end of docking pin.

5. Once overlay is in correct position, tighten vertical adjustment screws and replace cover in Siemens Biograph mCT system. Contact Siemens Installation Engineer for assistance replacing cover, if needed.

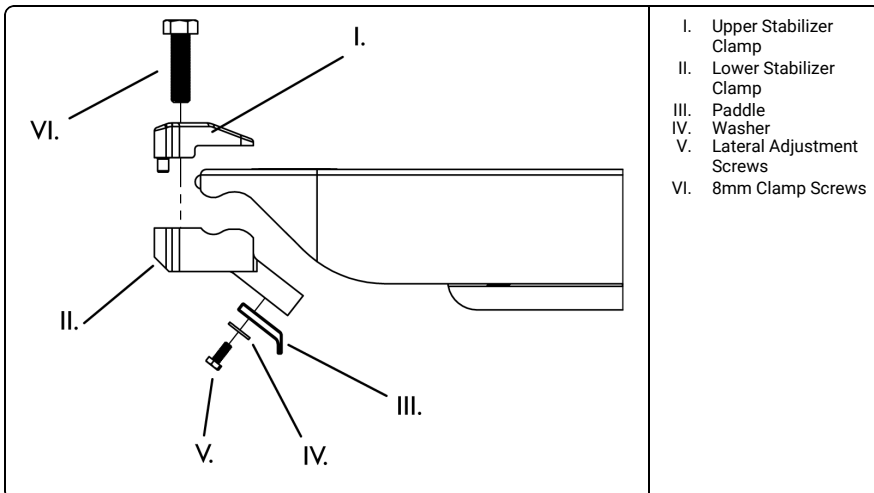
NOTE:

- Verify locking mechanism releases properly.
- Ensure mounting hole location allows for easy installation and removal of overlay.

STABILIZING LATERAL MOVEMENT OF OVERLAY (IF APPLICABLE)


NOTE: Tools required: 8mm socket tool (*provided*).

1. Secure upper and lower lateral stabilizer clamps onto edge profile of overlay using 8mm clamp screws.
2. Attach paddles to lower lateral stabilizer clamp using lateral adjustment screws.

3. Align overlay with cradle.
4. Slide paddles inward until contact is made with cradle.

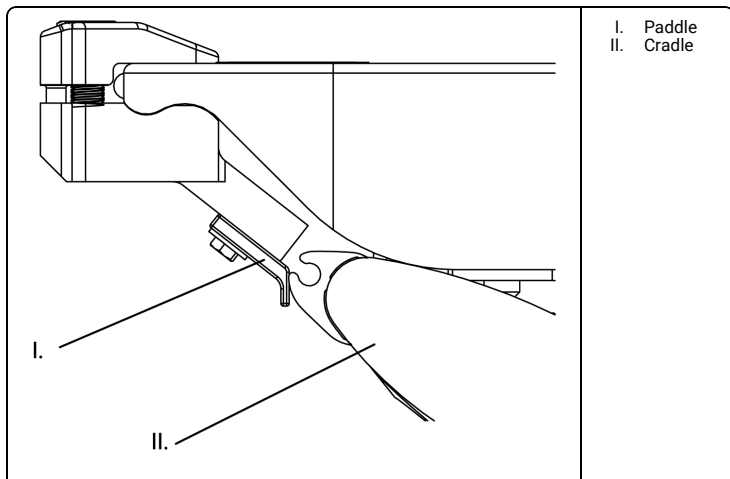5. Loosen lateral adjustment screws until contact is made with surface of paddle.
6. Tighten clamp screws to secure.


NOTE: Ensure forward most lateral stabilizer clamps do not extend past M2 on Indexing.

USING Lok-Bar™

⚠ WARNING

- Do not use Lok-Bar™ to secure positioning devices.

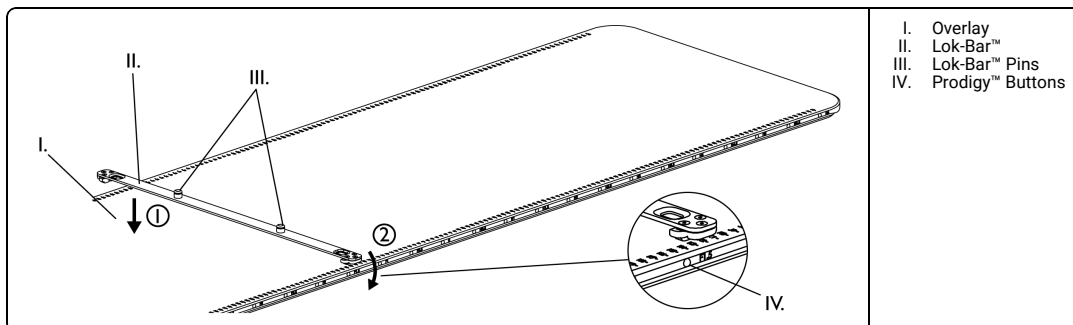1. Snap Lok-Bar™ at appropriate longitudinal position. Ensure Lok-Bar™ is secure.
2. Place positioning accessory onto Lok-Bar™ pins.

⚠ WARNING

- Avoid pinch points when overlay is in motion by ensuring patient does not grasp overlay sides.

REPROCESSING

⚠ WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

STORAGE

NOTE: Ensure device is not stored with weight resting on docking pin.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. UNIVERSAL COUCHTOP AND LOK-BAR ARE TRADEMARKS OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC