GE MR Posifix® Baseplate

REFERENCE GUIDE

INTENDED USE

Devices are used to aid in the support and positioning of patients during an MRI.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

⚠ WARNING

- Do not use if device appears damaged.
- Verify all angles of treatment and attenuation characteristics prior to treating patients.
- Do not reposition device with patient on it.
- Do not allow patient to reposition themselves.
- Ensure device is secure prior to use.

NOTE: • If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.

- Baseplates require the use of a headrest (*refer to Headrest instruction*).


MRI SAFETY INFORMATION

MR Safe

- Acrylic Posifix® Baseplate is safe for use within MR environment.

USING ACRYLIC POSIFIX® BASEPLATE

1. Place Posifix® baseplate onto overlay.
2. Attach quickfasteners to overlay to secure baseplate.

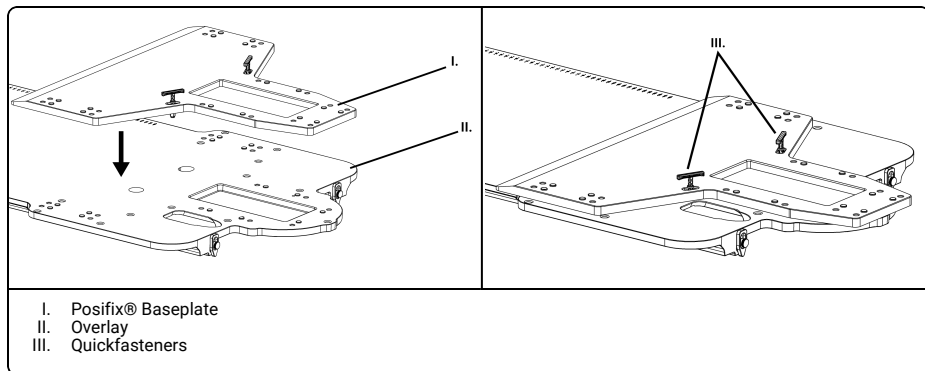3. Place desired headrest onto baseplate.
4. Position patient onto baseplate for thermoplastic setup (*refer to thermoplastic instruction*).
5. Attach thermoplastic quickfasteners to baseplate to secure thermoplastic.

REPROCESSING**⚠ WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.

1. If necessary, clean surface by removing visual contamination with 70% alcohol or Theracide (PDI Sani-Cloth Plus) wipes. If visual contamination cannot be removed, repeat cleaning steps and if necessary, discard device.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

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COPYRIGHT © 2024 ALL RIGHTS RESERVED. CQ MEDICAL IS A TRADEMARK OF MEDTEC LLC. POSIFIX IS A REGISTERED TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA.
MEDTEC LLC