Curvaform™ Head Support

INSTRUCTIONS FOR USE (IFU)

CE

INTENDED USE

The device provides noninvasive stereotactic head and neck immobilization by using a patient specific thermoplastic mask that conforms to the patient's features to provide accurate, reproducible positioning, repositioning and immobilization. The device allows the patient to undergo diagnostic imaging in the same position as that of treatment, enabling radiation therapy.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.


⚠ WARNING

- No modification of this equipment is allowed. If any part of this device experiences a catastrophic load, appears damaged or functions improperly, discontinue use immediately and contact CQ Medical at info@CQmedical.com.
- Ensure device is attached securely prior to treatment.
- Ensure accessories are securely attached in correct position prior to imaging.
- When positioning patient for first time, use setup sheet to record all adjustments. Setup sheet is available at www.CQmedical.com.
- Verify patient position with verification image and completed setup sheet prior to treatment.
- Ensure patient remains still for duration of setup and treatment.
- Ensure clearance is maintained between devices and all linac or imaging equipment before use.
- Do not stack multiple headrests on top of headrest riser.

- NOTE:**
- Not intended for SRS treatment or MR use.
 - If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within the European Union, also report to the competent authority of the Member State in which you are established.
 - Treatment through any device will lead to an increased skin dose. Thermoplastic masks will attenuate a radiotherapy beam and increase skin dose. Attenuation and increased skin dose should be taken into account during planning and treatment.

USING CURVAFORM™ HEAD SUPPORT


1. Secure compatible device to couchtop.
2. Place Curvaform™ Head Support onto compatible device, aligning Curvaform™ pins with the holes in the compatible device.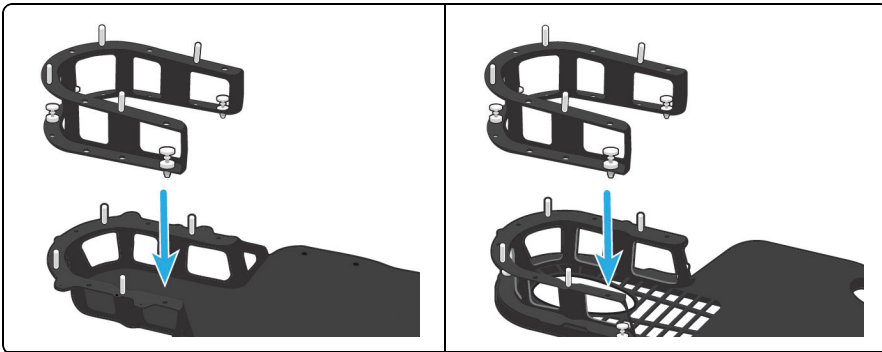

3. Press pin downward until it snaps into place. Repeat for remaining pins.

INSTALLING HEADREST RISER


1. Place headrest riser into Curvaform™ Head Support.
2. Place desired headrest on headrest riser. Adjust by sliding superior or inferior until desired position is reached.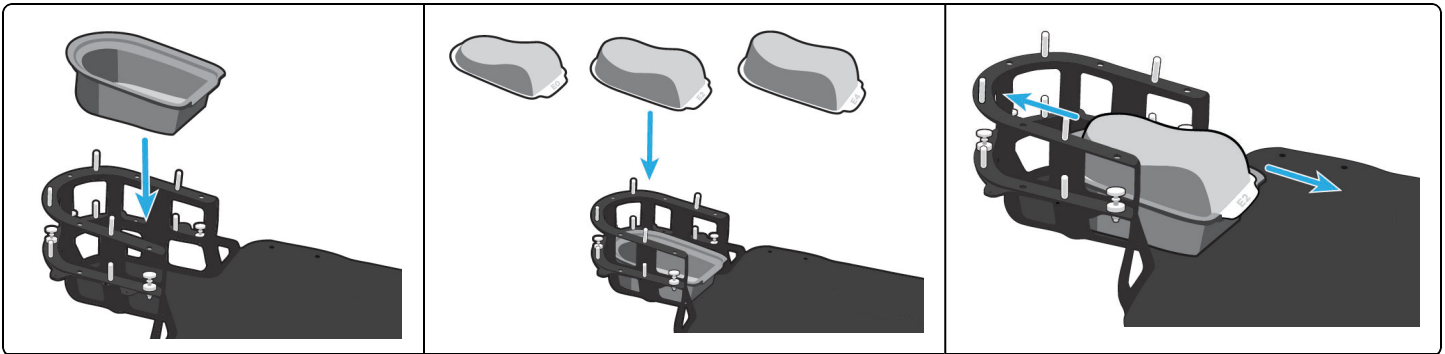

NOTE: Refer to your specific headrest IFU for more information.

INSTALLING MOLDABLE CUSHION SUPPORT

1. Place moldable cushion support on Curvaform™ Head Support.
2. Install moldable cushion.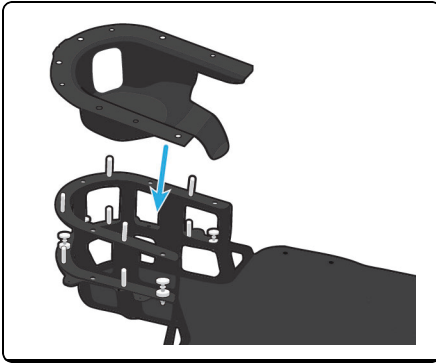

NOTE: Refer to your specific cushion IFU for more information.

REMOVING CURVAFORM™ HEAD SUPPORT

1. Remove all accessories from Curvaform™ Head Support.
2. Pull up on pins until pins release.
3. Lift Curvaform™ Head Support from compatible device.

REPROCESSING


⚠ WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.
- User should clean device between uses according to instruction.

1. If necessary, clean surface by removing visual contamination with 70% alcohol or Theracide (Super-Sani-Wipes) wipes. If visual contamination cannot be removed, repeat cleaning steps and if necessary, discard device.

MAINTENANCE

NOTE: Inspect device prior to use for signs of damage and general wear.

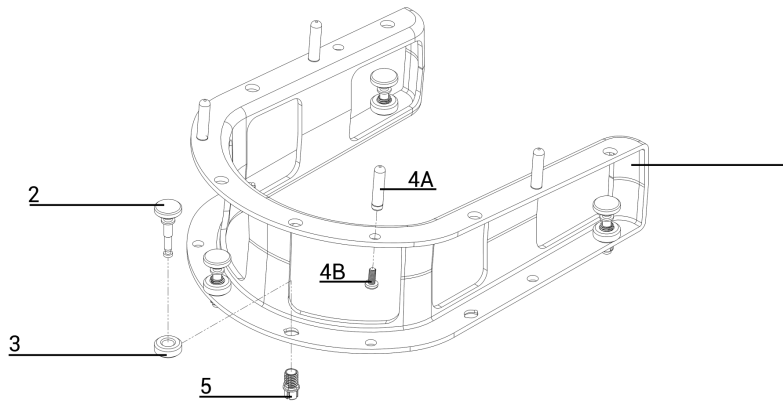PARTS LIST	
RT-4600-05	1. Curvaform™ Adapter
2008106	2.Pin
2008107	3. Screw
8002499	4. Locator Pins
2005148	4A. Locator Pins, SRS
3001479	4B. Screw (M5 x 0.8 x 12 mm)
2009627	5. Ferrule
REPLACEMENT PARTS	
8003471	Headrest Riser
8003473	Attachment Kit (contains 2008106, 2008107, 2009627)

COMPATIBLE DEVICES		
Part Number	Name	IFU
RT-4600-01	Encompass™ SRS Standalone Device	2005445
RT-4600KV-01	kVue™ Encompass™ SRS Insert	2005445
RT-4600-02	Encompass™ SRS Standalone Device compatible with bridges	2005445
RT-4552	Portrait™ Intracranial, Head & Neck Device *	2004503
RT-4552KV	kVue™ Portrait™ Intracranial, Head & Neck Insert *	2004689
RT-4552KV-01	Portrait™ Proton Insert *	2004689
TRT100	TotalRT Platform*	143-305
RT-4800-100	Alta™ Multipurpose Device *	2008272
MTIL6665, MTIL6670, MTIL6664	Type-S™ Extension *	143-251
20CFHNSUB7, 20CFHNSUB2	Type-S™ Overlay *	143-013
RT-4600-10, RT-4600-12, RT-4600-14	SRS Headrest	2005445
RT-4600-20	SRS Cushion Support	2008208

* Requires SRS Adapter (RT-4600-03) for use with Curvaform™

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COPYRIGHT © 2025 ALL RIGHTS RESERVED. CQ MEDICAL IS A REGISTERED TRADEMARK OF MEDTEC LLC. CURVAFORM IS A TRADEMARK OF MEDTEC LLC. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS.
Advena Limited