

Opti-Tatt™

Contents:
24 Opti-Tatt Tattoos

To maximize shelf-life,
keep Opti-Tatt in the
freezer between use.
Recommended storage
temperature is 0 °F (-18 °C).



 Qfix
440 Church Road
Avondale, Pennsylvania
United States
info@CQmedical.com

CQmedical.com | 143-346 | Rev. 1



Instructions for use located on the inside of this insert.

Indications for Use

Opti-Tatt is indicated for use by a healthcare professional to place reference marks on adult and adolescent patients undergoing external beam radiotherapy.

Intended Use

Opti-Tatt is a single-use, patient-marking device intended to be used by healthcare professionals when creating semi-permanent reference marks on patients undergoing external beam radiotherapy.

WARNINGS/CAUTIONS

Although Opti-Tatt is made of natural ingredients, an allergic reaction is possible.

- Adverse reactions can occur if the patient has fruit allergies.
- Do not shave the area before applying the tattoo or apply the tattoo to freshly shaved areas.
- Do not take a hot shower or apply hot water to the area for a few hours after application.

If a rash occurs, discontinue use immediately. If skin irritation occurs, an anti-itch cream can help minimize discomfort.

Opti-Tatt Patient Guidance

- Avoid excessive movements or sweating while the patch is on.
- Remove the patch when the two-hour time limit has been reached.
- Some ink may remain on the patch. This is normal and there is no need to reapply the Opti-Tatt patch.
- The tattoo may look light after removing the patch. The ink will darken in 24 - 48 hours.
- You may shower and resume normal activities once the patch is removed. Avoid direct scrubbing or moisturizing of the tattoo.

Instructions for Use

1. Prep the patient's skin by cleaning the area where the Opti-Tatt tattoo will be applied. Allow to dry before proceeding.
2. With scissors, carefully cut the desired tattoo from the sheet of tattoos.
3. Separate the two pieces of the Opti-Tatt patch and press the Opti-Tatt patch firmly on the patient's skin. No other action is required.
4. Notify the patient to leave the Opti-Tatt patch on the skin for two hours. After two hours have passed, the patient may remove and discard the patch.

To achieve optimal results and extend shelf-life, it is recommended that Opti-Tatt be stored in a freezer between uses. Tattoos stored at freezing temperatures (at or below 0 °F) have a shelf-life of one year.

Tattoos stored in a refrigerator (at or below 40 °F) have a shelf-life of six months, and tattoos stored at room temperature (41 °F and above) have a shelf-life of 30 days.