



Fiducial Marker

Device is intended to be placed into soft tissue or bone to establish a permanent, accurate, internal reference system.

Patients with known sensitivity to the materials listed below may suffer an allergic reaction to this device. If there is a known sensitivity to a device material, it is the responsibility of the physician to evaluate the risk/benefit prior to the use of the device.

If any serious incident occurs in relation to device, incident should be reported to manufacturer. If incident occurred within Australia, also report to the Therapeutic Goods Administration (www.tga.gov.au).

-
- MTNW887805** - Bone Markers, Gold (3) [99.99% Gold]
MTNW887808 - Soft Tissue Gold Marker, 0.8 x 3mm (3) [99.99%Gold]
MTNW887809 - Soft Tissue Gold Marker, 0.9 x 3mm (3) [99.99%Gold]
MTNW887812 - Soft Tissue Gold Marker, 1.2 x 3mm (3) [99.99%Gold]
MTNW887825 - Placement Needles (17GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 3mm (3) [99.99%Gold, Bone Wax]
MTNW887830 - Placement Needle (17GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 3mm (1) [99.99%Gold, Bone Wax]
MTNW887835 - Placement Needles (17GA ETW x 30cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 3mm (3) [99.99%Gold, Bone Wax]
MTNW887836 - Placement Needle (17GA ETW x 30cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 3mm (1) [99.99%Gold, Bone Wax]
MTNW887837 - Placement Needles (17GA ETW x 30cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 3mm (4) [99.99%Gold, Bone Wax]
MTNW887840 - Placement Needles (17GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 3mm (4) [99.99%Gold, Bone Wax]
MTNW887851 - Placement Needle (18GA ETW x 20cm) with bone wax and CyberMark™ Soft Tissue Gold Marker, 1 x 5mm (1) [99.99%Gold, Bone Wax]
MTNW887853 - Placement Needles (18GA ETW x 20cm) with bone wax and CyberMark™ Soft Tissue Gold Marker, 1 x 5mm (3) [99.99%Gold, Bone Wax]
MTNW887855 - CyberMark™ Soft Tissue Gold Marker, 1 x 5mm (4) [99.99%Gold]
MTNW887860 - Placement Needles (14GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 1.6 x 3mm (3) [99.99%Gold, Bone Wax]
MTNW887870 - Placement Needles (18GA ETW x 12cm) with bone wax and Soft Tissue Gold Marker, 0.9 x 3mm (3) [99.99%Gold, Bone Wax]
MTNW887871 - Placement Needle (18GA ETW x 12cm) with bone wax and Soft Tissue Gold Marker, 0.9 x 3mm (1) [99.99%Gold, Bone Wax]
MTNW887872 - Placement Needle (18GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 0.9 x 3mm (1) [99.99%Gold, Bone Wax]
MTNW887873 - Placement Needles (18GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 0.9 x 3mm (3) [99.99%Gold, Bone Wax]
MTNW887881 - Placement Needles (17GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 10mm (2) [99.99%Gold, Bone Wax]
MTNW887882 - Placement Needles (17GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 1.2 x 10mm (3) [99.99%Gold, Bone Wax]
MTNW887883 - Placement Needles (18GA ETW x 20cm) with bone wax and Soft Tissue Gold Marker, 0.8 x 10mm (2) [99.99%Gold, Bone Wax]

CUT HERE

150-001_1

Patient Name: _____

Patient ID: _____

Date of Implantation: _____

Health Institution

Name: _____

Address: _____

PATIENT IMPLANT CARD



LABEL HERE