



# PRODUCT GUIDE & USER MANUAL

RT-4543WALL  
Wall Mount





EC	REP	Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	 Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 712-737-8688 <a href="http://www.CQmedical.com">www.CQmedical.com</a>
----	-----	---	--

Access, AccuFix, Portrait and QUEST are trademarks of Qfix.

# TABLE OF CONTENTS

GENERAL PRECAUTIONS.....	4
WARNING STATEMENTS .....	4
PARTS INCLUDED.....	4
TOOLS REQUIRED.....	4
WARNING LABELS & DESCRIPTION.....	4
OPERATING INSTRUCTIONS.....	5
ACCESS SUPINE AND SUPINE MR .....	5
QUEST BOTTOM STOPPER .....	6
QUEST BREASTBOARD .....	7
ACCUFIX CANTILEVER BOARD.....	8
PORTRAIT HEAD AND NECK DEVICE .....	9
DEVICES WITH INDEXING NOTICES.....	10
WALL MOUNT INSTALLATION .....	12

# GENERAL PRECAUTIONS

## WARNING STATEMENTS

**! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT CQ MEDICAL AT +1 712-737-8688 OR INFO@CQMEDICAL.COM.**

- Maximum hanging weight of 22.5 kg (50 lbs) may be applied to the wall mount.
- Local installation requires use of properly sized and installed fasteners to support the rated load.
- Use of this product in a manner not specified in the IFU may be unsafe and is not recommended.
- Use of unapproved devices or attachments may be unsafe and is not recommended.

## PARTS INCLUDED

- Wall Mount plate (1)
- Wall Mount post (8)
- ¼-20 x 1 ¼ FH screws (8)
- Pivot locks (4)
- Pivot posts (4)
- Shoulder screws (4)
- Foam pad (1)

## TOOLS REQUIRED

- Phillips screwdriver and hex key

## WARNING LABELS & DESCRIPTIONS

Refer to [CQmedical.com](http://CQmedical.com) for a listing of symbols and their definitions.

# OPERATING INSTRUCTIONS

## SET-UP

### ACCESS™ SUPINE and SUPINE MR

1. Insert screws in the back of the Wall Mount plate, lining them up with the two center holes marked “D”. (Fig. 1)
2. Insert the Wall Mount posts on the front of the Wall Mount by tightening them onto the screws. Use a screwdriver to fully tighten the posts. (Fig. 2)
3. Refer to “Wall Mount Installation” section for installing the Wall Mount onto the wall.



Fig. 1

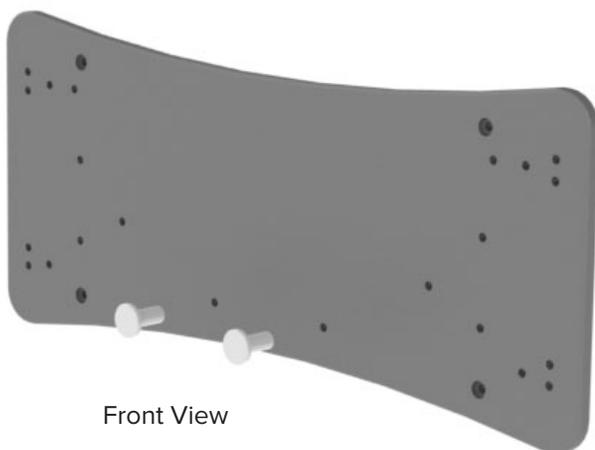


Fig. 2






# OPERATING INSTRUCTIONS

## SET-UP

### QUEST™ BOTTOM STOPPER

1. Insert screws in the back of the Wall Mount plate, lining them up with the holes marked “A”. (Fig. 3)
2. Insert the Wall Mount posts on the front of the Wall Mount by tightening them onto the screws. Use a screwdriver to fully tighten the posts. (Fig. 4)
3. Refer to “Wall Mount Installation” section for installing the Wall Mount onto the wall.

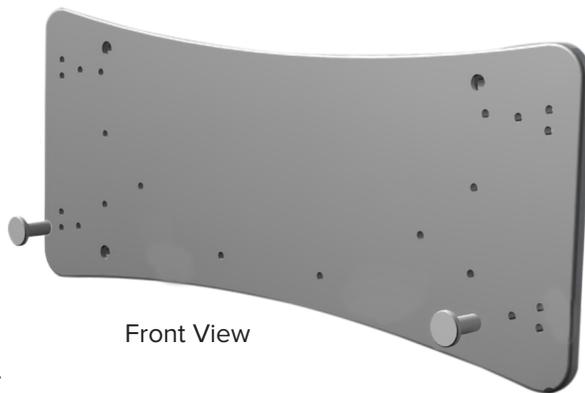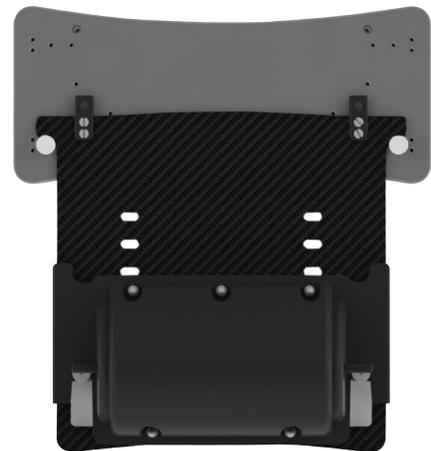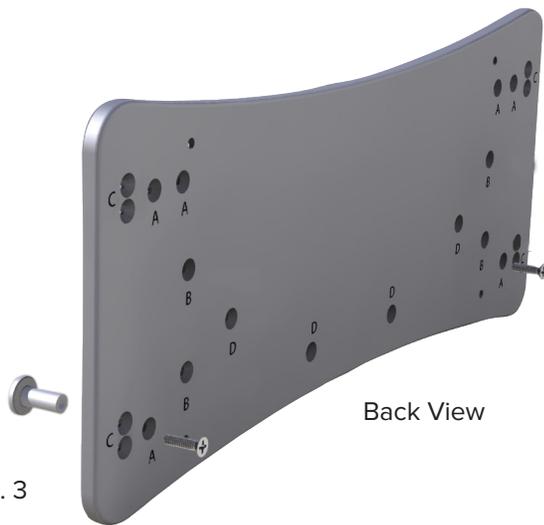

# OPERATING INSTRUCTIONS

## SET-UP

### QUEST™ BREASTBOARD

1. Insert screws in the back of the Wall Mount plate, lining them up with the holes marked “A”. (Fig. 5)
2. Insert the Wall Mount posts on the front of the Wall Mount by tightening them onto the screws. Use a screwdriver to fully tighten the posts. (Fig. 6)
3. Refer to “Wall Mount Installation” section for installing the Wall Mount onto the wall.



Fig. 5


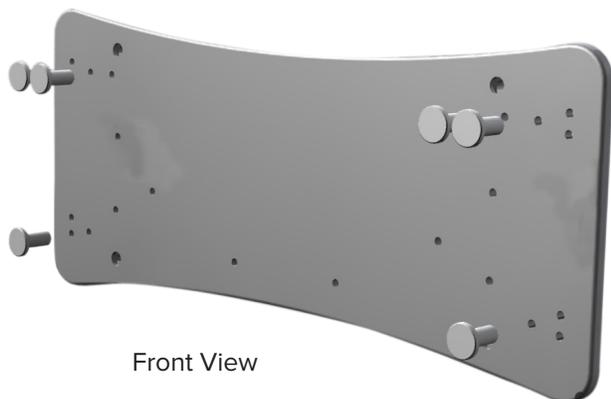


Fig. 6

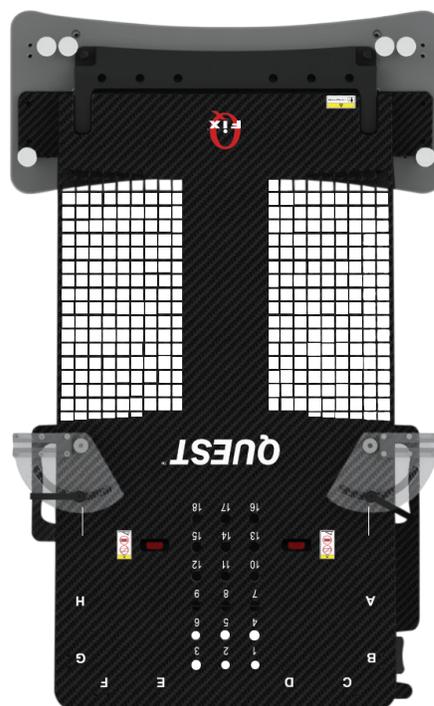

# OPERATING INSTRUCTIONS

## SET-UP

### ACCUFIX™ CANTILEVER BOARD

1. Insert screws in the back of the Wall Mount plate, lining them up with the holes marked “B and D”. (Fig. 7)
2. Insert the Wall Mount posts on the front of the Wall Mount by tightening them onto the screws. Use a screwdriver to fully tighten the posts. (Fig. 8)
3. Refer to “Wall Mount Installation” section for installing the Wall Mount onto the wall.

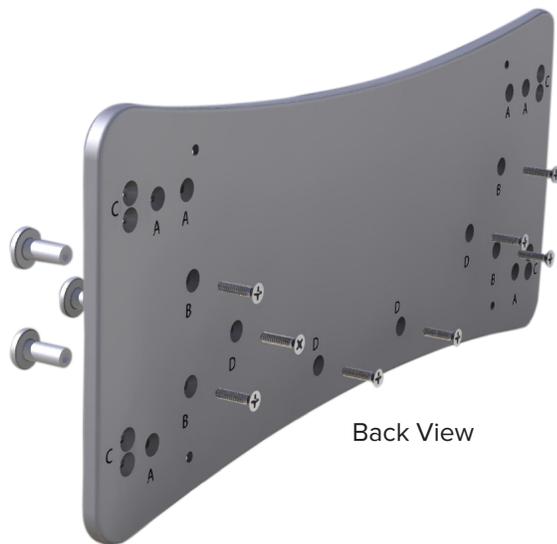


Fig. 7

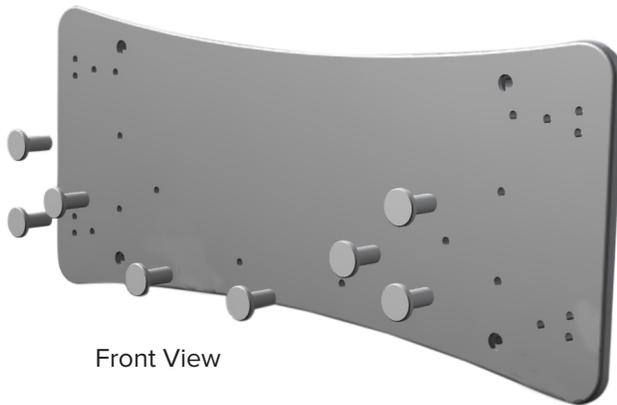


Fig. 8




# OPERATING INSTRUCTIONS

## SET-UP


### PORTRAIT™ HEAD AND NECK DEVICE

1. Insert screws in the back of the Wall Mount plate, lining them up with the further most holes on each side marked "A". (Fig. 9)
2. Insert the Wall Mount posts on the front of the Wall Mount by tightening them onto the screws. Use a screwdriver to fully tighten the posts. (Fig. 10)
3. Refer to "Wall Mount Installation" section for installing the Wall Mount onto the wall.

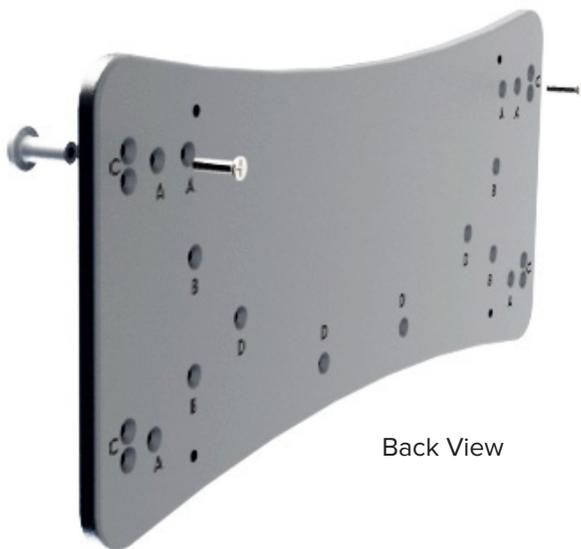

Back View

Fig. 9

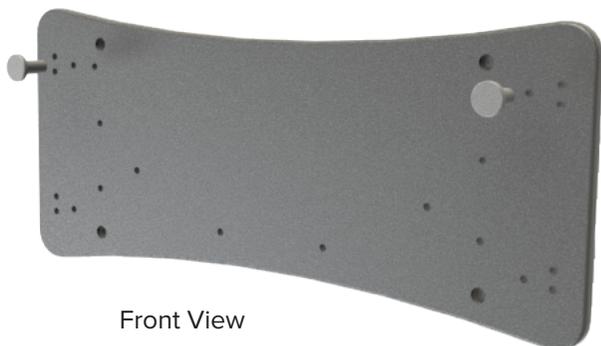

Front View

Fig. 10



Image of Portrait Head and Neck device mounted

# OPERATING INSTRUCTIONS

## SET-UP

### DEVICES WITH INDEXING NOTCHES

1. Insert screws in the back of the Wall Mount plate, lining them up with the two center holes marked “C”. (Fig. 11)
2. Line up the Pivot post with the corresponding screws and tighten down with a screwdriver in a clockwise motion. (Fig. 12)
3. If Device is less than 22 mm thick, remove backing paper from foam and install on the front of the wall mount. (Fig. 12)
  - 3a. If Device is greater than 22 mm thick, continue to step 4. (Fig. 13)
4. Refer to “Wall Mount Installation” section for installing the Wall Mount onto the wall.

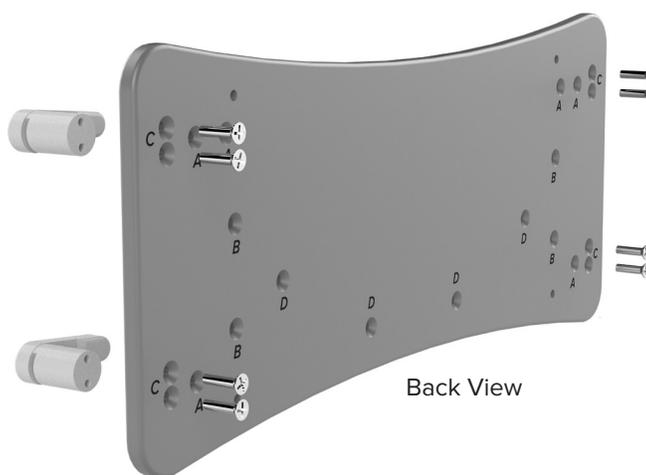


Fig. 11

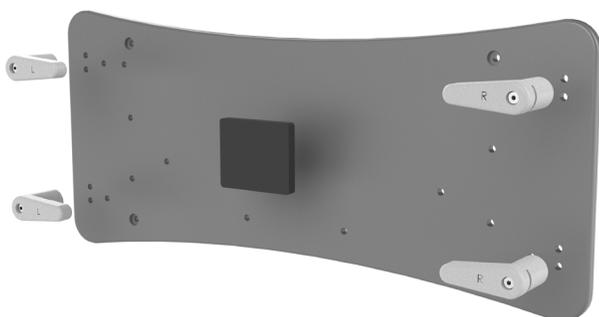


Fig. 12

Front View

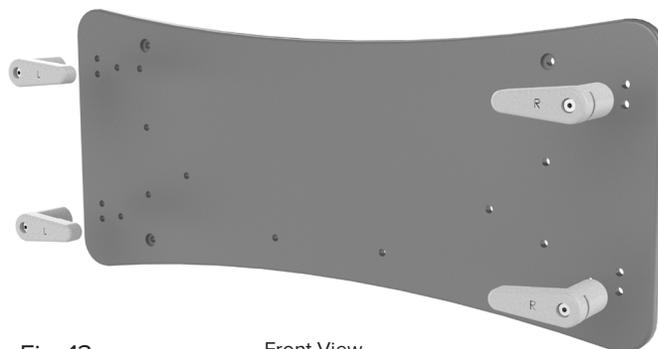


Fig. 13

Front View

# OPERATING INSTRUCTIONS

## SET-UP

### DEVICES WITH INDEXING NOTCHES

1. Once mounted onto the wall, line up indexing notches of the Device with the Pivot posts. Be sure that any part of the Device is not resting on the floor, and the pivot locks are in the unlocked position. (Fig. 14)
2. Turn the pivot locks in towards the Device counterclockwise to lock into place. (Fig. 15)
3. Turn the Pivot locks clockwise to unlock from the Wall Mount. (Fig. 14)

**! NOTE ! Top of Device should face Wall Mount plate during storage.**

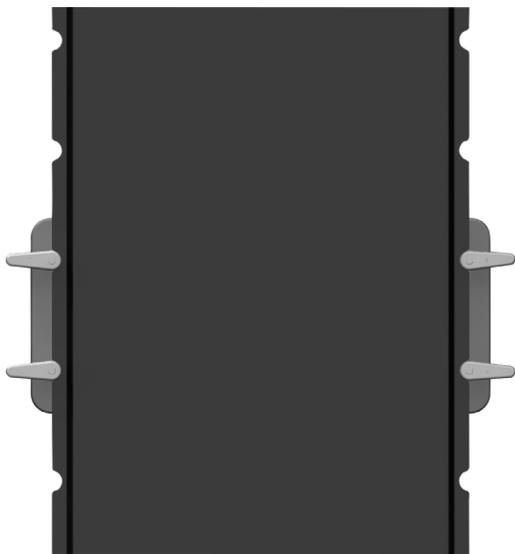


Fig. 14

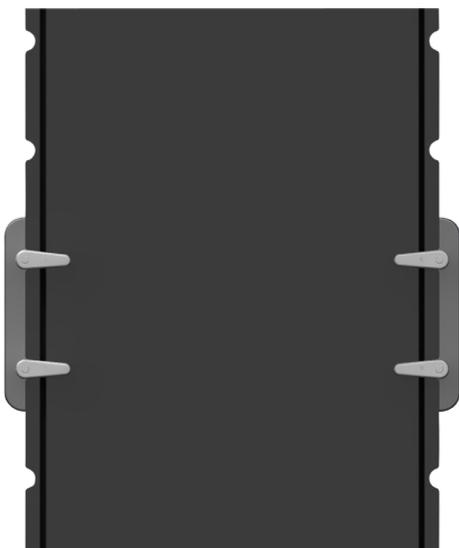


Fig. 15

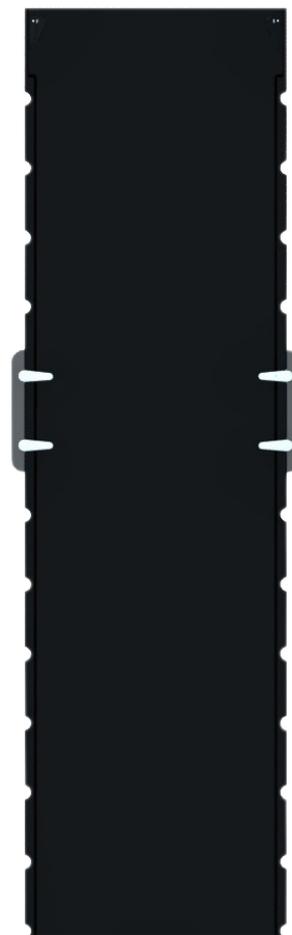


Image of Device mounted


# OPERATING INSTRUCTIONS

## WALL MOUNT INSTALLATION

### WALL MOUNT

**! NOTE !** Wall anchors and bolts must be chosen to support a load of 10 kg each (20 lbs). Follow local site codes and instructions for selection and installation details.

**! NOTE !** Qfix™ is not responsible for selection of anchors, bolts, or the wall mount installation.

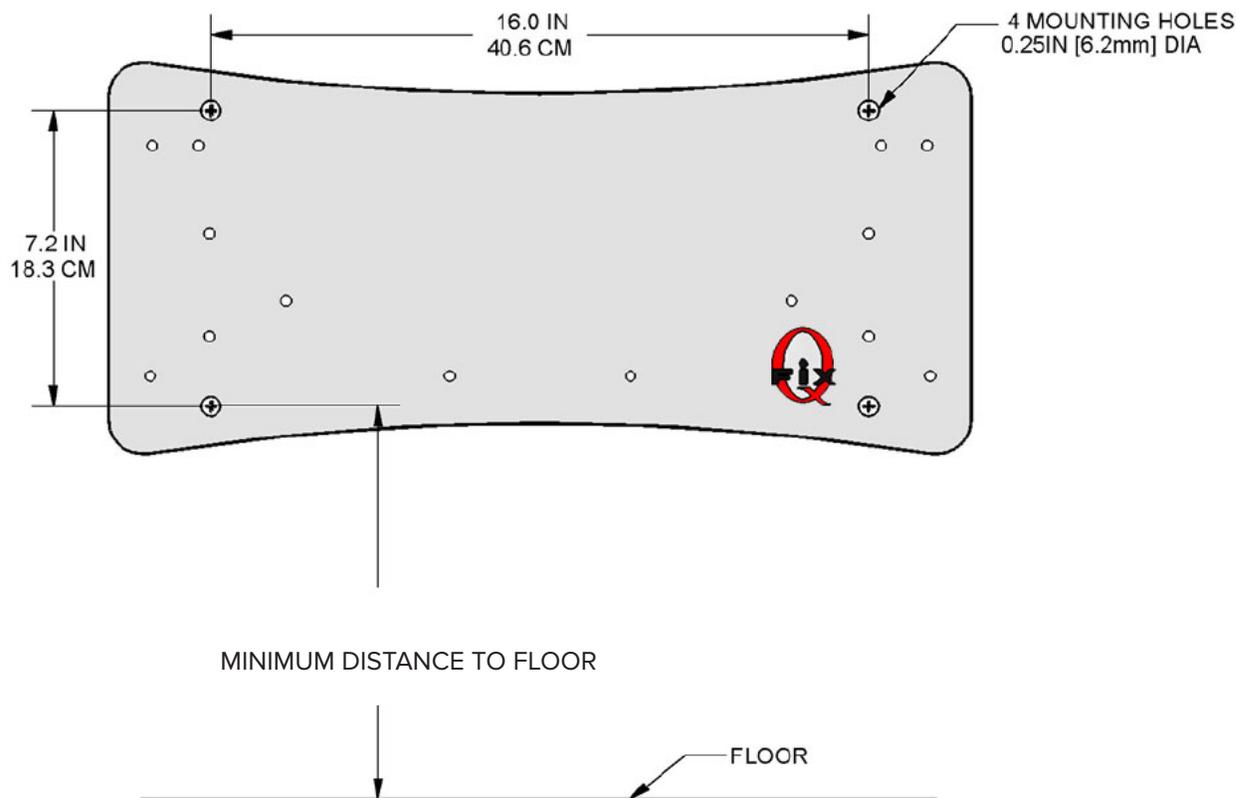

### MINIMUM DISTANCE TO FLOOR

- Access Supine (and MR)— 87 cm (34 in)
- QUEST Bottom Stopper— 80 cm (32 in)
- QUEST Breastboard— 80 cm (32 in)
- AccuFix Cantilever Board— 105 cm (42 in)
- Portrait (and MR) Head and Neck Device— 43 cm (17 in)
- Devices with Indexing Notches— At least half the height of the device









[www.CQmedical.com](http://www.CQmedical.com)

+1 712-737-8688 | +1 800-842-8688

[info@CQmedical.com](mailto:info@CQmedical.com)