




Siemens Eagle Mounting Latch Installation Manual

1 Tools/Equipment

Standard Allen Key Set


2 Procedure

- 2.1 From the underside of the CT overlay, remove the 6 – socket head cap screws (P/N 3000306) holding the latch to the overlay and remove the latch.

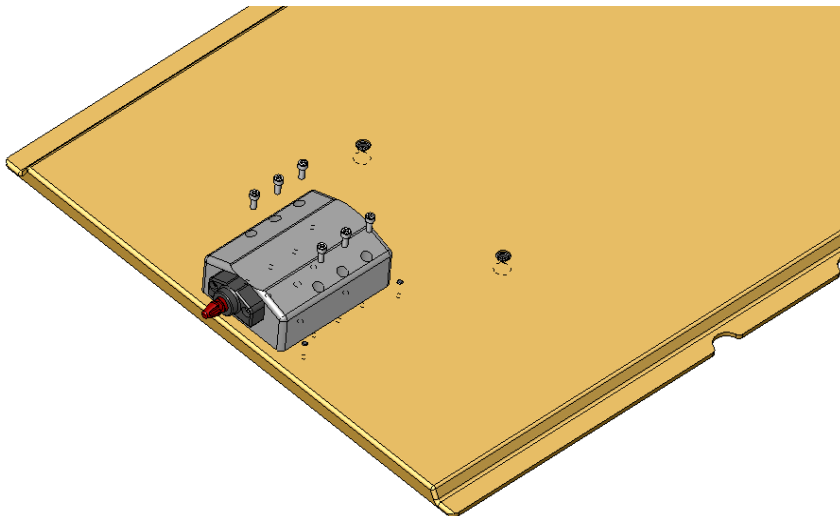




- 2.2 Remove the spring and the 4 socket head cap screws (P/N 3000306) holding the slide (P/N 2001252) in place. This will allow the removal of the slide handle (P/N 2000391).

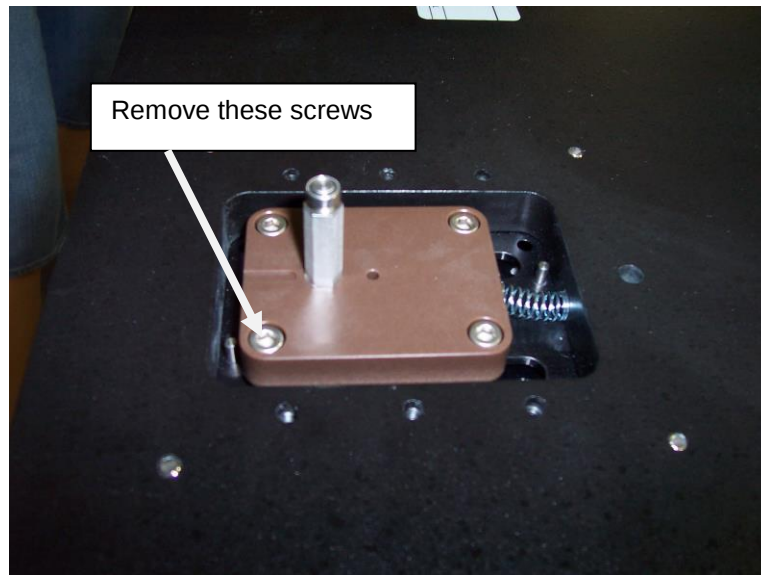




- 2.3 Remove and reinstall the cover plate P/N 2000183 and gasket P/N 2000388 using 4 Flat Head Cap Screws (P/N 3000116) as shown. When installing, use the screws to hold the gasket in place. NOTE: this should only be done if these parts are damaged.

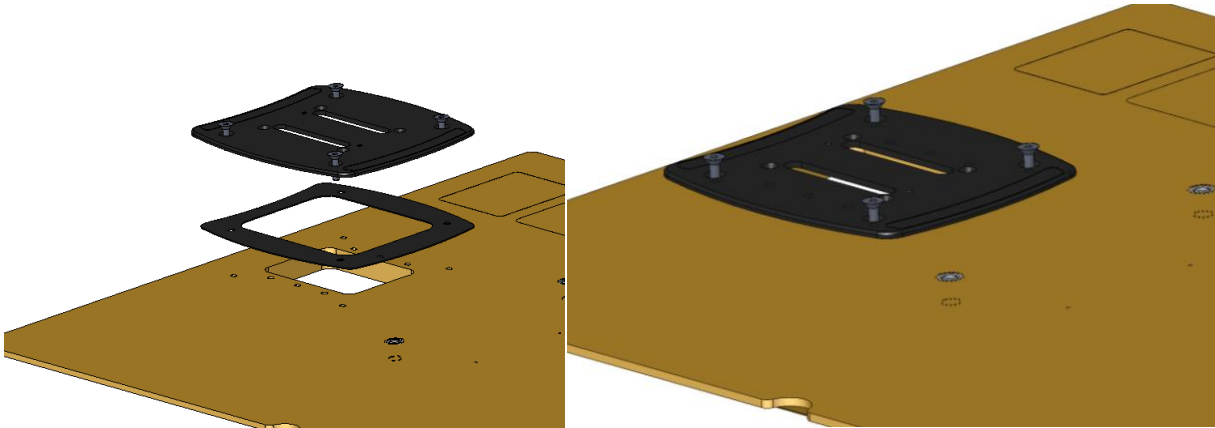

- 2.4 Install the handle P/N 2000391 (note orientation below) and P/N 2001252 from step using 4 SHCS (P/N 3000306). When tightening the screws make sure the handle slides freely front to back. Note the orientation of P/N 2001252 in the picture below. Add spring P/N 3000114 as shown and actuate the latch to ensure it moves freely and springs back into place.

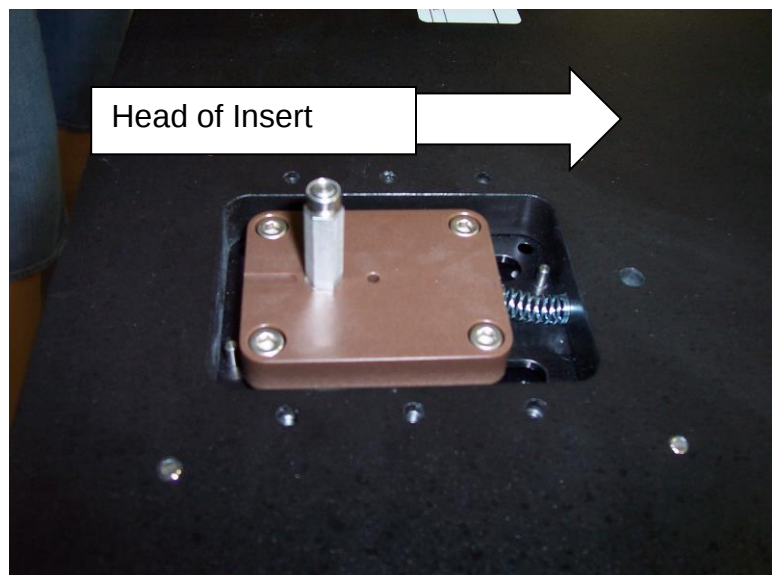



- 2.5 Install assembly P/N 2001238. Before installing make sure the pin is rotated to the orientation shown in the picture below. Install the 6 SHCS (P/N 3000306) and actuate the latch. The pin should rotate and retract as the latch is actuated.

