




Siemens Eagle Front Mounting Kit Installation Guide

1 Tools/Equipment

Metric Allen Key Set

2 Procedure

- 2.1 Remove the front guide blocks from the CT overlay by removing the 2 – M8 socket head cap screws from each block.
- 2.2 Install the front guide block P/N 2001153 using the plastic M8 socket head cap screws (P/N 3001064). Repeat this for the opposite side using front guide block 2001154

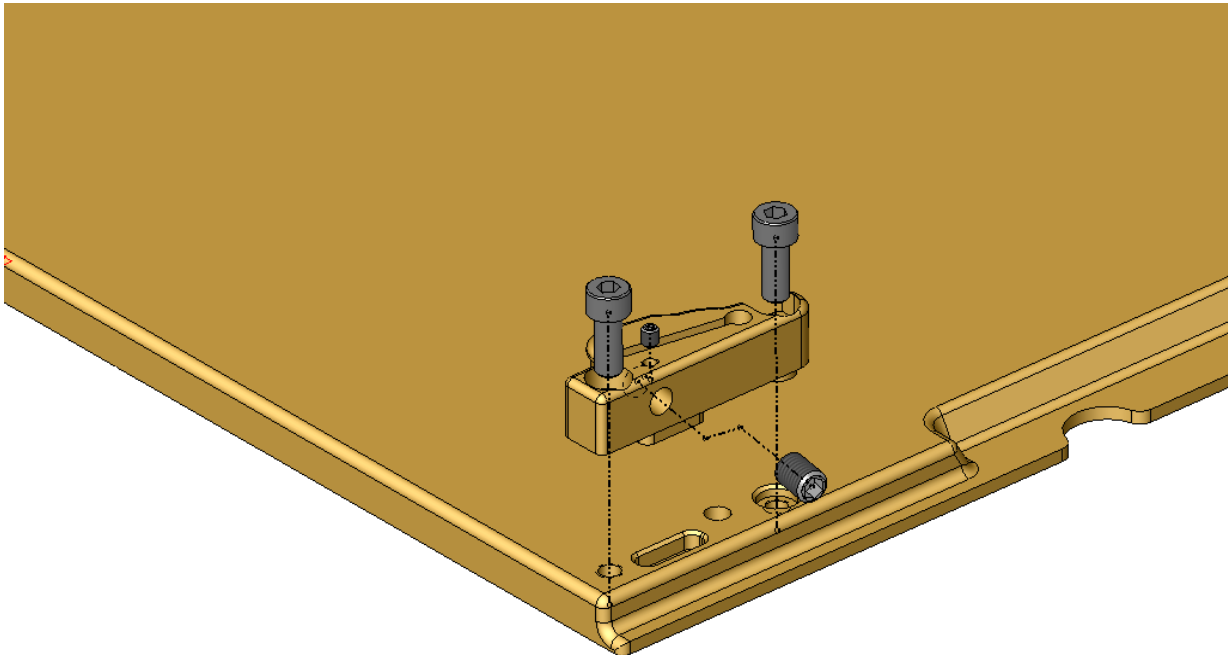



- 2.3 Adjust the front guide blocks by turning the adjustment set screw clockwise to tighten and counterclockwise to loosen so it engages front of the CT Overlay. The overlay should be centered on the cradle. These should be snug but still allow the overlay to be easily removed. Tighten the small set screw to lock the adjustment in place.

CAUTION: DO NOT OVER TIGHTEN this will cause the screw to strip.

