



positioning
patients for life.®

PRODUCT GUIDE & USER MANUAL

RT-4550WALL

Qfix CT Wall Mount






<div>EC</div> <div>REP</div>	Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	<div> Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 610-268-0585 www.Qfix.com</div>
------------------------------	---	--

TABLE OF CONTENTS

GENERAL PRECAUTIONS..... 4

 WARNING STATEMENTS 4

 PARTS INCLUDED..... 4

 TOOLS REQUIRED..... 4

OPERATING INSTRUCTIONS..... 5

 INSTALLATION..... 5

 OPERATION..... 6

GENERAL PRECAUTIONS

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6054 OR TECHSUPPORT@QFIX.COM.

- Maximum hanging weight of 22.5 kg (50 lbs) may be applied to the wall mount.
- Local installation requires use of properly sized and installed fasteners to support hold the rated load.
- Use of this product in a manner not specified in the User Manual may be unsafe and is not recommended.
- Use of unapproved devices or attachments may be unsafe and is not recommended.

WARNING LABELS & DESCRIPTIONS

Refer to Qfix.com for a listing of symbols and their definitions.

PARTS INCLUDED

- Mounting Board
- (8) Screws
- (4) Pivot Locks
- (4) Pivot Posts
- (4) Shoulder Screws

TOOLS REQUIRED


- Phillips screwdriver
- Allen wrench

OPERATING INSTRUCTIONS

INSTALLATION

CT WALL MOUNT

1. To install the CT Wall Mount, insert screws in the back of the Wall Mount, lining them up with the holes marked C, shown in the diagram of the back view.
2. After inserting screws in the back of the Wall Mount, line up the pivot post with the corresponding screws and tighten down with a screwdriver in a clockwise motion. Then, insert the pivot lock by screwing the shoulder screws into the pivot lock using an allen wrench to tighten.

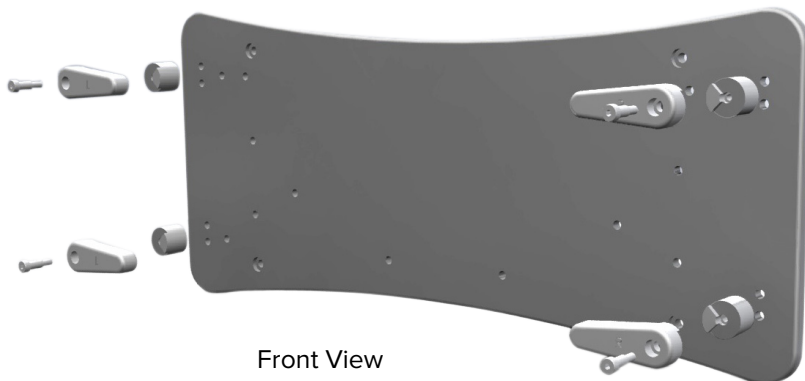



OPERATING INSTRUCTIONS

OPERATION

CT WALL MOUNT

! NOTE ! Top Of CT Overlay Should Face Wall Mount During Storage.


1. Line up indexing notches of the CT overlay with the pivot posts. Be sure that the CT latch is not resting on the floor, and the pivot locks are in the unlocked position. Turn the pivot locks in towards the overlay counterclockwise to lock into place.
2. To unlock the CT overlay, turn the pivot locks clockwise to unlock from the Wall Mount.

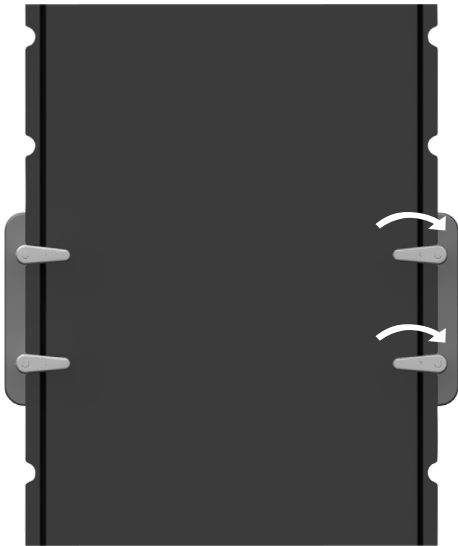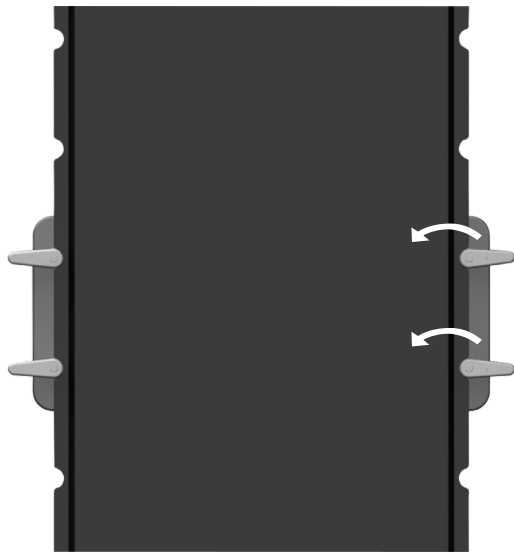

OPERATING INSTRUCTIONS

! WARNING ! WALL ANCHORS AND BOLTS MUST BE CHOSEN TO SUPPORT A LOAD 20 POUNDS EACH (10KG). FOLLOW LOCAL AND SITE CODES AND INSTRUCTIONS FOR SELECTION AND INSTALLATION DETAILS.

! NOTE ! Qfix is not responsible for selection of anchors, bolts, or the wall mount installation.

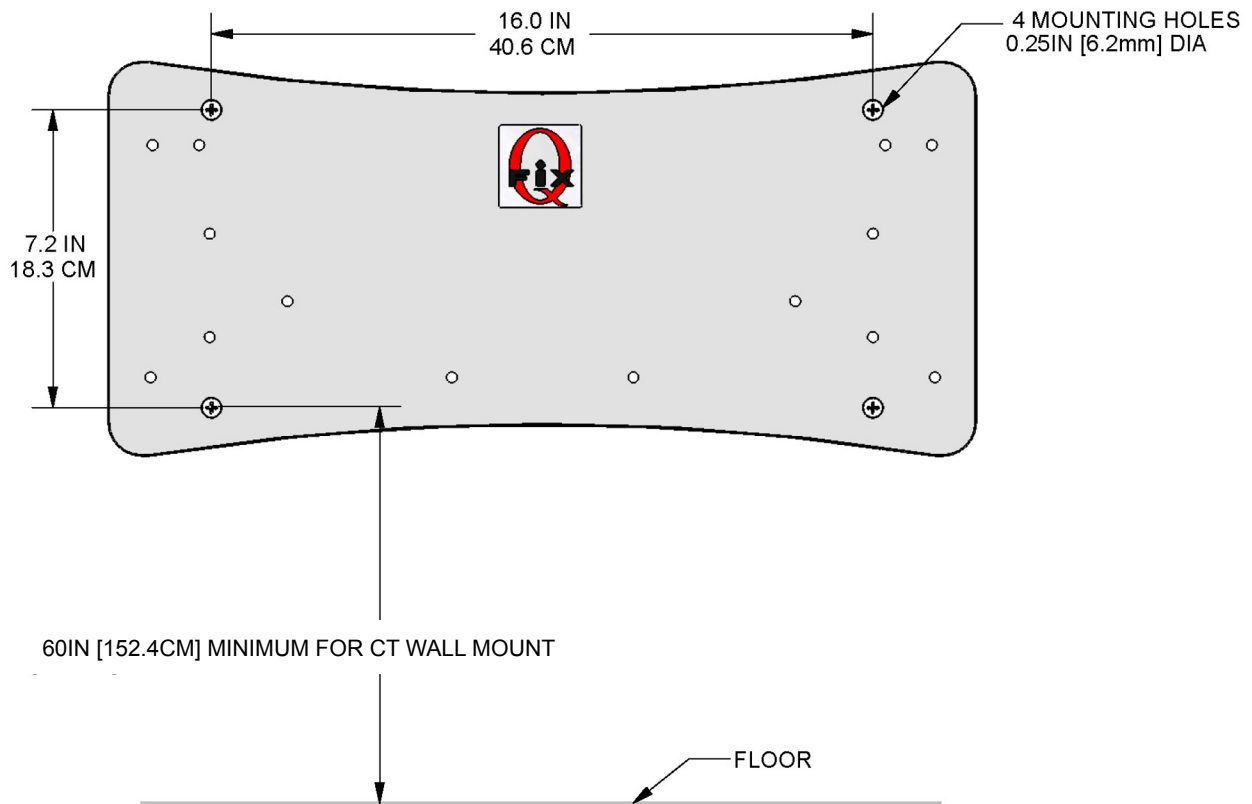

701796_1A ID



440 Church Road
Avondale, PA 19311 USA
www.Qfix.com

+1 610.268.0585 / 800.526.5247

+1 610.268.0588 / 800.831.8174

sales@Qfix.com