

# INSTALLATION MANUAL

## DoseMax™ Film Skin Replacement

**Tools/Equipment Required:** Masking Tape & Adhesive Cleaner

1. Peel the current film from an edge.
2. Prepare the surface by clearing all leftover adhesive with an adhesive remove.
3. Align the new film, making sure to cover the upper logo (Fig. 1). Each edge should be approximately 7.24 mm [.285"] from the outer edge of the device.
4. Place several pieces of tape along the edge to affix the film to the surface of the DoseMax (Fig. 2)

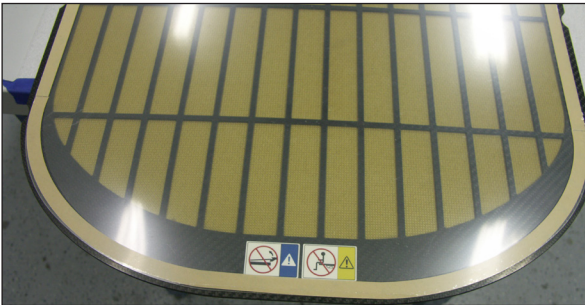


Fig. 1

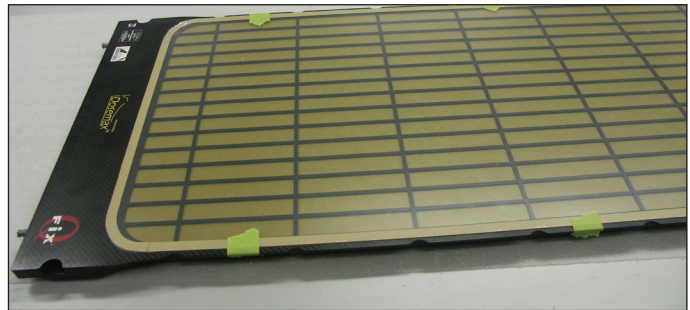


Fig. 2

5. Remove masking tape from superior half of film. Pull up and peel adhesive backing from film. Firmly apply to the surface of the device.
6. Remove tape and the remaining adhesive backing from anterior half of film. Firmly apply to the surface of the device.
7. Peel the protective covering from the film.

*DoseMax is a trademark of Qfix.*