



positioning  
patients for life.®

# PRODUCT GUIDE & USER MANUAL

RT-4551BAR-01

Sliding Locating Bar





Made in the USA by

Qfix

440 Church Rd, Avondale, Pennsylvania, USA

+1 610-268-0585 [www.Qfix.com](http://www.Qfix.com)

Exact® is a registered trademark of Varian Medical Systems.

# TABLE OF CONTENTS

GENERAL PRECAUTIONS..... 4

    WARNING STATEMENTS ..... 4

    SERIOUS INCIDENTS..... 4

    PATIENT TARGET GROUPS ..... 4

    INTENDED USERS ..... 4

    WARNING LABELS & DESCRIPTIONS ..... 4

OPERATING INSTRUCTIONS..... 5

    SET UP ..... 5

    REMOVAL..... 7

# GENERAL PRECAUTIONS

## WARNING STATEMENTS

**! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6054 OR TECHSUPPORT@QFIX.COM.**

**! WARNING ! THE SLIDING LOCATING BAR ALLOWS DEVICES TO BE INSTALLED IN AN OFFSET LATERAL POSITION. THIS OFFSET POSITION SHOULD BE TAKEN INTO ACCOUNT DURING PLANNING AND TREATMENT. ENSURE RED PINS ARE FULLY LOCKED BEFORE INSTALLING A DEVICE ONTO THE SLIDING LOCATING BAR.**

## SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both Qfix and your country's Competent Authority.

## PATIENT TARGET GROUPS

Patient's undergoing radiation therapy or diagnostic imaging procedures.

## INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

## WARNING LABELS & DESCRIPTIONS

Refer to [Qfix.com](http://Qfix.com) for a listing of symbols and their definitions.


# OPERATING INSTRUCTIONS

## SET UP

### LOCATING BAR INSTALLATION

The Sliding Locating Bar has two locating pins that match most standard positioning accessories. To attach, place either end of the Sliding Locating Bar in the appropriate Varian Exact® compatible index notches and snap into place.

Close up of positioning marking

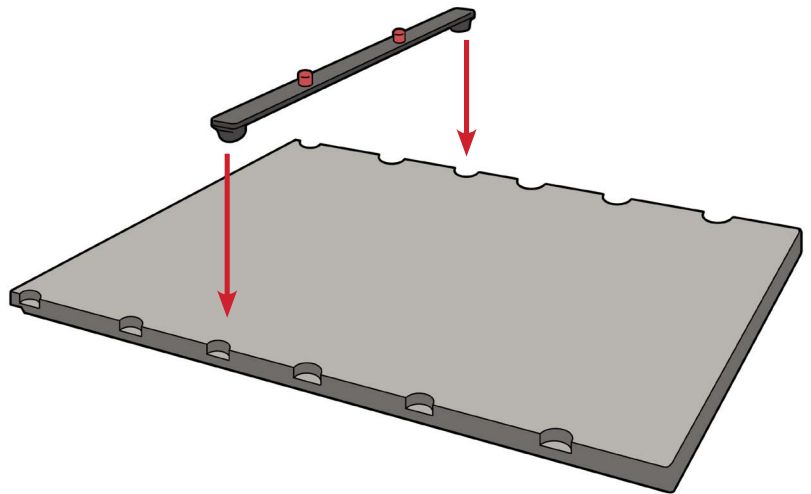

# OPERATING INSTRUCTIONS

## SET UP

### LOCATING BAR INSTALLATION

1. Loosen the red pins by twisting counter clockwise (Fig. 1).
2. Slide pins to the desired position (Fig. 2 & 3).
3. Lock pins by twisting clockwise (Fig 4).



Fig. 1



Fig. 2



Fig. 3



Fig. 4

# OPERATING INSTRUCTIONS

## REMOVAL

### LOCATING BAR REMOVAL

Pull up on either end of the Locating Bar.



440 Church Road  
Avondale, PA 19311 USA  
[www.Qfix.com](http://www.Qfix.com)

+1 610.268.0585 / 800.526.5247

+1 610.268.0588 / 800.831.8174

[sales@Qfix.com](mailto:sales@Qfix.com)