



positioning
patients for life.®

PRODUCT GUIDE & USER MANUAL

Silverman Head Supports

RT-4491R2, Set of 6, A–F

RT-4491R2-A, Size A

RT-4491R2-B, Size B

RT-4491R2-C, Size C

RT-4491R2-D, Size D

RT-4491R2-E, Size E

RT-4491R2-F, Size F

RT-4491-Q1, Silverman Q1

RT-4491-Q2, Silverman Q2

RT-4491-Q3-2G, Silverman Q3

RT-4491-Q4-2G, Silverman Q4

RT-4491R-M, Silverman “M”



WARNING STATEMENTS

! WARNING ! IF ANY PART OF THIS PRODUCT EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY.

! WARNING ! SILVERMAN HEAD SUPPORT SHAPE MAY VARY. IT IS RECOMMENDED THAT THE SAME SILVERMAN HEAD SUPPORT USED FOR SIMULATION SHOULD FOLLOW THE PATIENT THROUGHOUT THE ENTIRE COURSE OF TREATMENT.

OPERATING INSTRUCTIONS

SET-UP

1. Select a Silverman Head Support that is comfortable for the patient and achieves the desired head position.
2. Secure the Silverman Head Support by placing the peg holes over the pegs in the device that are designed to hold the head supports (Fig. 1).
3. Once the patient is positioned properly on the Silverman, begin forming the Moldcare® and thermoplastic mask using their respective instructions for use. .

REMOVAL

Lift Silverman Head Support off of pegs on device to remove (Fig. 2).

MAINTENANCE

CLEANING

! NOTE ! The Silverman Head Support is reusable and can be cleaned and reused after each course of treatment.

The Silverman Head Support can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface.

DISINFECTING

The following cleaning materials have been tested and found to be appropriate for cleaning the Silverman Head Support. Please refer to specific instructions from the cleaning agent manufacturer.

- Water
- 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- Cidex® 2.4% Activated Dialdehyde Solution
- Soap and Water

Moldcare® is a registered trademark of ALCARE CO, LTD.

Cidex® is a registered trademark of Johnson & Johnson.

Clorox® is a registered trademark of The Clorox Company.



Figure 1



Figure 2